## Acidified Medium

**For laboratory procedures only; other uses must be qualified by the end user.**

Product Description	REF Number	Unit Size
Acidified Medium	ART-4013-A	6 x 1.0 mL

### INTENDED USE

This product is intended to circumvent excessive hardening of the zona pellucida by dissolving it with an acidified medium.

### PRODUCT DESCRIPTION

The zona pellucida, an acellular structure surrounding the cells of the preimplantation embryo, can be dissolved at an acidic pH of 2.2 to 2.8. It has been suggested that one of the detrimental effects of in vitro culture is to cause an excessive hardening of the zona pellucida that may impede or slow the required emergence of the blastocyst-stage embryo from the zona pellucida prior to implantation. To alleviate this problem, a portion of the zona pellucida can be dissolved by carefully directing a stream of acidified medium over a specific area using a small micropipette (Cohen, Malter, Talansky & Grifo. *Micromanipulation of Human Gametes and Embryos*. New York, NY: Raven Press; 1992).

This product contains 10 mg/L of gentamicin, an aminoglycoside antibiotic.

### PRECAUTIONS AND WARNINGS

Do not use medium that shows evidence of particulate matter or cloudiness.

**Single Use:** To avoid problems with contamination, handle using aseptic techniques and discard any excess product that remains in the bottle or vial after procedure is completed.

Reproductive media products are intended for single use only. Re-use of reproductive media may result in using a product past its labeled expiration date or increase the risk of microbial contamination in a subsequent procedure if the practitioner fails to utilize adequate aseptic techniques.

Use of expired or microbial contaminated product may result in suboptimal conditions to promote fertilization and/or embryo quality during in-vitro culture. These conditions may result in the failure of the embryo to develop properly or to implant, potentially leading to a failed assisted reproductive procedure.

**Note:** Embryo is considered a general term. More precisely, SAGE considers the period of time initiating when a single diploid cell results from the fusion of male and female genome resulting in zygote formation with subsequent development from repeated mitotic divisions forming a solid mass or morula (typically day 4-5) and after which a fluid-filled cavity develops resulting in blastocyst formation (typically day 5-6) ending with embryo implantation that begins the end of the first week and is completed by the end of the second week post conception.

**Caution:** U.S. Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

This product contains the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.

### QUALITY ASSURANCE

Eight-cell MEA tested and passed with 80% or greater blastocyst. USP Endotoxin tested and passed with <1 EU/mL.

A Certificate of Analysis is available for this product.

### DIRECTIONS FOR USE

Embryos should be clamped and held with a microholding pipette in a drop of Quinn's Advantage® Medium with HEPES (REF #1023/1024) containing 5 mg/mL HSA (REF #3001/3003) under Sterile Oil for Tissue Culture (REF #4008). A micropipette filled with acidified medium is lowered into the drop and the acidified medium is gently expelled over a small 35-to 50-µm area while holding the needle tip immediately next to the zona pellucida. The size of the hole should be about 1/2 to 2/3 the length of an 8-cell blastomere. Use small circular motions to avoid excess acid in a single region. Immediately cease expulsion of acidic medium when the inside of the zona pellucida is pierced and move the embryo away from the acidic medium to the opposite side of the drop.

**Each laboratory should make its own determination of which medium to use for each particular procedure.**

Information on specific aspects of IVF, embryo culture, and cryopreservation is available in our Product Catalog.

### STORAGE INSTRUCTIONS AND STABILITY

Store unopened containers at or below –20 °C upon receipt. Warm to incubator (37 °C) temperature prior to use. Do not refreeze or expose to temperatures greater than 39 °C. The product is stable until the expiration date shown on the label.

- Remove desired volume of product using aseptic procedures.
- Once removed, do not return any volume of product to the original container.
- Do not use if the product becomes discolored, cloudy, turbid, or shows any evidence of microbial contamination.

### COMPLEMENTARY PRODUCTS

ART-1023/1024 Quinn's Advantage® Medium with HEPES

ART-3001/3003 Human Serum Albumin 100 mg/mL in normal saline

ART-4008P Oil for Tissue Culture

### RELATED PRODUCTS

SAGE In Vitro Fertilization™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE Support Line.

Quinn's Advantage® is a registered trademark of CooperSurgical, Inc.

**Call the SAGE SUPPORT LINE:  
In the U.S.: (800) 243-2974  
International: (203) 601-9818**

### EXPLANATION OF SYMBOLS

	Catalog Number
	Batch Number
	Use By (year, month, day)
	Do Not Reuse
	Upper limit of temperature
	Aseptic Technique Sterilization Membrane Filtered (SAL 10 <sup>-3</sup> )

**ATTENTION:** See instructions for use.

Authorized Representative in the European Community.

Product conforms to the Medical Device Directive 93/42/EEC

Manufacturer

U.S. Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

In Vitro Fertilization  
*a CooperSurgical Company*

SAGE In Vitro Fertilization, Inc.  
a CooperSurgical Company  
95 Corporate Drive  
Trumbull, CT 06611 USA

Leisegang Feinmechanik GmbH  
Leibnizstraße 32  
D-10625, Berlin GERMANY  
www.coopersurgical.com

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