Quinn's Advantage® Blastocyst Medium

**For laboratory procedures only;
other uses must be qualified by the end user.**

Product Description	REF Number	Unit Size
Quinn's Advantage® Fertilization (HTF) Medium	ART-1029	50 mL

INTENDED USE

This product is intended for in vitro procedures involving the culture of human embryos from the compaction phase on Day 3 of development to the blastocyst stage.

PRODUCT DESCRIPTION

This medium is a modification of Human Tubal Fluid (HTF) that was first described by Quinn and colleagues (Fertil Steril. 1984;41:202, 1985;44:493.). Modifications to the original formulation include the addition of an elevated concentration of glucose, selected nonessential and essential amino acids, alanyl-glutamine as a stable source of glutamine, MEM vitamin mix, increased levels of magnesium, and the presence of lactate in the form of calcium lactate. This medium contains no EDTA. This product contains 10 mg/L of gentamicin, an aminoglycoside antibiotic.

PRECAUTIONS AND WARNINGS

Do not use medium that shows evidence of particulate matter, cloudiness, or is not rose colored.

Single Use: To avoid problems with contamination, handle using aseptic techniques and discard any excess product that remains in the bottle or vial after procedure is completed.

Reproductive media products are intended for single use only. Re-use of reproductive media may result in using a product past its labeled expiration date or increase the risk of microbial contamination in a subsequent procedure if the practitioner fails to utilize adequate aseptic techniques.

Use of expired or microbial contaminated product may result in suboptimal conditions to promote fertilization and/or embryo quality during in-vitro culture. These conditions may result in the failure of the embryo to develop properly or to implant, potentially leading to a failed assisted reproductive procedure.

Note: Embryo is considered a general term. More precisely, SAGE considers the period of time initiating when a single diploid cell results from the fusion of male and female genome resulting in zygote formation with subsequent development from repeated mitotic divisions forming a solid mass or morula (typically day 4-5) and after which a fluid-filled cavity develops resulting in blastocyst formation (typically day 5-6) ending with embryo implantation that begins the end of the first week and is completed by the end of the second week post conception.

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).

This product contains the antibiotic gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic.

QUALITY ASSURANCE

Two-cell MEA tested and passed with 80% or greater blastocyst. USP Endotoxin tested and passed with <1 EU/mL.

A Certificate of Analysis is available for this product.

DIRECTIONS FOR USE

This is the preferred medium for the culture of human embryos from Day 3 of development to the blastocyst stage on Day 5/6. We currently recommend the use of Quinn's Advantage® Fertilization Medium (REF #1020/1021) for fertilization of oocytes, and Quinn's Advantage® Cleavage Medium (REF #1026/1027) for culture from Day 1 to Day 3. The media should be supplemented with an exogenous protein source such as Human Serum Albumin (REF #3001/3003) or Serum Protein Substitute (REF #3010/3011).

Each laboratory should make its own determination of which medium to use for each particular procedure.

Information on specific aspects of IVF, embryo culture, and cryopreservation is available in our Product Catalog.

STORAGE INSTRUCTIONS AND STABILITY

Store unopened containers refrigerated at 2°C to 8°C. Warm to incubator (37°C) temperature and equilibrate with desired atmosphere containing 5% CO₂ prior to use. Do not freeze or expose to temperatures greater than 39°C.

In order to ensure optimal performance, SAGE Media™ strongly recommends measuring the pH of the medium under laboratory working conditions and adjusting the level of CO₂ used to attain the desired pH range for optimal embryo development. The desired pH range for Quinn's Advantage® Blastocyst Medium is 7.3 ± 0.1. The product is stable until the expiration date shown on the label.

- Remove desired volume of product using aseptic procedures.
- Once removed, do not return any volume of product to the original container.
- Do not use if the product becomes discolored, cloudy, turbid, or shows any evidence of microbial contamination.

RELATED PRODUCTS

ART-1020/1021 Quinn's Advantage® Fertilization (HTF) Medium

ART-1026/1027 Quinn's Advantage® Cleavage Medium

ART-3001/3003 Human Serum Albumin 100 mg/mL in normal saline

ART-3010/3011 Quinn's Advantage® SPS Serum Protein Substitute

SAGE In Vitro Fertilization™ has a full line of products for the Reproductive Medicine Specialist. Please call or write for specific information or to receive a copy of our current catalog. For technical questions, or to reach our Customer Service Department, call the SAGE Support Line.

Quinn's Advantage® is a registered trademark of CooperSurgical, Inc.

Call the SAGE SUPPORT LINE:

In the U.S.: (800) 243-2974

International: (203) 601-9818

EXPLANATION OF SYMBOLS

Catalog Number

Batch Number

Use By (year, month, day)

Do Not Reuse

Temperature Limitation

Aseptic Technique Sterilization
Membrane Filtered (SAL 10⁻³)

ATTENTION:

See instructions for use.

Authorized Representative in the
European Community.

Product conforms to the Medical Device
Directive 93/42/EEC

Manufacturer

Rx ONLY

U.S. Federal law restricts this device
to sale by or on the order of a physician
(or properly licensed practitioner).

SAGE In Vitro Fertilization, Inc.
a CooperSurgical Company
95 Corporate Drive
Trumbull, CT 06611 USA

Leisegang Feinmechanik GmbH
Leibnizstraße 32
D-10625, Berlin GERMANY
www.coopersurgical.com

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