Arch Koh-Efficient™

Vaginal Fornices Delineator & Colpo-Pneumo Occluder™
for use with the Advincula Arch™ Uterine Manipulator Handle

Koh-Efficient™ for Electrocautery

KC-ARCH-25 (2.5 cm) • KC-ARCH-30 (3.0 cm)

KC-ARCH-35 (3.5 cm) • KC-ARCH-40 (4.0 cm)

Koh-Efficient™ for Harmonic Scalpel and Laser

KCS-ARCH-25 (2.5 cm) • KCS-ARCH-30 (3.0 cm)

KCS-ARCH-35 (3.5 cm) • KCS-ARCH-40 (4.0 cm)

Directions For Use (English)

STERILE • SINGLE-USE


CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

CE0086

DEVICE DESCRIPTION

The Arch Koh-Efficient™ is a sterile, single-use vaginal fornices delineator with an integrated Colpo-Pneumo Occluder™. It is made of medical grade plastics with an Ultem® polymeric rigid KOH Cup™. Harmonic Scalpel/Laser compatible versions (KCS-ARCH) are equipped with a stainless steel distal surface.

The Arch Koh-Efficient (Koh-Efficient) is compatible with the Advincula Arch™ Uterine Manipulator Handle (Handle) and the RUMI® Uterine Manipulator Tip (Tip). These three products, used in conjunction with the Cervical Sizer (Sizer), enable unobstructed placement of a standard Tip into the uterus. Once in place, the Koh-Efficient is advanced along the shaft of the Handle and locks the KOH Cup into place around the cervix.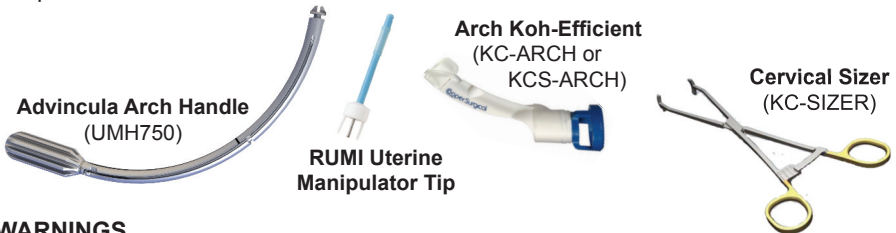

WARNINGS

- Do not use the Koh-Efficient if it has been opened or damaged.
- The integrated Colpo-Pneumo Occluder is designed to accommodate no more than 180 cc of inflation.
- The Koh-Efficient is single-use only, and should NEVER be reused.
- Using an incorrect size KOH Cup could result in patient injury.
- The Ultem KOH Cup (KC-ARCH) is not intended to be used with Harmonic or other types of ultrasonic or laser energy sources.
- The stainless steel KOH Cup (KCS-ARCH) is not intended to be used with electrocautery. Use of electrocautery may result in inadvertent injury to adjacent structures.

CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician
- The Arch Koh-Efficient is to be used **ONLY** with the Advincula Arch™ Handle (UMH750).

INTENDED USE / INDICATIONS FOR USE

The Koh-Efficient is a combination KOH Cup™ and Colpo-Pneumo Occluder™ that is indicated for use in laparoscopic procedures where it is desirable to locate the position of the vaginal fornices, typically during a total laparoscopic hysterectomy or supra-cervical laparoscopic hysterectomy.


CONTRAINDICATIONS

The Koh-Efficient should not be used in patients who are pregnant or who are suspected of being pregnant, in patients who have an IUD in place, in patients with suspected pelvic infections or in cases where the surgeon deems it inadvisable and finds it difficult to insert the silicone tip into the cervix or uterus.

PRECAUTIONS



Always confirm that the Koh-Efficient is locked in place prior to starting the colpotomy.

DIRECTIONS FOR USE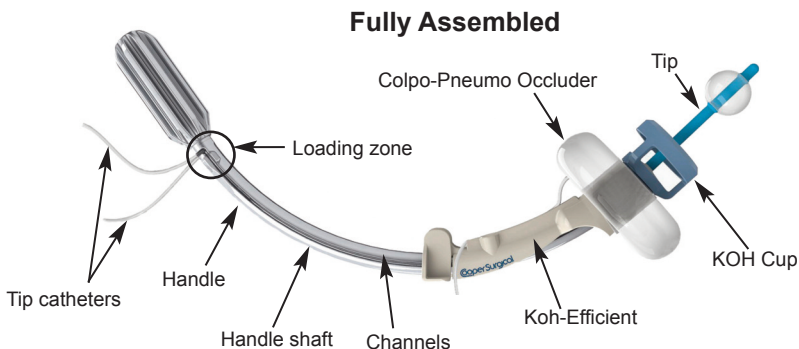


For complete information on how to prepare Arch Koh-Efficient System, please refer to the Directions for Use for the Advincula Arch Handle (UMH750), the RUMI Uterine Manipulator Tips and the Cervical Sizer (KC-SIZER).

1. Prepare the Handle for use by following the Handle's Directions for Use.
2. Attach the proper size Tip to the Handle. **Press the Tip catheters completely into the channel positions along the Handle shaft.** Confirm that the tubes protrude out of the channels at the Loading zone.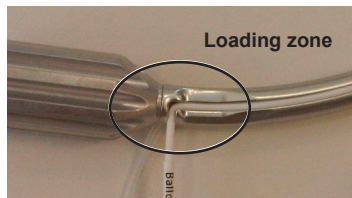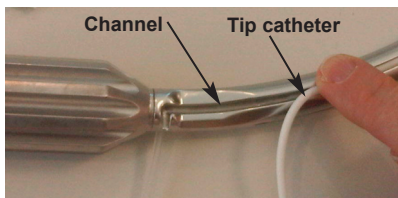


3. Using the Cervical Sizer, determine the proper size Koh-Efficient.


4. Slide the Koh-Efficient over the Tip and down the Handle shaft.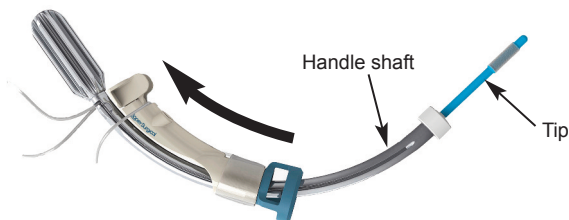

5. Position the Koh-Efficient's Pinch clamp over the Loading zone and press firmly on the Thumb pad to "snap" into place.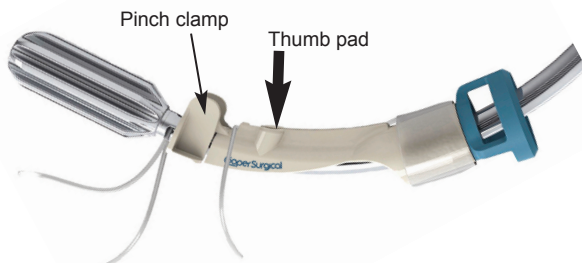

6. Test inflate the Colpo-Pneumo Occluder™ with 20 to 60 cc of sterile saline, then deflate prior to patient insertion.
7. Insert the Tip into the uterus and inflate per the Tip's Directions for Use.
8. Press on the Push pads to slide the Koh-Efficient up the Handle until it locks into place. *Note: The last 1/4 of an inch of travel will have increased resistance.*

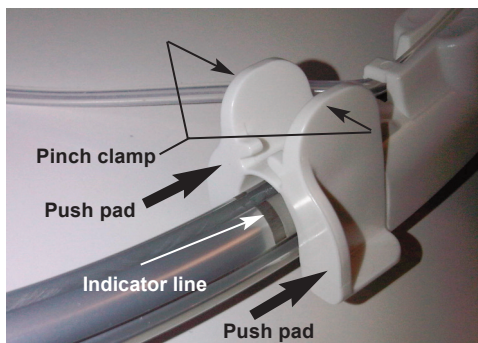9. Confirm the locked position by pulling back on the Push pad and visually verifying the indicator line on the shaft.
10. Confirm that the distal end of the KOH Cup™ resides in the fornices.
Note: If necessary, slide the Koh-Efficient back by squeezing the Pinch clamp to release.
11. Just prior to making colpotomy incisions, inflate the Colpo-Pneumo Occluder balloon with 60 to 120 cc of sterile saline or water (do not use air).
12. After the procedure, dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

EXPLANATION OF SYMBOLS

REF Reorder Number

LOT Batch Code

 Use By

 **ATTENTION:** See instructions for use.

STERILE EO Ethylene Oxide Sterilization

 Latex Free

 Do Not Reuse

CE 0086 Product conforms to the Medical Device Directive 93/42/EEC

EC REP Authorized Representative in the European Community.

The marks bearing the symbols “TM” and “®” are trademarks and registered trademarks of CooperSurgical, Inc.

Ultem® is a registered trademark of SABIC Innovative Plastics.

CooperSurgical is a registered trademark of CooperSurgical, Inc

© 2011 CooperSurgical, Inc. All rights reserved.

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As marcas que apresentam os símbolos “TM” e “®” são marcas comerciais e marcas registadas da CooperSurgical, Inc.
CooperSurgical é uma marca registada da CooperSurgical, Inc.
© 2011 CooperSurgical, Inc. Todos os direitos reservados.
Leisegang Feinmechanik GmbH