ACKRAD LABS
A CooperSurgical Company

BRONCHITRAC® “L” SELECTIVE LEFT BRONCHIAL SUCTIONING SYSTEM

STERILE R

DO NOT REUSE

INSTRUCTIONS English

Manufactured By:

CooperSurgical Trumbull, CT 06611 • (800) 243-2974 • (203) 601-5200

European Authorized Representative:

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003-149-01-03 Rev A

1. Remove from tray and put on gloves.
2. Remove the BRONCHITRAC “L” catheter and tear off the band.
3. Lubricate the distal end of the catheter with a suitable lubricant.
4. Insert the distal end of BRONCHITRAC “L” into the ET or tracheostomy tube and advance it. The catheter will automatically emerge pointing toward the patient’s left main stem bronchus.
5. Continue to advance the catheter into the left main stem bronchus utilizing your normal suction technique. The catheter has a hub with a slide cover. The slide can be used in the closed (full suction) or slightly open (partial suction) position.
6. Flush the catheter in the usual way, using the tray to hold the flush solution.

CAUTION: Do not force the BRONCHITRAC “L” catheter in a forward or rotational movement. If unexplained resistance is encountered check for the following conditions:

- A. The patient may BRONCHITRAC
- B. The ET tube may enter the left main carina. **EXTRE ENDOTRACHE**
- C. Catheter tip may the catheter 2-3

CAUTION: Federal

ACKRAD LABS
A CooperSurgical Company

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