

Extension Tubing and Channel Adaptor

This Extension Tubing (REF #7910-3003) has a female Luer at the proximal end for the connection to the output connector of the pump cassette on the Endoscopy Lavage Pump Model ENDO 100™ (REF #7910-3000).

The distal end male fitting will typically attach to the following devices—

- 1 Endoscopic Washing Catheters
- 2 Endoscopic Special Connecting Tubes (usually 4 to 6 inch lengths) for using the built-in water jet channel

and

- 3 Channel Adaptor (REF #7910-3005). The Adaptor, in combination with the Extension Tubing is used for large volume washing through the working channel of the endoscope. The Adaptor is connected to the distal tip of the Extension Tubing. By inserting the Adaptor through the rubber inlet seal of the endoscope, it becomes very convenient to quickly deliver the lavage from the pump to the working channel of the endoscope. This set-up also offers the advantage of leaving the working channel of the endoscope unobstructed for maximum suction capability.

For further assistance call Technical Service at

ERBE USA, Inc. at 800-778-3723

Part #35424 ▪ Revised 9/01