


CooperSurgical® F9 Fetal Monitor

Provides industry-leading accuracy, ease of use and optimal patient interface

Benefits of CooperSurgical F9 Fetal Monitor include:

- Large 12-inch touch-screen display facilitates operator interface
- Real-time color monitor provides instantaneous feedback to staff and facilitates patient interaction
- A 12-crystal, wide-beam, 1 MHz waterproof transducer yields industry leading signal acquisition and maintenance of FHR, even in challenging situations
- Configured to monitor twins for cost efficiency
- Signal Overlap Verification provides visual and audible indications when synchronous heart-rate signals are detected
- 24-hour data storage facilitates data transfer or reprinting
- Optional CooperSurgical Insight™ software permits transfer of patient data to a personal computer for study review and record storage
- EMR compatibility with CooperSurgical Insight Software

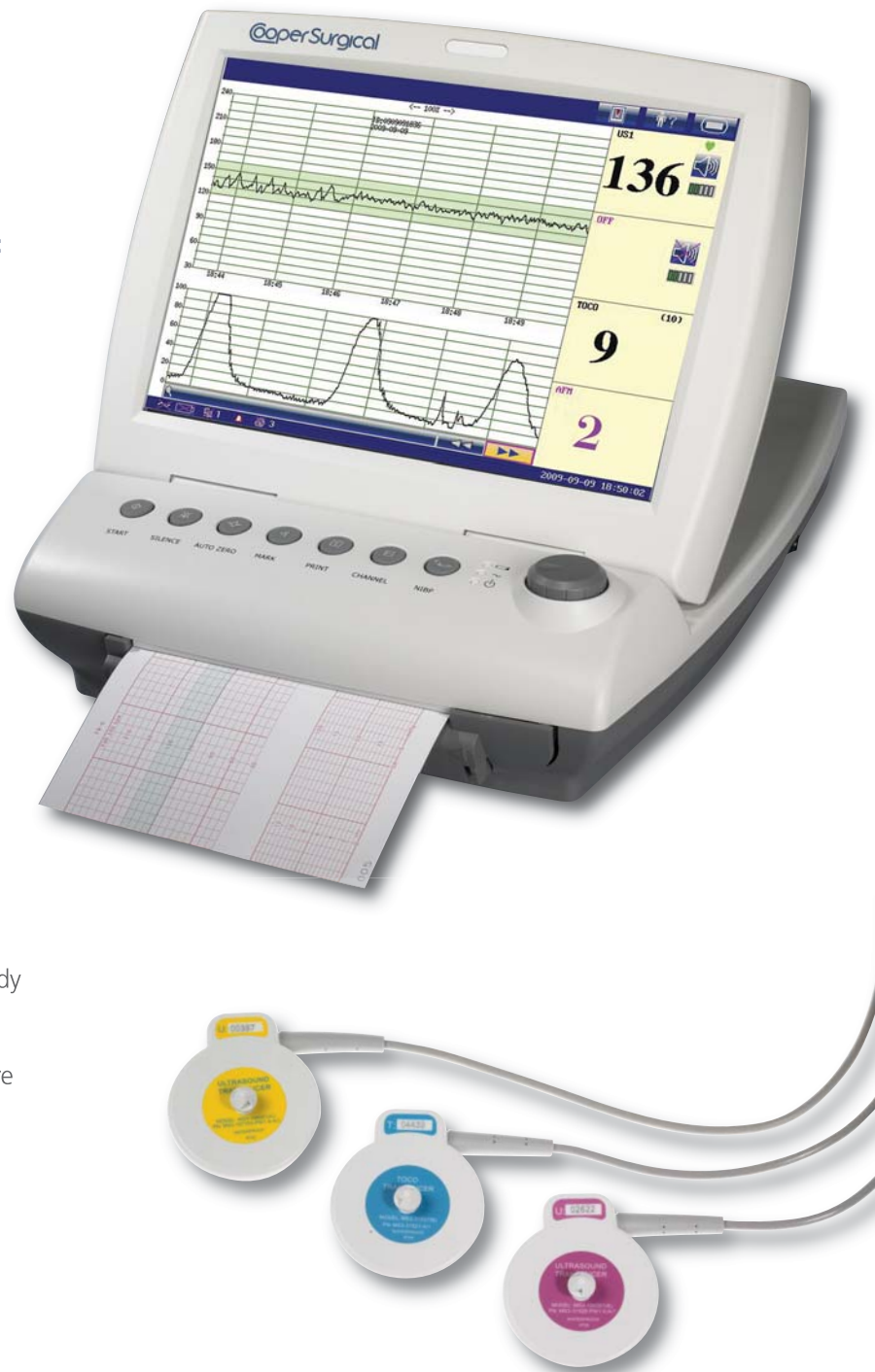

The CooperSurgical® F9 Fetal Monitor

Ease of operation and clinical accuracy for you...

Real-time information for your patient.
All in one compact lightweight unit.

The CooperSurgical Fetal Monitor combines the latest ultrasound technology and advanced user interface design within an ergonomically built light-weight unit that can be used in a variety of clinical settings. This advanced concept in fetal monitoring is easy to operate, provides the most accurate results available and fosters patient education and interaction during the examination.

Clinical Performance:

- **Industry leading ultrasound detection** designed with the latest pulse wave 12-Crystal, wide-beam, 1 MHz waterproof transducer. Ability to quickly capture and retain the signal even with an active fetus, reducing patient anxiety during the study
- **12-inch display screen** facilitates patient interaction while the study is in progress
- **Twins monitoring capability** is standard
- **Signal Overlap Verification** alerts the operator when synchronous heart-rate signals are detected, indicating that the signal may be from the same source
- **Auto fetal movement indicator** enhances real-time clinical evaluation of fetal heart rate signal with chart event markers and audible alerts
- **Remote fetal movement marker** enables comparison of maternal physical sensation with automatic chart events
- **Customizable fetal heart rate alarm** provides audible notification when the heart rate is outside of user-defined limits
- **Fetal heart rate offset** automatically shifts the heart rate by 20 BPM to facilitate clinical evaluation of twins. To further enhance readability, the tracings are different colors



CooperSurgical F9 Fetal Monitor Shown on Base Cart with Basket

Ease of Operation:

- The **12-inch color touch screen** facilitates operation and clearly displays exam results while testing is in process
- **Customizable alarm and control** settings can be changed to suit any operating environment to adjust for ambient noise and lighting levels
- **Compact unit with an integrated handle** folds for storage and easy transport. Ergonomic design includes docking stations for the transducers, printer and duplicate controls on the front deck
- **24 hour data memory** facilitates reprinting of study trace
- **Four hours of mobile operation** with optional integrated rechargeable lithium-ion battery

Enhanced Communications Capabilities:

- **Optional CooperSurgical Insight™ software** allows study data to be transferred to a computer system for study review and record storage
- **EMR compatibility** with PDF, JPG or TIF image file conversion and transfer using CooperSurgical Insight™ software means that virtually any EMR system can easily retrieve tracings




CooperSurgical F9 Fetal Monitor Shown on Deluxe Cart

Peak Signal Quality Reaches New Depths

The system incorporates industry-leading 12-Crystal transducers.

The result...

Superior signal quality is delivered over a broader area for better capture and maintenance with the ability to reach greater depths. Your ability to quickly isolate a preterm or active fetus or to obtain a strong signal in an obese patient will enhance your procedure and reduce patient anxiety in those challenging cases.

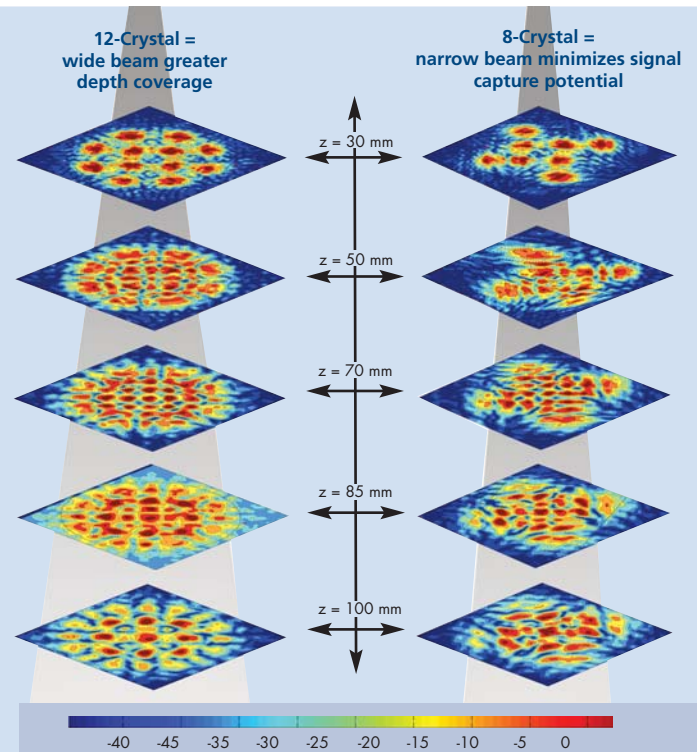



The CooperSurgical® F9 Fetal Monitor

Clinical Accuracy. Ease of Operation.
Enhanced Patient Communication.

Comprehensive Patient Care Includes Patient Education

The unit's unique large screen provides the basis for patient discussions that are interactive, individualized and informative.

CooperSurgical F9 Fetal Monitor System Components:

- 2 Ultrasound transducers
- 1 TOCO transducer
- 1 Event marker
- 2 Reusable belts
- Ultrasound gel
- Printer paper
- Power cord
- System user manual

Optional:

- CooperSurgical Insight™ software
- RS232 and ethernet cables
- Rechargeable lithium-ion battery



Configuration	F9 Antepartum
Dual-Fetal Heart Rate	Yes
TOCO	Yes
Fetal Movement	Yes
Auto Fetal Movement	Yes
Maternal ECG	No
Maternal NIBP	No
Maternal SPO ₂	No
Maternal Temperature	No

Order Information	
Part #	Description
MS9-108022-UL	CooperSurgical Fetal Monitor F9
MS9-108022-SWP	CooperSurgical Insight Software Package (Includes software and cables)
MS9-17692	Fetal Stimulator
M21-64072	Rechargeable Lithium-ion Battery
MS9-107634	Deluxe Cart
MS9-108166	Base Cart with Basket and Laptop Tray
MS9-108108	Base Cart with Basket
MS3-31528-A1	Transducer 1 (pink)
MS3-107705-A1	Transducer 2 (yellow)
MS3-31527-A1	TOCO Probe
MS3-31112	Event Marker
MS1R-02264	Transducer Belt (reusable)
M25R-75111	Recording Paper (90 mm x152 mm x150 p, US standard) (Pack of 3)

Marks bearing the symbols "TM" and "®" are trademarks and registered trademarks and registered trademarks of CooperSurgical, Inc.



CooperSurgical is the leading company dedicated to providing medical devices and procedure solutions that improve health care delivery to women regardless of clinical setting. Our company is fostering that position through expansion of its core businesses and introduction of advanced technology-based products which aid clinicians in the management and treatment of commonly seen conditions.

203.601.5200 | 800.243.2974 | coopersurgical.com | 95 Corporate Dr. | Trumbull, CT 06611
also at www.YouTube.com/CooperSurgical