

**Surgical Products**



# CoperSurgical Filshie® Clip System

**Expertly designed for safe and effective surgical contraception for women**

## **Features**

- Easy to Use
- Versatile
- Minimal Tubal Damage
- Large Tubal Capacity
- Highest Rate of Success



**CoperSurgical**

*Keeping You at the Forefront  
of Women's Health Care™*

# CooperSurgical

## Filshie® Clip System

### Used by Leading Surgeons, Recommended by Patients


More than 4 million women have undergone successful sterilization with the Filshie® Clip.\* Simple and reliable, the small titanium clip lined with soft Silastic® rubber is a safe and effective method of surgical contraception. Once in place, the clip occludes the fallopian tube, preventing sperm from reaching the egg in the fallopian tube. The Filshie Clip has the highest success rate among all current forms of permanent sterilization.\*

The Filshie Clip System of sterilization can be performed at the time of cesarean delivery, after a vaginal birth via mini-laparotomy, or as a laparoscopic procedure. The applicators for this sterilization method have been purposely designed for single- or dual-incision laparo-

scopy or postpartum procedures. Female sterilization with the clip can be performed in an outpatient setting, or in a hospital. Sterilization is immediate after this procedure.

In a recent study, more than 90% of women surveyed who had been sterilized using this system indicated they would recommend this method to other women.\*

The Filshie Clip's large curved upper jaw was designed for edematous tubes. The hinged titanium clip has an inner silicone rubber lining that completely occludes the fallopian tube following placement. When tubal necrosis occurs, the silicone rubber effectively maintains pressure on the tube. A thin layer of peritoneum may eventually cover the clip.

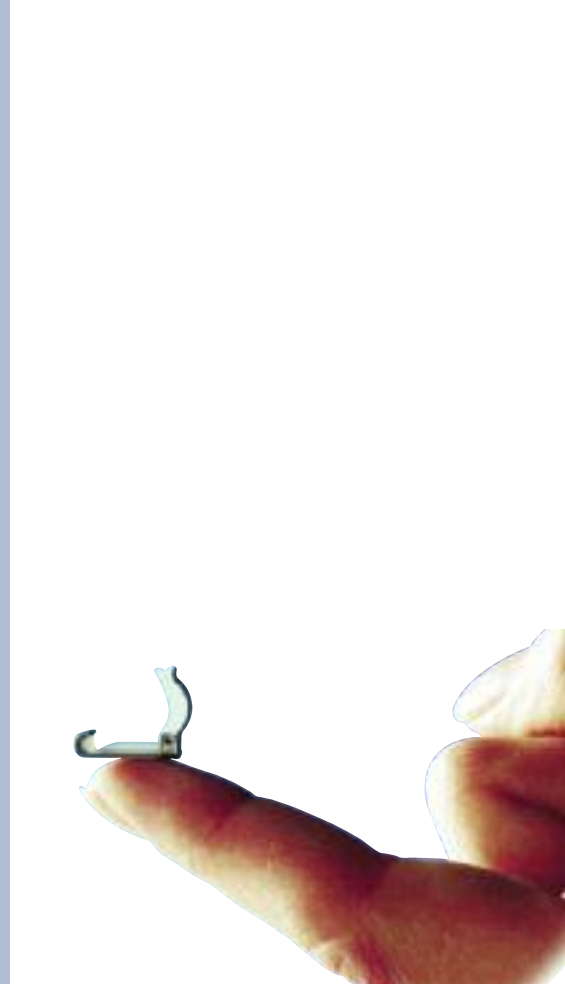

### Safe and effective

- No risk of bowel burn associated with other methods\*
- No tongs to perforate blood vessels
- No risk from the magnetic effects of an MRI
- Low failure rate—2.7 per 1000 patients\*

### Versatile with minimal tubal damage

- Affects approximately 4 mm of the fallopian tube—less than non-clip methods
- Large tubal capacity allows procedure for women with thicker fallopian tubes
- Ideal for postpartum procedures in cases where fallopian tubes can be swollen

### Ease of use and instruction

- Safe and easy
- Minimal procedure time required
- Quick patient recovery

\*Data on file at CooperSurgical, Inc.



applicators have been specifically designed for single- or dual-incision laparoscopy and for mini-laparotomy postpartum procedures

## SIMPLE APPLICATION TECHNIQUE

### DIAGRAM 1

Apply the clip across the entire diameter of the fallopian tube with the hooked end of the lower jaw visible through the mesosalpinx. Before closure, use the applicator as a manipulator to properly identify the tube(s) and placement of the clip.



### DIAGRAM 2

Squeeze the applicator to compress and flatten the upper jaw, locking it under the hooked end of the lower jaw.



### DIAGRAM 3

Remove the applicator leaving the locked Filshie Clip compressing the entire diameter of the tube within its jaw. One Filshie Clip per fallopian tube is required; clips are permanently implanted.





**Single-Incision**  
 (Available in 5 mm, 7.5 mm, and 8 mm sizes)



**7 mm Dual-Incision**



**Mini-Laparotomy**

**Applicators**

The range of application equipment is designed to be robust and reliable for single- and dual-incision laparoscopy and for mini-laparotomy postpartum procedures.

Manufactured from high-quality stainless steel, the simple-to-use applicators are easy to assemble and disassemble, and are easy to clean and maintain.

The applicators require servicing annually or after every 100 procedures, whichever comes first.

**ORDERING INFORMATION**

CLIPS	ITEM NO.	DESCRIPTION
	AVM-851	Box of Filshie Clips (20 pair per box)
AVM-851J	Box of Filshie Clips (10 pair per box)	

APPLICATORS	ITEM NO.	DESCRIPTION
	AVM-856	Mini-Laparotomy Filshie Clip Applicator
	AVM-857	7.5 mm Single-Incision Filshie Clip Applicator
	AVM-860	8 mm Single-Incision Filshie Clip Applicator
	AVM-863	7 mm Dual-Incision Filshie Clip Applicator
	AVM-880	5 mm Single-Incision Filshie Clip Applicator

To find out more about the Filshie Clip System, or to place an order:  
 Call 800.243.2974 or visit [www.coopersurgical.com](http://www.coopersurgical.com).