



CooperSurgical



Lone Star Disposable Retractor Rings

For simpler, better managed surgeries.

For a wide range of surgical procedures

Benefits

- Excellent visibility
- Adapts and repositions quickly
- Adjustable tension for optimal exposure
- Frees up OR staff

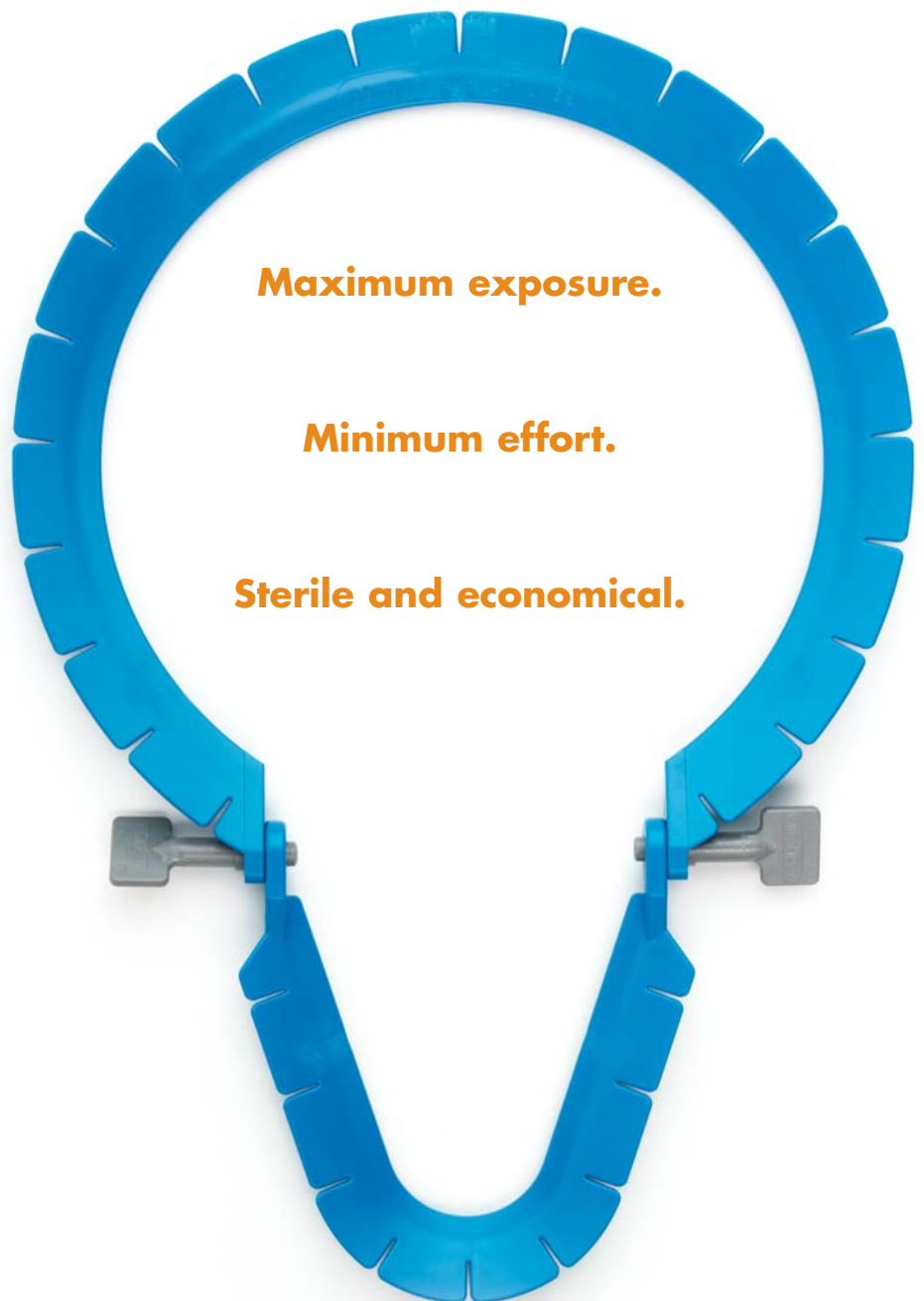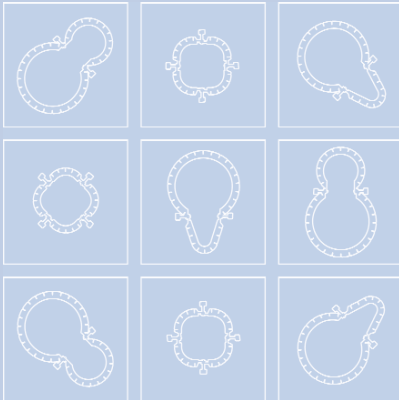

Maximum exposure.

Minimum effort.

Sterile and economical.


Lone Star Disposable Retractor Rings

Exceptional visibility and ease-of-use.

Looking to simplify surgeries? Our Disposable Retractor Rings are the perfect choice, delivering multiple advantages for optimum surgical results. They dramatically improve visibility in the surgical field by eliminating the need for manual retractors and the personnel to hold them. Their articulated design adapts to any region of the anatomy. Adjustable hinges and a system of Elastic Stay hooks allow for easy positioning and repositioning throughout the procedure.

Made of Noryl® resin, these superior Retractor Rings arrive in your OR sterilized and ready to use. Upon completion of procedure, simply dispose, and eliminate the costs of sterile processing.

Ease and Precision in 3 Simple Steps

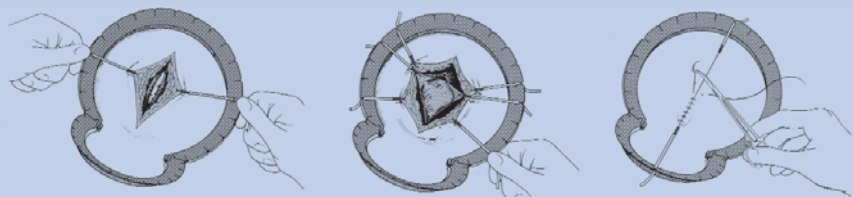

1. Place Ring around initial incision. Insert Elastic Stay hooks into tissue and retract in directly opposing quadrants. Slots on ring allow easy placement for proper tension.
2. Reposition or add hooks throughout procedure to maintain balanced retraction.
3. When closing, place stays along axis of the incision to create tension to facilitate precise staple or suture placement.

Disposable Retractor Ring Specifications

Packaged singly, sterile and ready for use.

3301G 26.5 cm x 11.3 cm

3302G 25.8 cm x 11.3 cm

3304G** 31.8 cm x 18.3 cm

3307G 14.1 cm x 14.1 cm

3308G** 28.6 cm x 18.3 cm

3309G 19.6 cm x 8.9 cm

3310G* 25.0 cm x 25.0 cm

**Can be assembled in several configurations.*


*** Packaged singly with two catheter clips.*



3301G



3302G

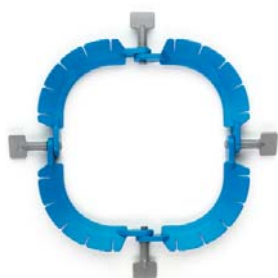

3307G



3308G



3309G



3310G



3304G

CooperSurgical

www.coopersurgical.com

To find out more about the Lone Star Disposable Retractor Rings, or to place an order call 800.243.2974.

Lone Star Retractor System™ is a trademark of CooperSurgical, Inc.
Noryl® is a registered trademark of GE.

81271 Rev 10/07

Copyright © 2007 CooperSurgical

95 Corporate Drive
Trumbull, CT 06611
800.243.2974
203.601.5200