Pilot® Guides Instructions for Use (English)

Device Description

Sterile, single-use Pilot Guides consist of either:

- **PG-5/10:** One (1) 5 mm and one (1) 10/12 mm Pilot Guide or
- **PG-15S:** One (1) 15 mm Pilot Guide

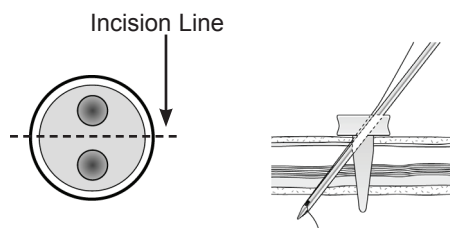
Indications for Use

The Pilot Guide is intended for use during endoscopic/laparoscopic surgery. It is to be used only by surgeons trained in endoscopic/laparoscopic surgery.


Instructions for Use

The Pilot Guides can be used in conjunction with the Carter-Thomason® Suture Passer to close trocar wounds in the following manner (see figures below):


Step 1

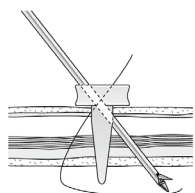- Insert Pilot Guide with holes aligned perpendicular to the incision.
- Push suture passer through Pilot Guide, fascia, muscle, peritoneum and into abdomen.
- Drop suture and remove suture passer.


Step 2

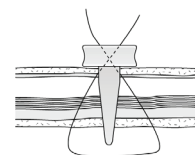Push suture passer through opposite side of Pilot Guide. Pick up suture. We recommend the use of Vicryl Size 0 braided Sutures. (CSI approved suture material via validations)

Step 3

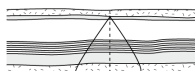Pull the suture up through the peritoneum, muscle, fascia, & Pilot Guide.



Step 4


Remove Pilot Guide. Tie suture to complete the closure.

If using the 15 mm Pilot Guide, position the guide with paired holes aligned perpendicular to the incision.

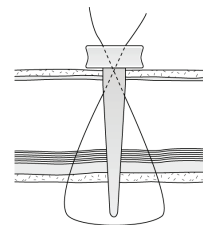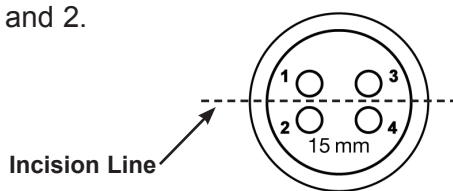
Perform Steps 1-3 using the holes labeled 1 and 2.

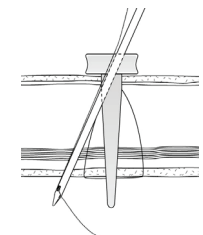With another piece of suture, repeat Steps 1-3 using holes labeled 3 and 4.

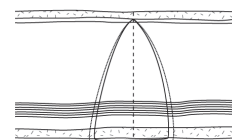Remove Pilot Guide. Tie each suture to complete the closure.

Explanation of Symbols

Reorder Number

Lot number

Use-by date

Sterilized using ethylene oxide

Do not re-use

Consult instructions for use

Caution

Authorized Representative in the European Community

Manufacturer

Do not use if package is damaged

Do not re-sterilize

R_xOnly

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

Not made with natural rubber latex

Pilot® and Carter-Thomason® are registered trademarks of CooperSurgical, Inc.
Coopersurgical® is a registered trademark of CooperSurgical, Inc.

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© 2017 CooperSurgical, Inc.
Pilot® e Carter-Thomason® sono marchi registrati di CooperSurgical, Inc.
CooperSurgical® è un marchio depositato di CooperSurgical, Inc.