11080 • Spatola in plastica Pap Perfect®

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CooperSurgical

CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611 USA

95 Corporate Drive
Trumbull, CT 06611 USA
Phone: (800) 243-2974
Fax: (800) 262-0105
www.coopersurgical.com

International:
Phone: (203) 601-9818
Fax: (203) 601-4747

EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands

11080 • Pap Perfect® Plastic Spatula

Directions for Use (English)

DEVICE DESCRIPTION

The Pap Perfect® Plastic Spatula gently rubs the mucosa, ensuring a rich sample from ectocervix. The polypropylene plastic is inert and will release collected cells easily onto the slide for Pap Smear or in the vial for Liquid Cytology. The Pap Perfect Plastic Spatula is a **Single use, Non-sterile** device. Discard after use.

CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- For use by trained medical staff only.

INDICATIONS FOR USE

The Pap Perfect Plastic Spatula is used to collect cell samples from the ectocervix for Pap smear or Liquid Cytology. (The endocervical sample is usually collected with a Cytobrush® Plus.)

CONTRAINDICATIONS

Do not use on patients suspected of infection, who demonstrate profuse bleeding or are suspected of carrying sexual transmitted diseases.

SAMPLING INSTRUCTIONS

1. Gently fixate the spatula tip in the center of the portio vaginalis.
2. Rotate the spatula slowly one (1) full turn.
3. Withdraw the Pap Perfect Plastic Spatula.

PREPARATION OF SAMPLE FOR PAP SMEAR CYTOLOGY

1. Smear the spatula evenly over a clean glass slide leaving room for the endocervical sample.
2. Fix immediately with cytology spray fixative or 95% ethanol.

PREPARATION OF LIQUID CYTOLOGY

Consult your local laboratory for detailed instructions.

EXPLANATION OF SYMBOLS

REF	Reorder Number	i	Consult instructions for use.
LOT	Batch Code	Factory	Manufacturer
Hourglass	Use-By Date	EC REP	Authorized Representative in the European Community.
No reuse	Do not re-use	CE	Product conforms to the Medical Device Directive 93/42/EEC
No damage	Do not use if package is damaged		Not made with natural rubber latex.
Non sterile	Non-sterile		

Marks bearing © are registered trademarks of CooperSurgical, Inc.
© 2014 CooperSurgical, Inc.

CooperSurgical

CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611 USA

95 Corporate Drive
Trumbull, CT 06611 USA
Phone: (800) 243-2974
Fax: (800) 262-0105
www.coopersurgical.com

International:
Phone: (203) 601-9818
Fax: (203) 601-4747

EMERGO EUROPE
Molenstraat 15|
2513 BH, The Hague
The Netherlands

This page left blank intentionally

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Les marques accompagnées du symbole ® sont des marques déposées de CooperSurgical, Inc.