
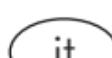
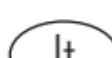
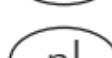
# Cytobrush® Plus

Rx Only 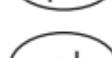

-  Návod na použití
-  Brugsanvisning
-  Gebrauchsanweisung
-  Οδηγίες χρήσης
-  Instructions for use
-  Instrucciones de uso
-  Kasutusjuhend
-  Notice d'utilisation
-  Kyäyttöohjeet
-  Használati utasítás
-  Istruzioni per l'uso
-  Vartojimo instrukcija
-  Lietošanas pamācība
-  Gebruiksaanwijzing
-  Instrukcja użytkowania
-  Instruções de uso
-  Návod na použitie
-  Navodilo za uporabo
-  Bruksanvisning

# Cytobrush® Plus

## Nesterilní

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# Cytobrush® Plus

## Non-sterile

Cytobrush® Plus is a single use device. Discard after use.

### **CAUTION**

For use by trained medical staff only.

### **INDICATIONS FOR USE**

Cytobrush® Plus is used to collect cell samples from the endocervix for

1. Pap smears (the ectocervical sample is usually collected with a spatula).
2. Diagnosis of infections, e.g. Chlamydia.

### **CONTRAINDICATIONS**

Do not use Cytobrush® Plus after the first ten (10) weeks of gestation in pregnant patients.

Do not use Cytobrush® Plus for endometrial sampling.

### **WARNINGS**

Never allow Cytobrush® Plus to enter too far into the endocervix. A few bristles of the brush should at all times be seen in the lower part of the cervix.

**CAUTION:** The product is for single use and not to be reused due to risk of contamination.

### **ADVERSE REACTIONS**

Slight bleeding may occur during sampling and is acceptable. Check that bleeding has ceased before end of gynaecological examination.

### **SAMPLING INSTRUCTIONS**

1. Gently insert Cytobrush® Plus into the endocervical canal. A few bristles of the brush should be seen outside the cervix.
2. Rotate the brush slowly one (1) turn.
3. Withdraw Cytobrush® Plus

### **PREPARATION OF SAMPLE FOR CYTOLOGY**

#### **(Pap smear)**

- Roll Cytobrush® Plus over designated slide area.
- Fix immediately with cytology spray fixative or 95% ethanol.

### **PREPARATION OF SAMPLE FOR MICROBIOLOGY**

#### **(e.g. Chlamydia)**

Consult your local microbiology laboratory for detailed instructions regarding specific tests.

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CooperSurgical, Inc.