



ACKRAD LABS
a CooperSurgical Company

IRRIJET[®] G P Single-Use

WOUND IRRIGATION SYSTEM

PRODUCT NO. 55-0004

LOT NO. XXXXX

EXP. YYYY-MM

FOR SINGLE USE ONLY

STERILE R

Sterile in unopened
undamaged package

Caution: Federal law (U.S.) restricts this device to sale by or on the order of a physician.

INSTRUCTIONS

IMPORTANT: For proper priming of the system, please see Step 3.

1. Peel open pouch.

2.a. Remove protective tube from spike and insert the spike into a bag of irrigating solution.

b. Attach syringe to female luer of double check valve.

CAUTION: DO NOT OVERTIGHTEN SYRINGE.

3. TO OPERATE

a. Push plunger in SHARPLY to prime the system.

b. If necessary, loosen the syringe to allow fluid to flow into the double check valve. Retighten the syringe and repeat step 3a.

c. Irrigate the wound, using a pumping action with syringe plunger.

Manufactured By:

CooperSurgical

Trumbull, CT 06611

(800) 243-2974

(203) 601-5200

35756-04-04 Rev. A