

LUMAX™ S-Series Cable Installation

INSTRUCTIONS FOR USE

Installing S-Series Mounting System

The new LuMax™ S-Series catheter has been manufactured to allow for rapid and convenient set-up for cystometry procedures. Follow these step by step instructions to get started with S-Series catheters.

- 1 Cut template along dotted lines.
See Page 2

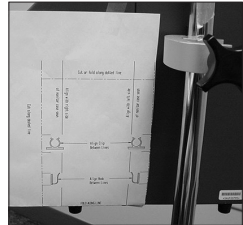


Figure 1

- 2 Fold the template along the lower line and place flap against the bottom of the LuMax™ monitor.

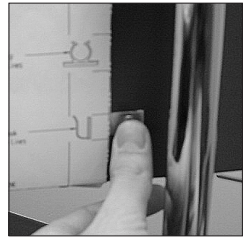


Figure 2

- 3 Align solid line on template to case seam on either side of LuMax™ monitor. See Figure 1

- 4 Remove adhesive backing from the hook and clip.

- 5 Align and secure the hook and clip to the designated areas on LuMax™ monitor. See Figures 2 & 3

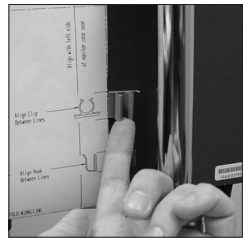


Figure 3

- 7 Attach the semi-permanent transmission cables to the LuMax™ monitor by inserting key connector (middle key at 12:00) and turning one quarter turn clockwise until snug. See Figure 4



Figure 4

- 8 For rapid identification of the transmission cables attach the included Pves and Pabd labels to both ends of the cables. See Figure 5

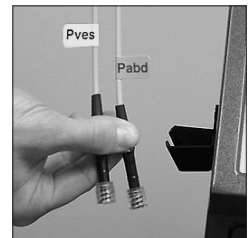


Figure 5

- 9 When not in use, the free end of the transmission cable should be stored in the clip. See Figure 6



Figure 6

**For assistance call CooperSurgical
Technical Services at 800-216-8477**

Using S-Series Catheters

- 1 Open S-Series catheter pouch to expose the bar code handle.
- 2 Grasp the barcode handle on the catheter and firmly insert the S-Series catheter tabs into the groove on the transmission cable. See Figure 1



Figure 1

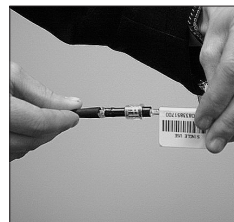


Figure 2

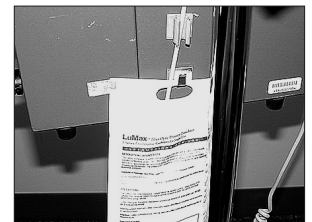


Figure 3

- 3 Secure the catheter/cable connection by rotating the spin-lock collar on the transmission cable until snug. See Figure 2

- 4 After securing the S-Series catheter/cable connection, suspend the catheter pouch from the hook and place the barcode handle into the clip on the side of the LuMax™ monitor. See Figure 3

- 5 Proceed with catheter scanning (Model 4114UF) and calibration.

Manufactured by:


CooperSurgical

95 Corporate Drive, Trumbull, CT 06611

800-243-2974

LUMAX™ S-Series Cable Installation

TEMPLATE FOR CLIP PLACEMENT

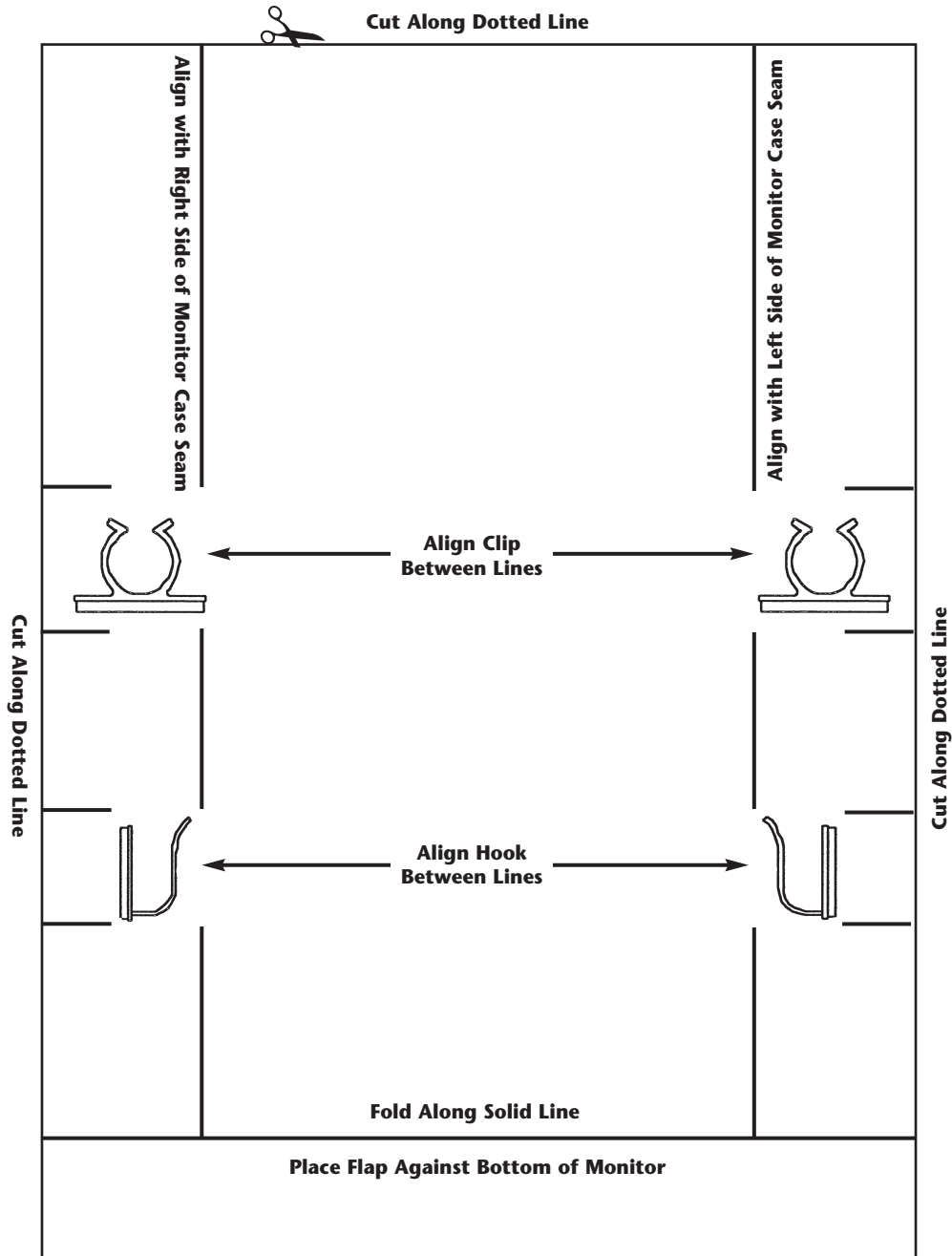

Manufactured by:

CooperSurgical

95 Corporate Drive
Trumbull, CT 06611

800-243-2974