Cytobrush® Plus GT Scored

R_x Only (2)

- (cs) Návod na použití
- (da) Brugsanvisning
- (de) Gebrauchsanweisung
- (el) Οδηγίες Χρήσης
- (en) Instructions for Use
- (es) Instrucciones de uso
- (et) Kasutusjuhend
- (fr) Mode d'emploi
- (fi) Käyttöohjeet
- (hu) Használati utasítás
- (it) Istruzioni per l'uso
- (lt) Vartojimo instrukcija
- (lv) Lietošanas pamācība
- (nl) Gebruiksaanwijzing
- (pl) Instrukcja użytkowania
- (pt) Instruções de utilização
- (sk) Návod na použitie
- (sl) Navodila za uporabo
- (sv) Bruksanvisning

Cytobrush® Plus GT Scored

Nesterilní

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Cytobrush® Plus GT Scored

Non-sterile

Cytobrush® Plus GT Scored is a single use device.
Discard after use.

CAUTION

For use by trained medical staff only.

INDICATIONS FOR USE

Cytobrush® Plus GT Scored is used to collect cell samples from the endocervix for liquid cytology and microbiology. The device has a prescored shaft and should NOT be used for pap smear due to risk of unintentional shaft breakage.

CONTRAINDICATIONS

Do not use Cytobrush® Plus GT Scored after the first ten (10) weeks of gestation in pregnant patients.

Do not use Cytobrush® Plus GT Scored for endometrial sampling.

WARNINGS

Never allow Cytobrush® Plus GT Scored to enter too far into the endocervix. A few bristles of the brush should at all times be seen in the lower part of the cervix.

CAUTION: The product is for single use and not to be reused due to risk of contamination.

ADVERSE REACTIONS

Slight bleeding may occur during sampling and is acceptable. Check that bleeding has ceased before end of gynaecological examination.

SAMPLING INSTRUCTIONS

1. Gently insert Cytobrush® Plus GT Scored into the endocervical canal. A few bristles of the brush should be seen outside the cervix.
2. Rotate the brush slowly one (1) turn.
3. Withdraw Cytobrush® Plus GT Scored.

PREPARATION OF SAMPLE

Consult your laboratory for detailed instructions regarding specific tests.

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CooperSurgical, Inc.