Incontinence Dish, Incontinence Dish with Support, Incontinence Ring Flexible Pessaries Rx Only

For stress urinary incontinence with mild prolapse

AVAILABLE MODELS

Single-Patient Use € 0086

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

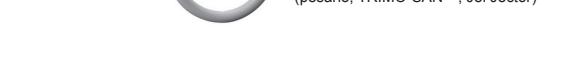
The Milex® INCONTINENCE DISH and INCONTINENCE RING Pessaries are medical devices made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and or rectum.

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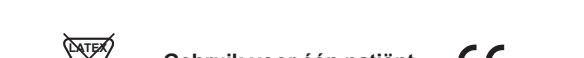
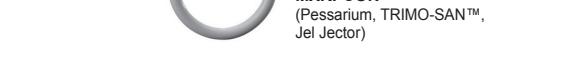
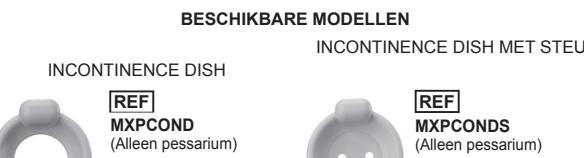
WARNINGS

- The INCONTINENCE RING Pessary contains a wire coil and must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Use only TRIMO-SAN. Other materials have not been tested for compatibility.

INDICATIONS FOR USE

- Stress urinary incontinence
- The INCONTINENCE DISH with SUPPORT is indicated if stress urinary incontinence with a mild uterine prolapse is complicated by a mild cystocele.

CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

RECOMMENDED CLEANING INSTRUCTIONS*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
- Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
- Following scrub, inspect device for visible soil residue.
- Rinse
 - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
 - Allow the device to dry.

*New pessaries are powdered with food-grade powder.

PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

To ensure the desired correction of your condition, your healthcare professional needs your full cooperation.

- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner. It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

REPORT ANY OF THE FOLLOWING SYMPTOMS TO YOUR PHYSICIAN

- Any difficulty in urinating
- Any discomfort
- Any changes in the color or consistency of vaginal discharge
- Any increase in the amount of vaginal discharge or vaginal bleeding
- Any foul odor associated with vaginal discharge
- Vaginal itching
- If the pessary falls out

For medical emergencies and for all medically related advice, consult your healthcare professional.

FOR THE PHYSICIAN / HEALTHCARE PROFESSIONAL

Review these instructions with the patient to establish use regimen.

INSTRUCTIONS

- Wear dry gloves. When necessary, lubricate only the entering end of the pessary with TRIMO-SAN. Compress the pessary (bring sides together) as shown in Figure 1 for the INCONTINENCE DISH and Figure 1a for the INCONTINENCE RING.

Figure 1

Figure 1a

- Use one finger of the opposite hand to depress the perineum. Hold the pessary almost parallel with the introitus (see Figures 2 and 2a). Direct the entering end of the pessary past the cervix into the posterior fornix. Allow the pessary to open into shape after passing the introitus.

Figure 2

Figure 2a

- Use the index finger to guide the pessary along the lower vaginal wall, behind the cervix, and into the posterior fornix.
- Use the index finger to bring the knob up behind the symphysis pubis (see Figures 3 and 3a).

Figure 3

Figure 3a

Note: If the patient is unable to urinate with the pessary in position, remove it and fit her with the next smaller size. Repeat as necessary.

- If the patient can void without difficulty, and the pessary remains in position upon re-examination, and the patient is comfortable with the pessary in place, this is a good indication that the correct size may have been selected. Patient experience may vary.
- Examine the patient while she is in the standing position to ensure the pessary has not shifted position. The patient should not feel the pessary once it is in position. The pessary should not be too loose as it may turn or be expelled and it should not be too tight as it may cause discomfort.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

- The healthcare professional should be able to sweep one finger between the pessary and the vaginal walls. If there is not enough space to do this, the next smaller size should be tried. If excessive space exists, the pessary will not be effective and may rotate or even be expelled.

- It is sometimes necessary to re-fit the patient with a different size or type of pessary after a period of time. Do not assume that a replacement will always be the same size as the previous one. Check the fitting to ensure continued patient comfort and relief of symptoms. The useful life of a pessary is limited. Examine frequently for signs of deterioration (such as cracks or breaks in silicone outer surface). A pessary should be replaced if damaged.

- Ulcerations and erosions frequently occur in cases of complete prolapse due to irritation of the exteriorized cervix or vaginal wall. Whenever possible, reducing the mass and treating the irritation are primary steps before using a pessary. Prolapse reduction may resolve cervical vaginal irritation. Verification of cervical cytology (pap) and or biopsy is clinically indicated.

- During each visit, the vagina should be carefully inspected for evidence of pressure or allergic reaction. The patient should be questioned concerning douching, discharge, disturbance of bowel function or urination. It may be necessary to fit another size or an entirely different type of pessary.
- At the physician's discretion, the patient can be instructed in the proper removal, cleaning and reinsertion techniques for her own pessary. This process can be performed nightly or even weekly by the patient under ideal circumstances.

TO REMOVE

Use the index finger to depress the perineum. Hook other index finger under the knob of the INCONTINENCE pessary and pull down. Fold the pessary by bringing the sides together, angling it so that it is almost parallel to the introitus, and gently ease the pessary out.

RECOMMENDED FOLLOW-UP

- Have patient return within 24 hours for first examination.
- Have patient return for second examination within 3 days.
- Have patient return for examination every few months.

Note: The above schedule of follow-up examinations may be altered to fit the needs of the individual patient at the discretion of the healthcare provider.

EXPLANATION OF SYMBOLS

REF Reorder number

LOT Batch code

U Use-by date

I Consult instructions for use

NTR Not made with natural rubber latex

RxOnly CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician

EC REP Authorized Representative in the European Community

Manufacturer

Dawn® is a registered trademark of Procter & Gamble, Cincinnati, Ohio 45202
TRIMO-SAN™ is a trademark of CooperSurgical, Inc.
Milex® and CooperSurgical are registered trademarks of CooperSurgical, Inc.
© 2017 CooperSurgical, Inc.

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Milex® et CooperSurgical sont des marques déposées de CooperSurgical, Inc.