# WALACH

**PAPETTE**<sup>®</sup> Cervical Cell Collector (STERILE)

REF 908000-1

Read all safety information and instructions listed below before using this product!

**CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.**

#### INTENDED USE

- For Cervical Cytology utilizing the Conventional Pap Smear.
- The *PAPETTE* is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.

#### INSTRUCTIONS

1. Contact the cervix with the *PAPETTE* and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix.
2. Push gently, and rotate the *PAPETTE* in either direction.
3. Remove the *PAPETTE* and draw each side once across the slide (like a paint brush), thus transferring both ectocervical and endocervical cell samples in one stroke.
4. Dispose of *PAPETTE* in accordance with local Medical Hazardous Waste practices.

Note: It is advisable to inform your pathologist or lab that the Wallach *PAPETTE* is being used to obtain samples.

**PAPETTE**<sup>®</sup> U.S. Patent #4,700,713

## WALACH

**Surgical Devices**

95 Corporate Drive, Trumbull, CT 06611 USA  
Tel: (203) 799-2000 • Fax: (203) 799-2002

IMMS028-1 • Rev. A • 1/09

Side 1

# WALACH

**PAPETTE®** Cervical Cell Collector (STERILE)  
utilizing the ThinPrep® Pap Test™

REF 908000-1

Read all safety information and instructions listed below before using this product!

**CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.**

### INTENDED USE

- For Cervical Cytology utilizing the ThinPrep® Pap Test™
- The *PAPETTE* is the most gentle, yet effective means of simultaneously taking both ectocervical and endocervical cell samples.

### INSTRUCTIONS

1. Contact the cervix with the *PAPETTE* and insert the central bristles into the cervical canal deep enough to allow the shorter bristles to fully contact the ectocervix.
2. While maintaining gentle pressure in the direction of the cervix, rotate the brush 5 times in a clockwise direction.
3. Rinse the brush into the PreservCyt® solution for use with the ThinPrep® Pap Test™ by pushing the brush into the bottom of the vial 10 times, forcing the bristles apart. Swirl the brush vigorously to further release material. Discard the collection device.
4. Tighten the PreservCyt® vial cap so that the torque line on the cap passes the torque line on the vial.
5. Dispose of *PAPETTE* in accordance with local Medical Hazardous Waste practices.

*PAPETTE*® U.S. Patent #4,700,713

Trademarks: *ThinPrep*® and PreservCyt® are registered trademarks of Cytyc Corp.

### EXPLANATION OF SYMBOLS

Attention, see instructions for use

Do not use if pouch has been opened or damaged

Batch Code

Expiration Date

Single-Use Only

**REF**

Reorder Number

Symbol for Method of Sterilization Using Ethylene Oxide

For Professional Use Only

Storage Temperature (max 30°C / 86°F; min 10°C / 50°F)

CE Mark (for medical devices only) made in accordance with 93/42/EEC medical devices directive

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Marchi commerciali: **ThinPrep®** e PreservCyt® sono marchi depositati di Cytyc Corp.
Rappresentante europeo autorizzato: Leisegang Feinmechanik GmbH