



Five Steps to Improve Your Endometrial Sampling Technique

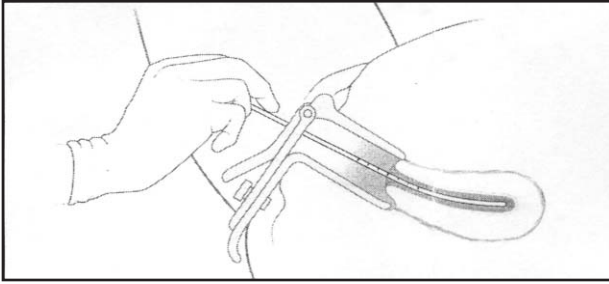


1 SITE

Insert *SelectCells* device through the cervix and into the uterine cavity up to the fundus.

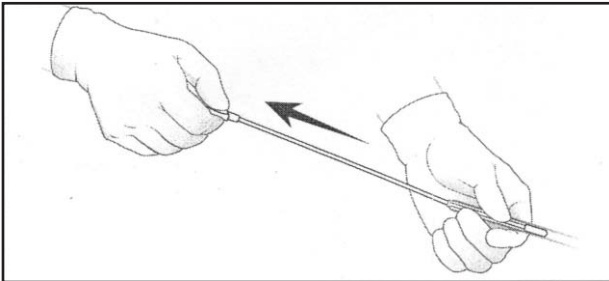


2 SUCTION

Adequate suction can easily be created by holding the *SelectCells* device firmly in place and pulling the piston rod with one smooth, rapid motion until it stops at the proximal end.

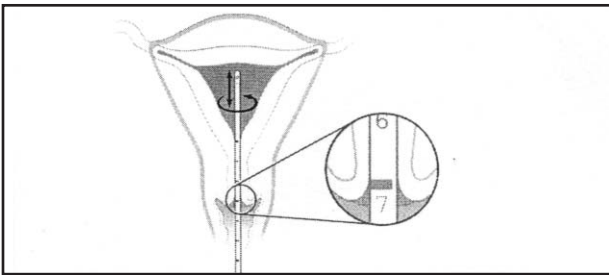


3 SAMPLE

Using the index finger and thumb, rotate the *SelectCells* device through 360 degrees with a gentle “to and fro” motion several times. The ideal movement is continuous, smooth, and uninterrupted.

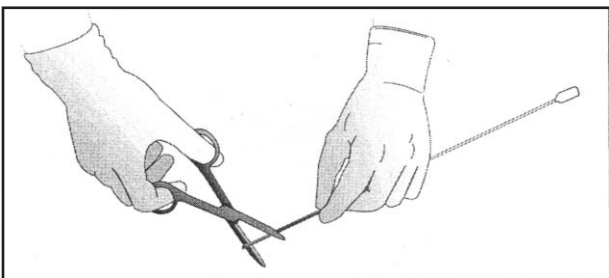

4 SNIP

After withdrawing the *SelectCells* device from the uterus, cut the tip off at the distal opening.

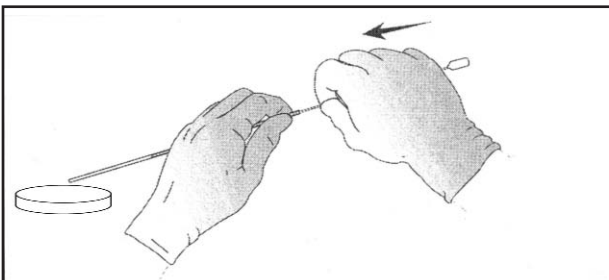

5 EXPEL

Holding the *SelectCells* device at the base of the sheath with one hand, grasp the piston rod 2 cm from the base and thread the piston rod back in, using short movements to expel the contents into a specimen container.

Refer to the instructions for use for complete directions and precautions.

@operSurgical

95 Corporate Drive • Trumbull, CT 06611 USA
Phone: (800) 243-2974 • Fax: (800) 262-0105

International:
Phone: (203) 601-9818 • Fax: (203) 601-4747

SM501400 • Rev A • 2/07