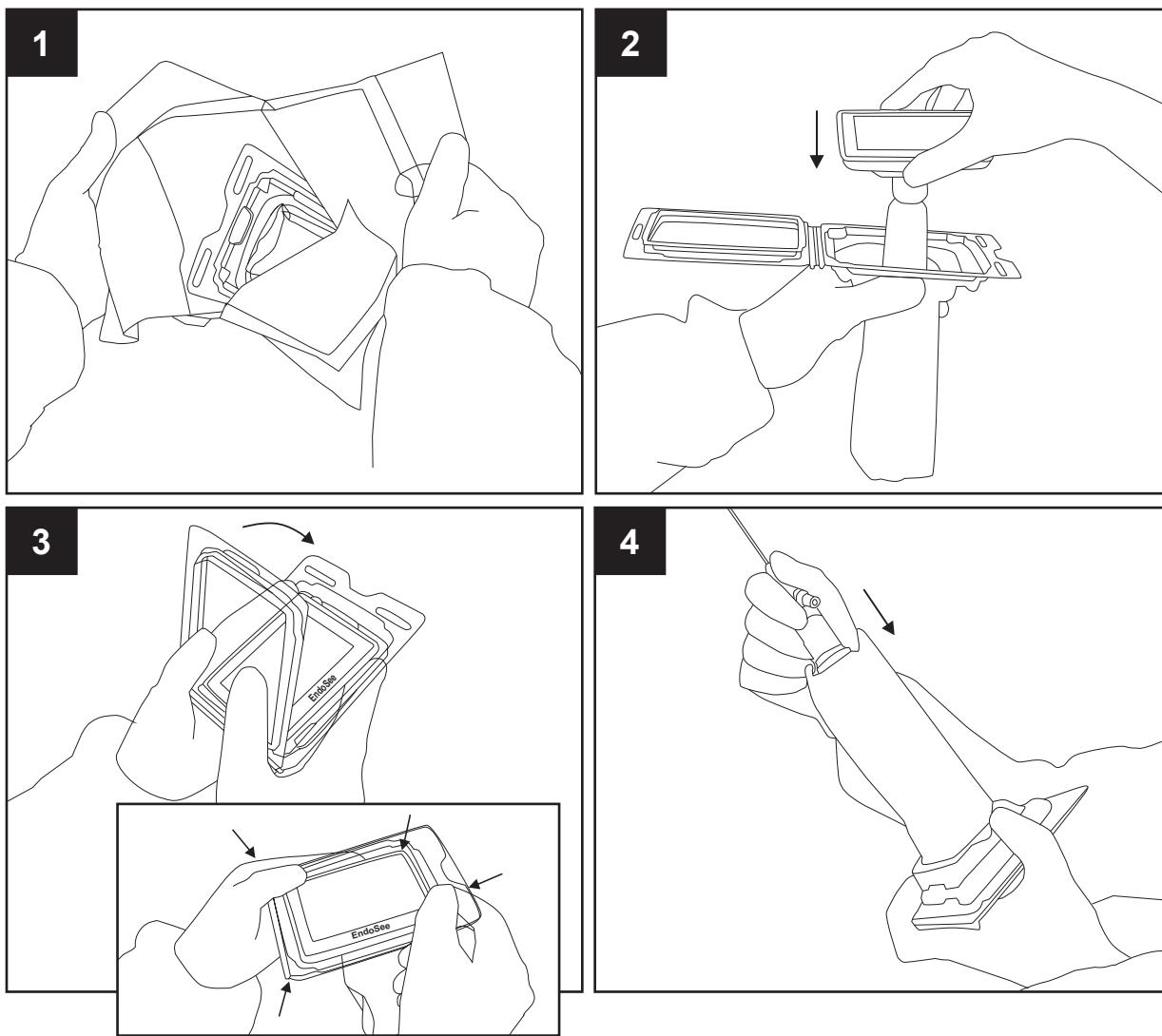
Endosee® Drape

Instructions for Use (English)

REF ESDRAPE • OR-ESDRAPE**Instructions for Use (aseptic technique):**

1. Circulator: Peel the package and present the drape to the sterile user.
2. Insert the Endosee handle, monitor on top, into the drape without a cannula attached. The insertion can be done by the non-sterile circulator being careful not to touch the outside of the drape.
3. Close the screen cover of the drape careful not to touch the Endosee device and press the corners ensuring all four locks are fully seated.
4. Connect a cannula to Endosee monitor through the distal gasket.
Caution: Avoid pinching the gasket by holding the drape near the gasket.

Removal Instructions

Open the drape, keeping it vertical. Remove the Endosee device. Inspect it and refer to Endosee Handheld Monitor user manual for further reprocessing instructions.

Indications For Use:

The Sterile Drape for Endosee® is used to cover the Endosee handheld monitor in the clinical and surgical settings where equipment protection is necessary to prevent contamination during various procedures.

Caution:

Discard the cover if any tear or breach is noticed during use.

Explanation of Symbols

Reorder number

Sterilized using ethylene oxide

Batch number

U.S. Federal law restricts this device to sale by or on the order of a physician.

Use by date

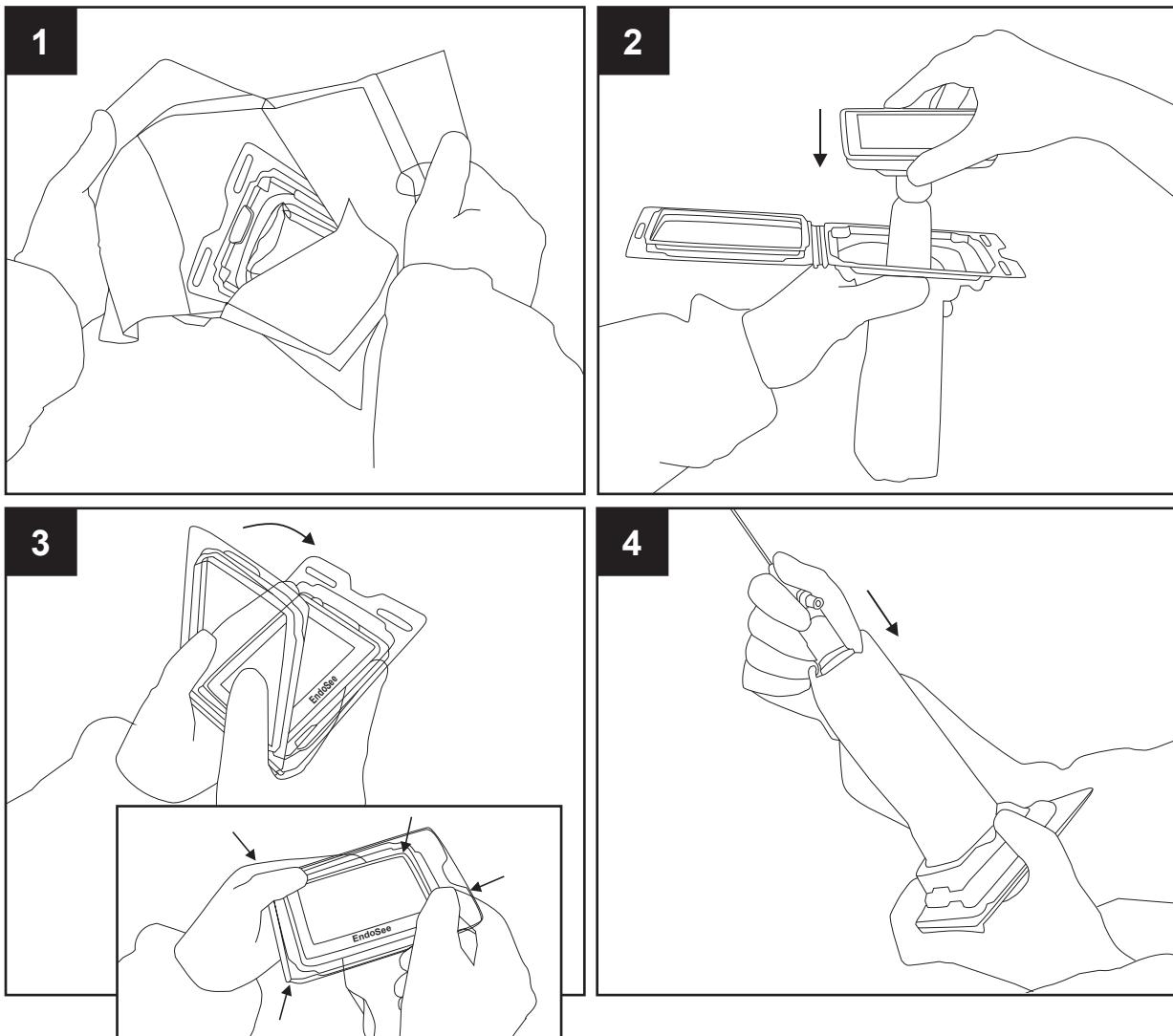
Authorized Representative in the European Community.

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Endosee® is a registered trademark of CooperSurgical, Inc.