How to use and take care of your

English

RxOnly

€0086

Arcing and Omniflex Silicone Vaginal Diaphragm

MXWS+Size Arcing Diaphragm
MXWF+Size Omniflex Diaphragm

CooperSurgical

DEVICE DESCRIPTION

The Wide-Seal Silicone Vaginal Diaphragm is a shallow cup with a flexing rim. When the diaphragm is coated with spermicidal gel and placed over the cervix (the opening of the uterus), it prevents sperm from entering the uterus and provides a reservoir for the spermicidal gel or cream. The diaphragms are available in Arcing or Omniflex styles.

AVAILABLE SIZES

60 mm, 65 mm, 70 mm, 75 mm
80 mm, 85 mm, 90 mm and 95 mm.

Read the entire sheet carefully, even if you have used a diaphragm in the past. If you have additional questions, ask your doctor or healthcare specialist.

DIAPHRAGM PLACEMENT

The diagram shows the vaginal walls, the cervix (protruding into the vaginal vault) and the pubic notch. When the diaphragm is properly fitted and inserted, it fits over the cervix and is tucked into the pubic notch and against the rear vaginal wall.

WARNINGS

- Keep your diaphragm and spermicide out of the REACH of CHILDREN.
- If you are using any vaginal medication check the labeling and/or check with your doctor or healthcare specialist to assure there is no interaction with the diaphragm.
- All methods of contraception must be used according to directions; however, no method can guarantee 100% results against pregnancy.

CONTRAINDICATIONS

- An association has been reported between diaphragm use and Toxic Shock Syndrome (TSS), a serious condition which can be fatal.
- Known sensitivities to silicone and/or prior history of Toxic Shock Syndrome.

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IMPORTANT— Continuous prolonged wearing of the diaphragm may encourage the growth of certain bacteria in the vagina. These bacteria can lead to symptoms of Toxic Shock Syndrome.

SÍNTOMAS DE TOXIC SHOCK SYNDROME (TSS)

- Súbito,��燒 (102 °F / 39 °C, o más)
- Náusea, vómitos y dolor abdominal
- Deshidratación
- Rash over much of body (may resemble sunburn)
- Weakness, tenderness and/or aching or pain of muscles and/or joints
- Sore throat, red eyes, and general feeling of weakness

If you have a sudden high fever and one or more of the above symptoms remove the diaphragm and call your doctor or other healthcare professional immediately.

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- With clean hands, place the previously washed diaphragm on a flat surface with the dome of the cup down. Add spermicidal gel or cream into the cup of the diaphragm.

With your forefinger, spread the gel around the inside of the cup, under and around the surface of the silicone lip. Applying a thin film of gel or cream will help to hold the diaphragm in place.

Remember, in order to be effective, your diaphragm must always be used with spermicidal gel or cream.

INTRODUCIÓN DEL DIAFRAGMA

2. The diaphragm can be inserted while standing with one leg on a low chair or stool, lying down, squatting, or sitting forward on the edge of a chair. Remember, however, that the position of the cervix and vaginal vault changes with each position that you take, so always be certain that the cervix is covered.

3. Hold the diaphragm with the dome down. Arcing Diaphragms are designed for insertion by pressing two fingers into the dome of the diaphragm together. Omniflex Diaphragms can be folded for insertion at any point. Now spread the lips of the vagina with one hand and gently insert the diaphragm (dome down) into the vagina along the rear wall as far as it will comfortably go. Your index finger can guide the diaphragm as you gently push it into place. Do not pinch the diaphragm.

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- Tuck the front rim of the diaphragm up into the pubic notch so it is pressed against the front wall of the vagina.

As an added precaution, feel the cervix through the silicone shield (it is shaped like the tip of a dented nose) to make certain that the diaphragm is securely in place and that the entire cervix is covered.

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6. Do not douche or remove the diaphragm for at least six hours after intercourse. The diaphragm should be removed as soon as this is possible. Removal of the diaphragm before six hours may increase the risk of pregnancy. Additional gel or cream should be applied if you desire to have intercourse again.

The diaphragm should never be left in place continuously, over 24 hours.

REMOVIMIENTO DEL DIAFRAGMA

7. To remove the diaphragm, insert your index finger, behind the front rim to break the suction. With your finger inside the rim pull the diaphragm down and withdraw it, but be careful not to puncture the silicone dome with your fingernail. Do not pinch the diaphragm—simply pull it gently with your finger.

LIMPIEZA

After each use, follow the cleaning instructions provided below for the Arcing and Omniflex Diaphragms.

NOTA:

CooperSurgical has followed a policy of supplying its diaphragms only to Licensed Practitioners.

RECOMENDACIÓN

When you have a routine physical examination, your doctor may find it necessary to refit you. Physiological changes may require a different size Arcing or Omniflex diaphragm than you initially needed.

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If you have questions or uncertain about the insertion, placement and use of your diaphragm, ask your doctor or healthcare specialist.

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