

PRO-CEPTION™ FERTILITY PAK (12 per Box)


MXFEPAK (Small) and MXFEPAKL (Large) • Directions for Use

DEVICE DESCRIPTION. Pro-CEPTION Fertility Pak is a disposable vaginal tampon that can easily be inserted to maintain a pool of seminal fluid near the cervical Os to help improve fertilization success rates. They can also be used following IUI or ICI procedures.


CAUTION. U.S. Federal law restricts this device to sale by or on the order of a physician.

PRO-CEPTION™ FERTILITY PAKS should be used only under the supervision of your personal physician. The optimum time for enhancing fertility using PRO-CEPTION FERTILITY PAK is during the fertile time of your menstrual cycle. You should consult your doctor as to this period.

DIRECTION FOR USE. The purpose of the PRO-CEPTION FERTILITY PAK to retain the semen within the vagina after ejaculation: hence its use is simple. In the average case, it is suggested that the PRO-CEPTION FERTILITY PAK be introduced soon after ejaculation by simply compressing the tampon and inserting it beyond the pubic bone — as far as your fingers can place it.

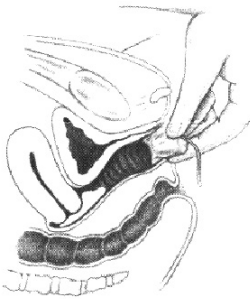

Method of inserting pak

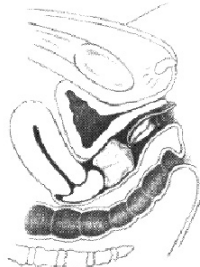

The Pro-CEPTION Fertility Pak in place, facilitating the formation of the seminal pool.


DO NOT REMOVE THE PLASTIC COVERING since it is vital in maintaining spermatozoal life in the vagina as long as possible. **USE AS IS.** The PRO-CEPTION FERTILITY PAK should be left in place 3 to 4 hours and then removed by the string. **DO NOT LEAVE** in place longer than four hours.

Do not wash or reuse this fertility pak.

Since each woman differs from another, it is recommended that you ask your doctor about the use of PRO-CEPTION FERTILITY PAK in your particular case. Physician will also advise you as to how many months you should use the PRO-CEPTION FERTILITY PAKS for a fair evaluation.

EXPLANATION OF SYMBOLS

REF Reorder Number

 Do Not Reuse

 Latex Free

 **ATTENTION:**
See instructions for use.

LOT Batch Code

CooperSurgical is a registered trademark of CooperSurgical, Inc.
Pro-CEPTION™ is a trademark of CooperSurgical, Inc.

© 2013 CooperSurgical, Inc.



Made in the USA

95 Corporate Drive, Trumbull, CT 06611 USA
Phone: (800) 243-2974, Fax: (800) 262-0105
www.coopersurgical.com

International
Phone: (203) 601-9818
Fax: (203) 601-4747

MIL0635 • Rev. B • 1/13