

MX5030 • Milex® TRIMO-SAN™ for Pessary Wearers

Instructions for Use

Rx Only

Please Read Carefully

A pessary is considered a foreign body in the vagina. For long-term pessary users, it is important to maintain the normal acidic pH of the vagina. TRIMO-SAN™, a deodorant vaginal gel may help pessary wearers restore and maintain a normal acidic pH of the vagina.

PURPOSE OF TRIMO-SAN

- To help restore and maintain normal vaginal acidity.
- To coat walls of the vagina with a lubricating film that may help reduce odor-causing bacteria, since bacteria can flourish in abnormal vaginal pH.

INFORMATION ABOUT TRIMO-SAN

1. TRIMO-SAN Gel will not stain skin or natural fabrics.
2. TRIMO-SAN Gel helps eliminate odors, which may be associated with wearing a pessary.
3. Dosage of TRIMO-SAN Gel may be adjusted from one-half an applicator to a full applicator to obtain the desired results.

WARNING: Chemicals in various preparations can interact with the pessary material resulting in discoloration or deterioration of the pessary.

TRIMO-SAN does not have a deleterious effect on any Milex pessary. Testing should be performed before using any other vaginal preparation with a pessary unless specifically prescribed by your health care professional.

TRIMO-SAN and PREGNANCY

TRIMO-SAN has **NOT** been tested for use during pregnancy.

- CooperSurgical does not recommend the use of TRIMO-SAN during pregnancy.
- If you wish to become pregnant, do not use TRIMO-SAN or any other personal hygiene product vaginally (douches, vaginal sprays, jellies or suppositories) for 6 hours prior to or following intercourse. Any product used within this time period could interfere with conception.

GENERAL INFORMATION

- New pessary wearers should use a 1/2 applicator of TRIMO-SAN three (3) times the first week after original insertion of the pessary.
- Use 1/2 applicator of TRIMO-SAN twice a week thereafter unless otherwise directed by your health care professional.
- Reaction or irritation caused by TRIMO-SAN is very rare, but should it occur, discontinue use.
- If symptoms persist or worsen, contact your health care professional.

THIS PRODUCT IS INTENDED FOR MULTIPLE USE BY ONE PATIENT ONLY.

PROPER USE OF THE MILEX JEL-JECTOR

1. Unscrew the cap from the tube.
2. Pierce the end of the tube with top of cap (a small amount of TRIMO-SAN may escape when the top of the tube is pierced).
3. Attach the threaded end of the Milex Jel-Jector to the TRIMO-SAN tube (see ①).
4. While holding the tube with the attached Milex Jel-Jector (see ②), squeeze the tube from the bottom until TRIMO-SAN reaches the specified dosage level.
5. Detach the Milex Jel-Jector from the tube. Gently insert the Jel-Jector approximately 1/2 inch into the vagina. Squeeze the bulb vigorously once to disperse the gel. Keep the bulb compressed until the Milex Jel-Jector is removed from the vagina.

NOTE: For those wearing an INFLATOBALL™ or CUBE Pessary, remove the pessary per the physician's instructions before using TRIMO-SAN.

6. Clean the Milex Jel-Jector immediately after using by flushing the Jel-Jector (compressing bulb) several times, holding the Jel-Jector under water (see ③).

NOTE: The "pin" hole at the top of the bulb is an air release hole. Do NOT tape over this hole or otherwise block this hole. This "pin" hole is not a defect in the bulb.




7. After each use, wipe the threads of the tube and inside the cap. Recap tightly.

IMPORTANT

IF YOU HAVE ANY OF THE FOLLOWING SIGNS OR SYMPTOMS, CONSULT A HEALTH CARE PROFESSIONAL

- A. A discharge, which is unusual in color, consistency or amount.
- B. A burning or itching sensation in the vaginal area.
- C. A foul odor associated with vaginal discharge.
- D. Any of the above especially if accompanied by unusual swelling or tenderness, fever, pains, or cramps in the lower abdomen.

NOTE: If this product is being used with a pessary, follow your health care professional's instructions explicitly. Consult your health care professional for all other medically-related advice.

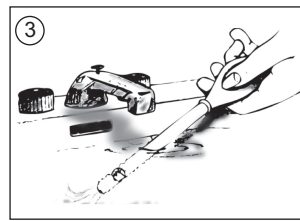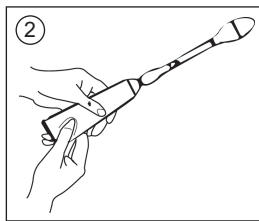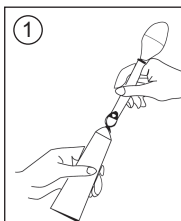

Milex®, TRIMO-SAN™ and INFLATOBALL™ are trademarks of CooperSurgical, Inc. CooperSurgical is a registered trademark of CooperSurgical, Inc. © 2018 CooperSurgical, Inc.

CooperSurgical

95 Corporate Drive
Trumbull, CT 06611 USA
37992-B • Rev. A • 04/18

www.coopersurgical.com

Phone: (800) 243-2974
Fax: (800) 262-0105

Made in the USA