# TRANSWARMER®

## INFANT TRANSPORT MATTRESS

with WarmGel®

**REF 20421**

**Intended Use:** Provides warmth during transport of infant within the hospital or between hospitals.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician.

**WARNINGS: Failure to follow these warnings can lead to severe burns**

- Routinely and regularly check the infant's skin for redness and the infant's core temperature
- Do not apply an additional heat source to the infant when using this product
- Consult a physician when using this product on premature newborns or infants
- Consult a physician when using this product in a non-transport environment
- Single use only. Do not reuse or reheat. Dispose of after use.
- Store at room temperature

**Instructions for Use:**

1. Locate metal disc and grasp with thumb and forefingers of both hands.
2. Flex (bend) disc rapidly until crystals begin to form.
3. Massage pack to soften and to increase the activation rate.
4. After pack is fully activated, indent center with palm to form a small nest.
5. Lay infant in nest on the non-woven fabric surface.
6. Check infant skin regularly for redness and monitor infant core temperature.

**Store at room temperature**

Temperature when activated: 40 °C (from 24 °C start)  
104 °F (from 75 °F start).

**Contents:** Sodium acetate (food grade), water and thickeners.

All contents are non-toxic; however, if contents come in contact with skin or eyes, flush with water. Do not swallow.

▼ PLACE INFANT ON OTHER SIDE ▼

Do not  
re-use

Not made with  
natural rubber  
latex

Caution

Consult instructions  
for use

**Manufactured for:**

**CoperSurgical**

95 Corporate Drive  
Trumbull, CT 06611 USA

Phone: (800) 243-2974

Fax: (800) 262-0105

International

Phone: +1 (203) 601-9818

Fax: +1 (203) 601-4747

**EC REP**

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands


Made in the USA

(01) 1 0017436 20421 3

| LETTER | REVISION                             | DATE | APPROVED          |
|--------|--------------------------------------|------|-------------------|
| G      | See Sheet 1 for revision information |      | See MasterControl |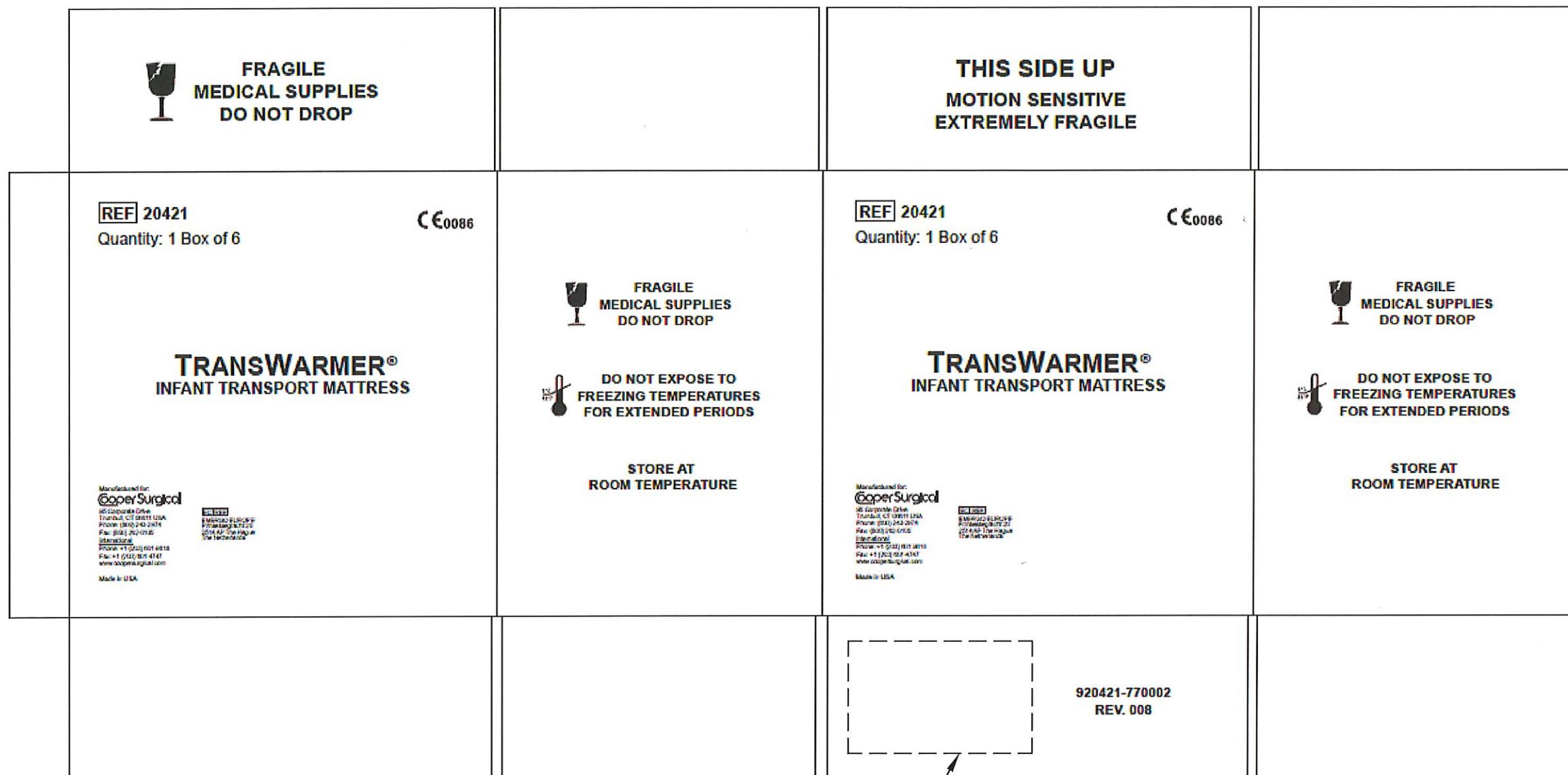

Location for Box Makers Stamp

- NOTES:
1. NO CHANGES IN MATERIALS AND/OR PROCESSES WITHOUT WRITTEN APPROVAL FROM CSI.
  2. OUTER SHIPPING CARTON (CASE) -10-1/8" x 8-3/8" x 11-1/2", 200#K
  3. PRINT COLOR: BLACK
  4. LOT NUMBER & EXPIRATION DATE TO BE CLEARLY IDENTIFIED ON OUTER CARTON
  5. ARTWORK DESIGNED IN ADOBE ILLUSTRATOR

UNLESS OTHERWISE SPECIFIED  
DIMENSIONS ARE IN INCHES

Frac. : +/- 1/64  
.XX : +/- .01  
.XXX : +/- .005  
ANGLE : +/- 1 DEG.

|  |              |          |  |       |                        |
|--|--------------|----------|--|-------|------------------------|
| <b>CONFIDENTIAL AND PROPRIETARY</b><br>This information is the property of CooperSurgical. It is submitted in strict confidence and is not to be disclosed or utilized without written permission. | DRAFTSMAN    | DATE     | DESCRIPTION:                               |       |                        |
|  | Dan Giardina | 08.23.04 | TransWarmer® Mattress Assembly (Outer Box) |       |                        |
|  | MFG ENGR     | -        | PART NUMBER:                               | 20421 |                        |
|  | APPROVED     |          | SIZE:                                      | B     | SHEET OF 2 4           |
|  | DG           | 10.18.04 | SCALE                                      | NONE  | FILE TYPE AUTOCAD .DWG |

**CooperSurgical**

95 CORPORATE DR. (203) 601-5200 PHONE  
TRUMBULL, CT 06611 (800) 262-0105 FAX

FRAGILE  
MEDICAL SUPPLIES  
DO NOT DROP

THIS SIDE UP  
MOTION SENSITIVE  
EXTREMELY FRAGILE

REF 20421

Quantity: 1 Box of 6

CE0086

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

Manufactured for:  
**CooperSurgical**

95 Corporate Drive  
Trumbull, CT 06611 USA  
Phone: (800) 243-2974  
Fax: (800) 262-0105  
International  
Phone: +1 (203) 601-9818  
Fax: +1 (203) 601-4747  
www.coopersurgical.com

Made in USA

CE REP  
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

FRAGILE  
MEDICAL SUPPLIES  
DO NOT DROP

 DO NOT EXPOSE TO  
FREEZING TEMPERATURES  
FOR EXTENDED PERIODS

STORE AT  
ROOM TEMPERATURE

REF 20421

Quantity: 1 Box of 6

CE0086

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

Manufactured for:  
**CooperSurgical**

95 Corporate Drive  
Trumbull, CT 06611 USA  
Phone: (800) 243-2974  
Fax: (800) 262-0105  
International  
Phone: +1 (203) 601-9818  
Fax: +1 (203) 601-4747  
www.coopersurgical.com

Made in USA

CE REP  
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands


FRAGILE  
MEDICAL SUPPLIES  
DO NOT DROP

 DO NOT EXPOSE TO  
FREEZING TEMPERATURES  
FOR EXTENDED PERIODS

STORE AT  
ROOM TEMPERATURE

920421-770002  
REV. 008

| LETTER | REVISION                             | DATE | APPROVED          |
|--------|--------------------------------------|------|-------------------|
| G      | See Sheet 1 for revision information |      | See MasterControl |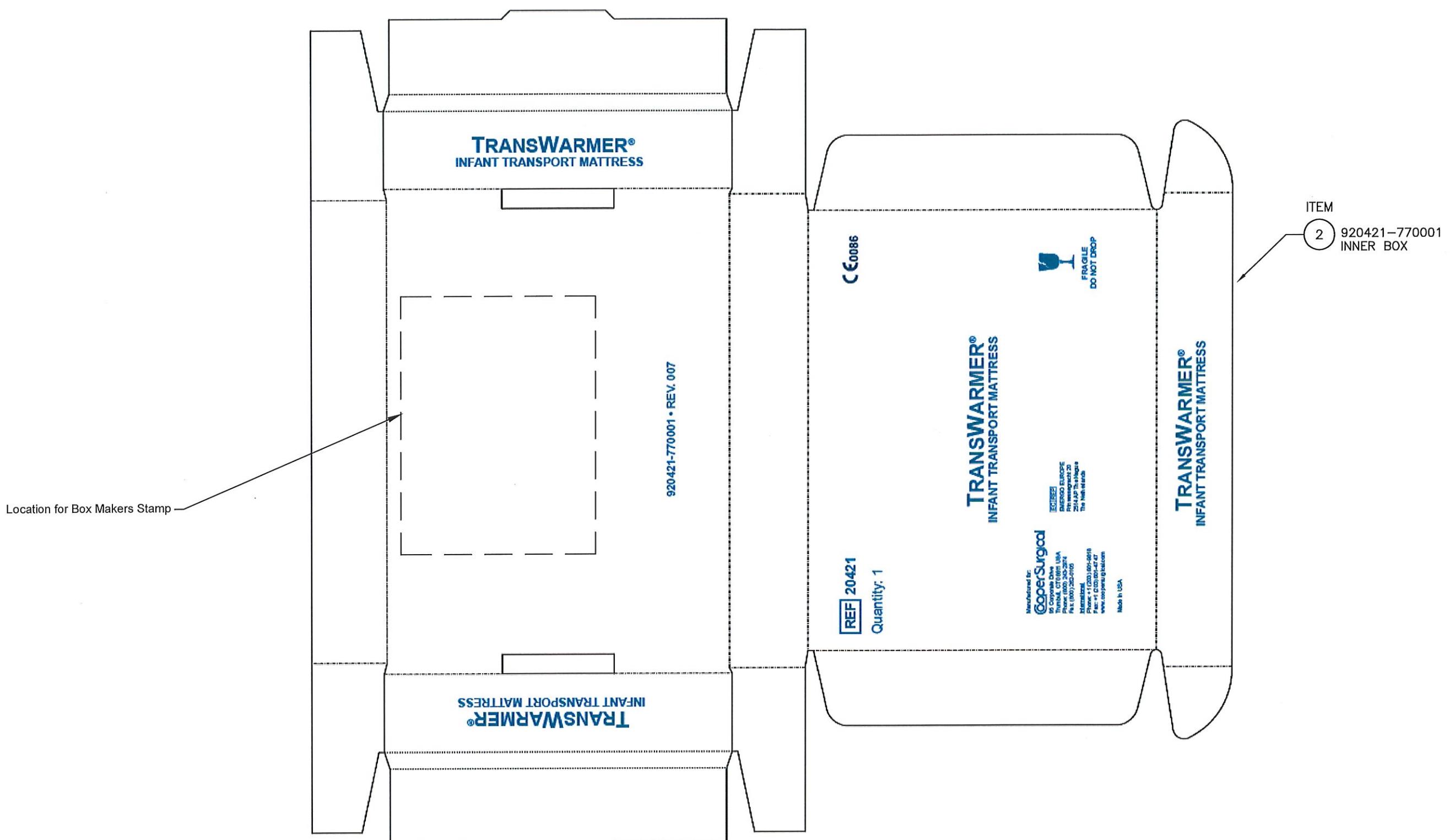

- NOTES:
1. NO CHANGES IN MATERIALS AND/OR PROCESSES WITHOUT WRITTEN APPROVAL FROM CSI.
  2. INNER BOX - 9-1/2" x 7-1/2" x 1-5/8", WHITE, D/C, 200#, B-FLUTE
  3. PRINT COLOR: PMS 541 BLUE
  4. LOT NUMBER & EXPIRATION DATE TO BE CLEARLY IDENTIFIED ON INNER BOX
  5. ARTWORK DESIGNED IN ADOBE ILLUSTRATOR

UNLESS OTHERWISE SPECIFIED  
DIMENSIONS ARE IN INCHES

|                    |
|--------------------|
| Frac. : +/-. 1/64  |
| Joe : +/- .00      |
| Jox : +/- .005     |
| Angle : +/- 1 DEG. |

MATERIAL:  
SEE NOTES

FINISH:  
SEE NOTES


| CONFIDENTIAL AND PROPRIETARY  |  | DRAFTSMAN    | DATE   |
|---|--|--------------|--|
| This information is the property of CooperSurgical. It is submitted in strict confidence and is not to be disclosed or utilized without written permission. |  | Dan Giardina | 08.23.04   |
| MFG ENGR  |  | —            | —  |
| APPROVED  |  | APPROVED     | PART NUMBER: 20421                                     |
| DG  |  | 10.18.04     | SIZE: B SHEET OF 3 4 SCALE NONE FILE TYPE AUTOCAD .DWG |

|  |  |  |
|--|--|--|
| <b>CooperSurgical</b>                                      | 95 CORPORATE DR.<br>TRUMBULL, CT 06611 | (203) 601-5200 PHONE<br>(800) 262-0105 FAX |
| DESCRIPTION:<br>TransWarmer® Mattress Assembly (Inner Box) |  |  |

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

920421-770001 • REV. 007

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

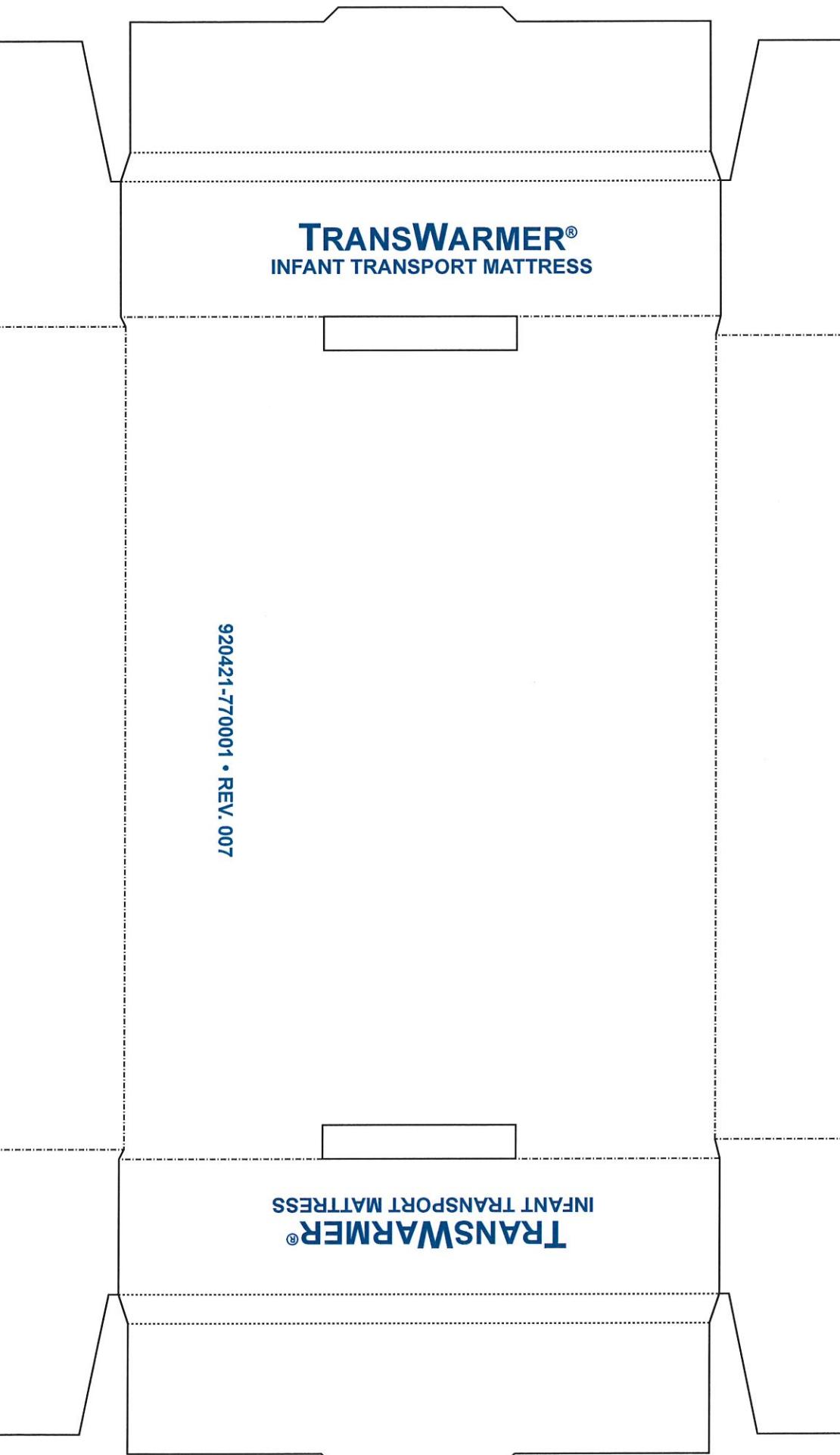**REF 20421**

Quantity: 1

**CE0086**

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

FRAGILE  
DO NOT DROP

Manufactured for:  
**CooperSurgical**  
99 Corporate Drive  
Trumbull, CT 06611 USA  
Phone: (800) 243-2874  
Fax: (800) 282-0105  
International  
Phone: +31 (20) 501-9818  
Fax: +31 (20) 501-4747  
www.coopersurgical.com

Made in USA

**TRANSWARMER®**  
INFANT TRANSPORT MATTRESS

| LETTER | REVISION                             | DATE | APPROVED          |
|--------|--------------------------------------|------|-------------------|
| G      | See Sheet 1 for revision information |      | SEE MASTERCONTROL |

5-1/2"

|   |  |
|---|--|
| <p>8-1/2</p> <p>Manufactured for:<br/><b>CooperSurgical</b></p> <p>95 Corporate Drive<br/>Trumbull, CT 06611 USA<br/>Phone: (800) 243-2974<br/>Fax: (800) 262-0105<br/><a href="http://www.coopersurgical.com">www.coopersurgical.com</a></p> <p>International:<br/>Phone: +1 (203) 601-9818<br/>Fax: +1 (203) 601-4747</p> <p>Made in the USA</p> <p>EC REP<br/>EMERGO EUROPE<br/>Prinsessegracht 20<br/>2514 AP The Hague<br/>The Netherlands</p> <p>920421-770003 • Rev. 003 • 09/18</p> | <p style="text-align: center;"><b>CooperSurgical</b></p> <p><b>REF 20421 • TransWarmer® Infant Transport Mattress with WarmGel® Instructions for Use (English)</b> <b>C E0086</b></p> <p><b>INTENDED USE:</b> Provides warmth during transport of infant within the hospital or between hospitals.</p> <p><b>CAUTION:</b> U.S. Federal law restricts this device to sale by or on the order of a physician.</p> <p><b>WARNINGS:</b> Failure to follow these warnings can lead to severe burns</p> <ul style="list-style-type: none"> <li>• Routinely and regularly check the infant's skin for redness and the infant's core temperature</li> <li>• Do not apply an additional heat source to the infant when using this product</li> <li>• Consult a physician when using this product on premature newborns or infants</li> <li>• Consult a physician when using this product in a non-transport environment</li> <li>• Single use only. Do not reuse or reheat. Dispose of after use.</li> <li>• Store at room temperature</li> </ul> <p><b>INSTRUCTIONS FOR USE</b></p> <p><b>Do Not Reuse • Disposable • Store at room temperature</b></p> <p>Temperature when activated: 40 °C (from 24 °C start), 104 °F (from 75 °F start)</p> <p><b>To Activate:</b></p> <ol style="list-style-type: none"> <li>1. Locate metal disc and grasp with thumb and forefingers of both hands.</li> <li>2. Flex (bend) disc rapidly until crystals begin to form.</li> <li>3. Massage pack to soften and to increase the activation rate.</li> <li>4. After pack is fully activated, indent center with palm to form a small nest.</li> <li>5. Lay infant in nest on the non-woven fabric surface.</li> <li>6. Check infant skin regularly for redness and monitor infant core temperature.</li> </ol> <p><b>CONTENTS</b></p> <ul style="list-style-type: none"> <li>• Sodium acetate (food grade), water and thickeners.</li> <li>• Do not swallow.</li> <li>• All contents are non-toxic; however, if contents come in contact with skin or eyes, flush with water.</li> </ul> <p style="text-align: center;">1</p> |
|---|--|

FRONT PAGE  
(ENGLISH TITLE FACING OUT)

920421-770003 • Rev. 003 • 09/18

BACK PAGE

## NOTES:

1. NO CHANGES IN MATERIALS AND/OR PROCESSES WITHOUT WRITTEN APPROVAL FROM CSI.
2. PAPERSTOCK: 3 SHEETS, 8-1/2" X 11", 20 LB BOND, WHITE
3. PRINT COLOR: BLACK, 2 SIDES
4. ORDER OF LANGUAGES: ENGLISH, GERMAN, SPANISH, FRENCH, ITALIAN
5. FINISHING: FINAL FOLD TO 5-1/2" X 8-1/2", SADDLESTITCH, WITH ENGLISH TITLE FACING OUT, BOOKLET.
6. ARTWORK DESIGNED IN ADOBE INDESIGN

| UNLESS OTHERWISE SPECIFIED DIMENSIONS ARE IN INCHES  | MATERIAL: | FINISH:   |
|--|-----------|-----------|
| Frac. : +/- . <sup>1</sup> / <sub>64</sub><br>JOG : +/- . <sup>1</sup> / <sub>32</sub><br>JOX : +/- . <sup>1</sup> / <sub>.005</sub><br>ANGLE : +/- 1 DEG. | SEE NOTES | SEE NOTES |

| CONFIDENTIAL AND PROPRIETARY  | DRAFTSMAN    | DATE         |
|---|--------------|--------------|
| This information is the property of CooperSurgical. It is submitted in strict confidence and is not to be disclosed or utilized without written permission. | Dan Giardina | 08.23.04     |
| MFG ENGR  | —            | —            |
| PART NUMBER   | 20421        |              |
| APPROVED  | DG           | 10.18.04     |
| SIZE  | B            | SHEET OF 4 4 |
| SCALE   | NONE         |              |
| FILE TYPE   | AUTOCAD .DWG |              |

|  |  |  |
|--|--|--|
| <b>CooperSurgical</b>  | 95 CORPORATE DR.<br>TRUMBULL, CT 06611 | (203) 601-5200 PHONE<br>(800) 262-0105 FAX |
| DESCRIPTION:<br>TransWarmer® Mattress Assembly (IFU, multilingual) |  |  |

**INTENDED USE:** Provides warmth during transport of infant within the hospital or between hospitals.

**CAUTION:** U.S. Federal law restricts this device to sale by or on the order of a physician.

**WARNINGS: Failure to follow these warnings can lead to severe burns**

- Routinely and regularly check the infant's skin for redness and the infant's core temperature
- Do not apply an additional heat source to the infant when using this product
- Consult a physician when using this product on premature newborns or infants
- Consult a physician when using this product in a non-transport environment
- Single use only. Do not reuse or reheat. Dispose of after use.
- Store at room temperature

**INSTRUCTIONS FOR USE**

**Do Not Reuse • Disposable • Store at room temperature**

Temperature when activated: 40 °C (from 24 °C start), 104 °F (from 75 °F start)

**To Activate:**

1. Locate metal disc and grasp with thumb and forefingers of both hands.
2. Flex (bend) disc rapidly until crystals begin to form.
3. Massage pack to soften and to increase the activation rate.
4. After pack is fully activated, indent center with palm to form a small nest.
5. Lay infant in nest on the non-woven fabric surface.
6. Check infant skin regularly for redness and monitor infant core temperature.

**CONTENTS**

- Sodium acetate (food grade), water and thickeners.
- Do not swallow.
- All contents are non-toxic; however, if contents come in contact with skin or eyes, flush with water.

# 20421 • TransWarmer® Infant Transport Mattress with WarmGel® Instructions for Use • English (Continued)

---

## EXPLANATION OF SYMBOLS

|            |                                    |               |  |
|------------|------------------------------------|---------------|--|
| <b>REF</b> | Reorder number                     |               | Consult instructions for use                         |
| <b>LOT</b> | Batch code                         |               | Do not re-use  |
|            | Not made with natural rubber latex |               | Caution  |
|            | Use-by date                        | <b>EC REP</b> | Authorized Representative in the European Community. |

MR Conditional

The **TransWarmer® Infant Transport Mattress** was determined to be MR-conditional.

Non-clinical testing demonstrated that the TransWarmer Infant Transport Mattress is MR Conditional. A patient with this device can be scanned safely immediately after placement under the following conditions:

### **Static Magnetic Field**

- Static magnetic field of 3-Tesla or less
- Maximum spatial gradient magnetic field of 720-Gauss/cm or less

### **MRI-Related Heating**

In non-clinical testing, the TransWarmer Infant Transport Mattress produced the following temperature rise during MRI performed for 15-min of scanning (i.e., per pulse sequence) in the 3-Tesla (3-Tesla/128-MHz, Excite, HDx, Software 14X.M5, General Electric Healthcare, Milwaukee, WI) MR system:

*Highest temperature change +1.6 °C (2.88 °F)*

Therefore, the MRI-related heating experiments for the TransWarmer Infant Transport Mattress at 3-Tesla using a transmit/receive RF body coil at an MR system reported whole body averaged SAR of 2.9 -W/kg (i.e., associated with a calorimetry measured whole body averaged value of 2.7-W/kg) indicated that the greatest amount of heating that occurred in association with these specific conditions was equal to or less than +1.6 °C (2.88 °F).

### **Artifact Information**

MR image quality may be compromised if the area of interest is in the exact same area or relatively close to the position of the TransWarmer Infant Transport Mattress. Therefore, optimization of MR imaging parameters to compensate for the presence of this device may be necessary. The maximum artifact size (i.e., as seen on the gradient echo pulse sequence) extends approximately 35-mm relative to the size and shape of the TransWarmer Infant Transport Mattress.

|                   |                       |                       |                       |                       |
|-------------------|-----------------------|-----------------------|-----------------------|-----------------------|
| Pulse Sequence    | T1-SE                 | T1-SE                 | GRE                   | GRE                   |
| Signal Void Size  | 4,049-mm <sup>2</sup> | 3,600-mm <sup>2</sup> | 6,031-mm <sup>2</sup> | 4,616-mm <sup>2</sup> |
| Plane Orientation | Parallel              | Perpendicular         | Parallel              | Perpendicular         |

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