

Milex™ Pessary Fitting Kit (English)
Pessar-Anpassungskit Milex™ (Deutsch / German)
Equipo de colocación de pesarios Milex™ (Español / Spanish)
Kit de pose de pessaire Milex™ (Français / French)
Kit pessari di prova Milex™ (Italiano / Italian)

**Fitting Instructions for the
 Health Care Professional**



**FITTING
 PESSARIES ARE
 NOT
 TO BE
 WORN MORE
 THAN
 15 MINUTES!**



CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.



Latex Free



Silicone
 (Latex Free)

CooperSurgical

Made in the USA

95 Corporate Drive
 Trumbull, CT 06611 USA
 Phone: (800) 243-2974
 Fax: (800) 262-0105
 www.coopersurgical.com

International
 Phone: (203) 601-9818
 Fax: (203) 601-4747



EMERGO EUROPE
 Molenstraat 15
 2513 BH, The Hague
 The Netherlands



CooperSurgical, Inc.
 95 Corporate Drive
 Trumbull, CT 06611 USA

CooperSurgical

Made in the USA

95 Corporate Drive
Trumbull, CT 06611 USA
Phone: (800) 243-2974
Fax: (800) 262-0105
www.coopersurgical.com

International
Phone: (203) 601-9818
Fax: (203) 601-4747



EMERGO EUROPE
Molenstraat 15
2513 BH, The Hague
The Netherlands