
CooperSurgical®

ALLY UPS® Adapter Drape

Adapter Drape/Adapterhoes/Protection de l'adaptateur/Adapter und Abdeckmaterial/Telo dell'adattatore/어댑터 드레이프/Cubierta del adaptador/Adaptör Örtüsü

Instructions for Use/Gebruiksaanwijzingen/
Notice d'utilisation/Gebrauchsanweisung/
Istruzioni per l'uso/사용법 설명서/
Instrucciones de uso/Kullanım Talimatları

REF AU-AD

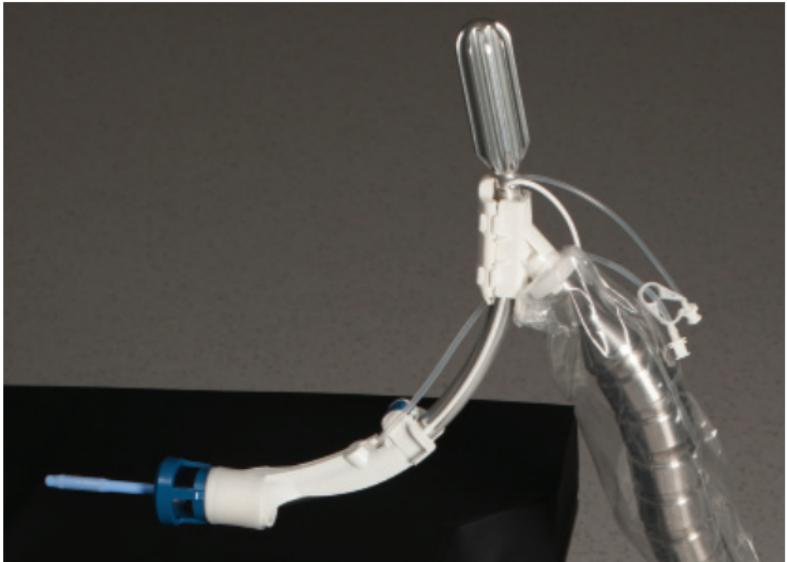€ 2797

Table of Contents (ENGLISH)

Caution2
Indications for Use2
Contraindications2
Warnings2
Device Description3
Instructions for Use3
Before Attaching the Adapter Drape3
Attaching Adapter and Draping the ALLY UPS® Arm ..	.3
Before Attaching the Manipulator Handle5
Attaching the Manipulator Handle6
Attaching the RUMI® II Manipulator7
Attaching the Advincula Arch™ Manipulator8
Removing the Manipulator Handle from the Adapter ..	.9
Removing the Adapter Drape from the ALLY UPS® ..	.9
Disposal of Adapter Drape9
Explanation of Symbols10

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

INDICATIONS FOR USE

The Adapter Drape for the ALLY Uterine Positioning System® (ALLY UPS®) is intended to assist the surgical staff in mounting, positioning and holding uterine manipulators during gynecological laparoscopic surgical procedures. It is intended for use by trained operating room personnel in an operating room environment.

CONTRAINDICATIONS

CooperSurgical Manipulator Handles should not be used in patients who are pregnant or who are suspected of being pregnant, planned gamete intrafallopian transfer procedures, in patients who have an IUD in place, in patients with suspected pelvic infections and in cases where the surgeon deems it inadvisable or finds it difficult to insert the silicone tip into the cervix or uterus.

WARNINGS

- Failure to properly follow instructions for the Adapter Drape (Adapter), including instructions supplied with the ALLY Uterine Positioning System® (ALLY UPS®) and one of the following manipulator handles: the RUMI® II Uterine Manipulator Handle (UMH650), Advincula Arch™ Uterine Manipulator Handle (UMH750) may lead to injury and result in improper functioning of the system.
- The Adapter Drape is only intended for use with the RUMI® II (UMH650) or Advincula Arch (UMH750) and the ALLY UPS®. Use of these devices with any other Manipulator is not recommended, and could lead to injury and improper functioning of the device.
- ALWAYS position the patient, the OR table, and the ALLY UPS® prior to attaching the ALLY UPS® Flexible Arm/Adapter to the Manipulator Handle.
- ALWAYS have the patient under general endotracheal anesthesia when the ALLY UPS® is attached to the Manipulator Handle.

DEVICE DESCRIPTION

The ALLY UPS® Adapter Drape (Part Number AU-AD) is a sterile, single-use disposable device that is used to attach the RUMI® II Uterine Manipulator Handle, or the Advincula Arch™ Uterine Manipulator Handle to the ALLY Uterine Positioning System® exclusively.

Photo 1

RUMI® II Connected to the ALLY UPS® Adapter Drape

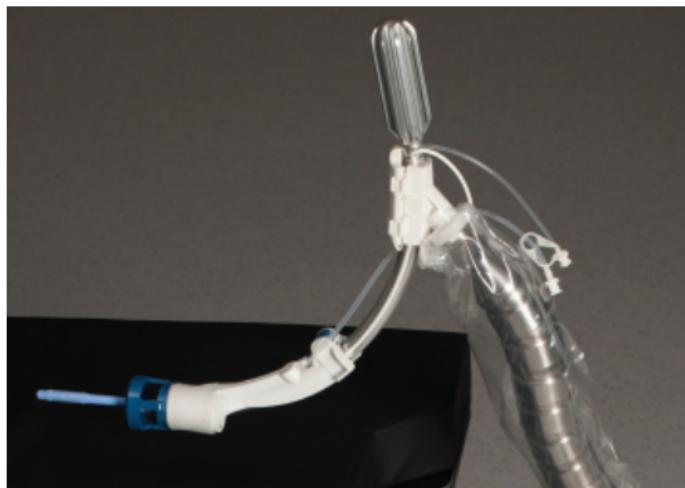
Photo 2

Advincula Arch Connected to the ALLY UPS® Adapter Drape

INSTRUCTIONS FOR USE

Before Attaching the Adapter Drape

1. Be familiar with the ALLY UPS® Instructions for moving and locking the Arm in position.
2. Prior to attaching the Adapter Drape to the ALLY UPS® Flexible Arm, make sure the patient is positioned and prepped.
3. Position the Flexible Arm so that the distal end is vertical.

Attaching Adapter and Draping ALLY UPS® Arm

1. Remove the Adapter Drape from the package. DO NOT pull apart the drape.
2. Locate the two Pinch Clips inside. Align and press them into the corresponding metal interface at the distal end of the flexible arm and confirm that it locks in place.

ALLY UPS® Adapter Drape (ENGLISH)

Photos 3a-b show the connection of the adapter onto the top of the ALLY UPS® Flexible Arm.

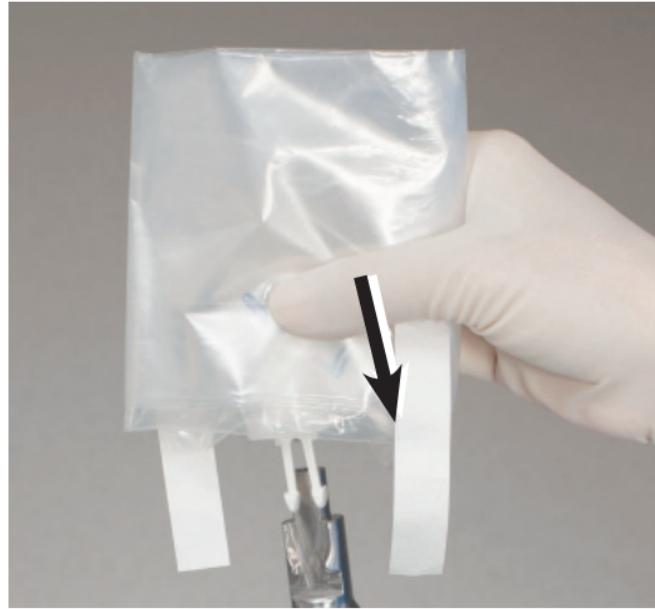
Photo 3a

Installing the Adapter Drape with the Pinch Clips

Photo 3b Pinch Clips in the Seated Position

3. Once the Adapter is locked in place use the Pull Tabs to slide the sterile Drape all the way down the ALLY UPS® Flexible Arm, until it reaches the Main Unit. The ALLY UPS® is now fully draped. Refer to Photos 4a-d.

ALLY UPS® Adapter Drape (ENGLISH)

- 4.** Ensure that the Adapter is securely in place on the arm.

Photo 4a
Pulling on the Tabs

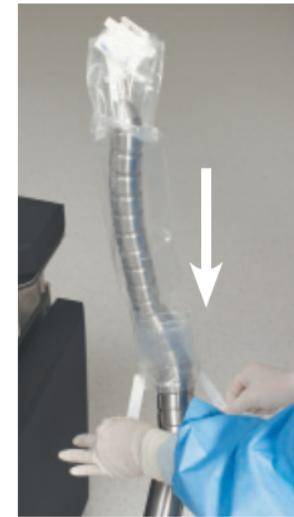
Photo 4b
Drape Partially
Pulled Down

Photo 4c
Top of the Drape
with the Adapter

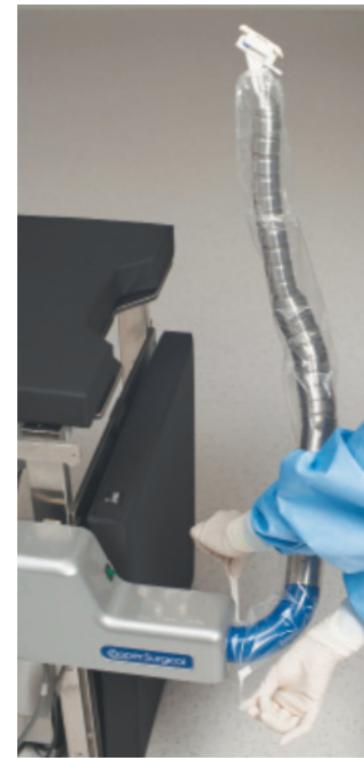
Photo 4d
Bottom of the Drape

Adapter Drape Fully Extended on the ALLY UPS®' Flexible Arm

Before Attaching the Manipulator Handle

Ensure that:

- 1.** The Manipulator is inserted.
- 2.** The draped ALLY UPS® Arm and Adapter are placed in approximate position for attachment near the proximal end of the Manipulator Handle.

Attaching the Manipulator Handle

1. Open the Latch on the Adapter Drape (Photo 5)

Photo 5 Opening the Latch

2. Use of the different manipulators require the Adapter to be rotated about the distal end of the Flexible Arm. Notice the labelling on the Adapter Drape indicating the orientation of these handles (see Photo 6.)

Photo 6 Images on the Adapter Drape

3. Determine which Manipulator is being used. If rotating the Adapter Drape is necessary to align with the correct Manipulator Handle position as shown in Photo 6, rotate the Adapter, not the Flexible Arm.

If the RUMI® II
Manipulator is
intended to be used,
continue to the
next pag.

Or, if the Advincula
Arch™ Manipulator
is intended to be
used, continue to
page 7.

Attaching the RUMI® II Manipulator

NOTE: Before attaching the Adapter on the Flexible Arm to the Manipulator identify the locating points (A and B). As shown in Photo 7, Pivot Point “A” is located on the underside of the RUMI® II Manipulator. Mating Hole “B” is inside the cradle of the Adapter.

4. With one hand hold the Manipulator Handle steady (with the handle upwards), hold the Adapter with the other hand, and press the Foot Pedal.
5. Move the Flexible Arm in such a way that the Manipulator Handle is seated in the Adapter’s cradle and “A” and “B” are aligned and mated. Take care to minimize the movement of the Manipulator Handle within the patient.
6. Close and lock the Latch. Ensure that the Manipulator Handle is securely attached to the Adapter Drape. See Photo 9.
7. Remove pressure on the Foot Pedal to lock the Flexible Arm in this position.

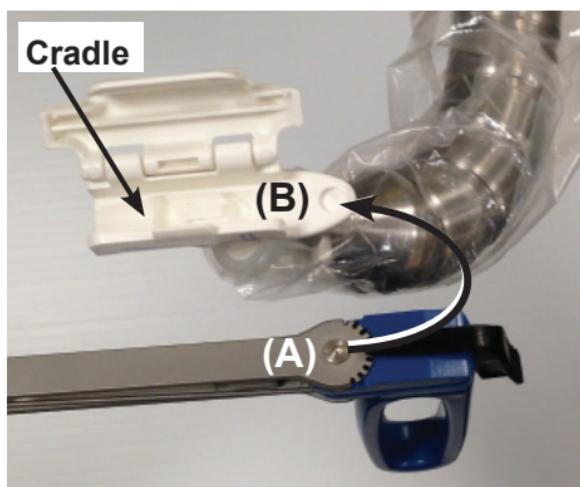
Photo 7 Examining the RUMI® II Manipulator

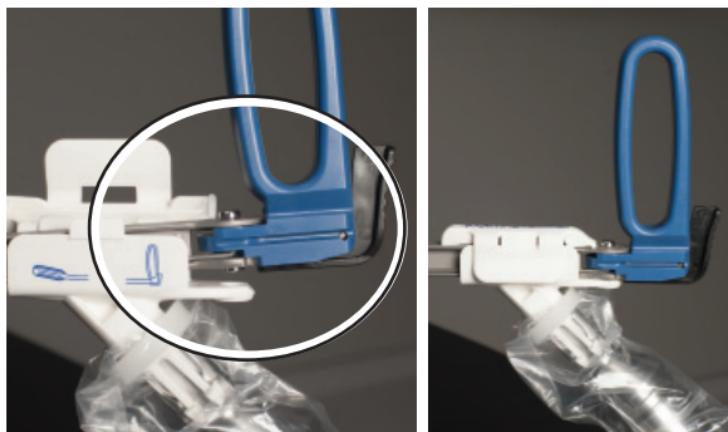
Photo 8
Pivot Point on the
Handle lined up with the
RUMI® II Manipulator
Mating Hole in the Adapter

Photo 9
Closed Position

Attaching the Advincula Arch Manipulator

NOTE: Before attaching the Adapter on the Flexible Arm to the Manipulator, identify the locating points (A and B). As shown in Photo 10, Mating Part "A" is located on the underside of the Advincula Arch Manipulator. Point "B" is inside the cradle of the Adapter.

4. Hold the Manipulator Handle steady with one hand, hold the Adapter with the other hand, and press the Foot Pedal.
5. Move the Flexible Arm in such a way that the Manipulator Handle is seated in the Adapter's cradle and "A" and "B" are aligned and mated. Take care to minimize the movement of the manipulator within the patient.

See Photos 10 and 11 for installing the Advincula Arch™ Manipulator.

6. Close and lock the Latch. Ensure that the manipulator is securely attached to the Adapter Drape. See Photo 12.
7. Remove pressure on the Foot Pedal to lock the Flexible Arm in this position.

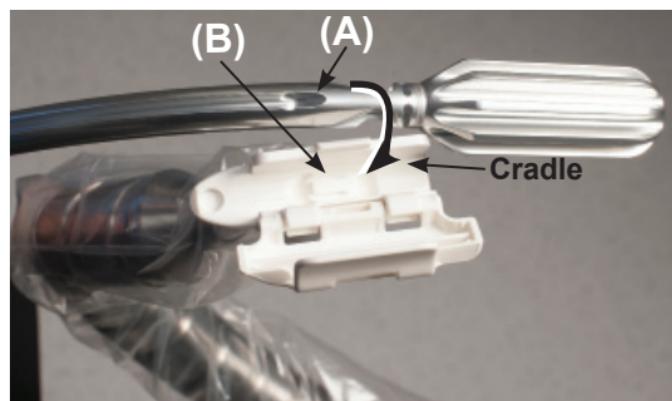
Photo 10 Examining the Advincula Arch™ Manipulator

Photo 11 Advincula Arch™ Manipulator in Position

Photo 12 Closed Position

Removing the Manipulator Handle from the Adapter

To detach the Manipulator Handle, unlatch and open the Latch on the Adapter while holding the Manipulator Handle. Actuate the Flexible Arm and move the Adapter away from the Manipulator Handle. Take care not to excessively move the Manipulator inside the vagina.

Removing the Adapter Drape from the ALLY UPS®

To detach the Adapter from the ALLY UPS® Flexible Arm after the procedure is complete, squeeze the two Pinch Clips together with one hand (see Photo 13) while lifting the Adapter upward with the other hand without twisting. Then pull the Adapter Drape up off of the Flexible Arm to allowing cleaning of the Arm. Refer to the ALLY UPS® Instructions for Use for cleaning instructions.

Photo 13
Removing the Adapter Drape from the ALLY UPS® Arm

DISPOSAL OF ADAPTER DRAPE

Dispose of in accordance with all applicable Federal, State and local Medical / Hazardous waste practices once the procedure is completed.

EXPLANATION OF SYMBOLS

Reorder number

Batch code

Use-by date

Sterilized using ethylene oxide

Do not re-use

Do not resterilize

Consult instructions for use

Caution

Do not use if package is damaged

Rx Only

Caution: U.S. Federal law restricts this device to sale by or on the order of a physician.

Not made with natural rubber latex

Authorized Representative in the European Community.

Product conforms to the Medical Device Directive 93/42/EEC

Manufacturer

Advincula Arch™ is a trademark of CooperSurgical, Inc.

RUMI® II and ALLY Uterine Positioning System® are registered trademarks of CooperSurgical, Inc.

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© 2021 CooperSurgical, Inc.
CooperSurgical, Inc.'in bir ticari markasıdır CooperSurgical, Inc. firmasının ticari markalarıdır.
RUMI® II and ALLY Uterine Positioning System® kayıtlı CooperSurgical, Inc.'in ticari markalarıdır