Arch Koh-Efficient™

Vaginal Fornices Delineator & Colpo-Pneumo Occluder™

for use with the Advincula Arch™ Uterine Manipulator Handle

Koh-Efficient™ for Electrocautery

Koh-Efficient™ for Harmonic Scalpel and Laser

2.5 cm • 3.0 cm • 3.5 cm • 4.0 cm

Instructions For Use (English)

CE₂₇₉₇

STERILE
SINGLE-USE

DEVICE DESCRIPTION

The Arch Koh-Efficient™ is a sterile, single-use vaginal fornices delineator with an integrated Colpo-Pneumo Occluder™. It is made of medical grade plastics with an Ultem® polymeric rigid KOH Cup™. Harmonic Scalpel/Laser compatible versions (KCS-ARCH) are equipped with a stainless steel distal surface. Not made with natural rubber latex.

The Arch Koh-Efficient (Koh-Efficient) is compatible with the Advincula Arch™ Uterine Manipulator Handle (Handle) and the RUMI® Uterine Manipulator Tip (Tip). These three products, used in conjunction with the Cervical Sizer (Sizer), enable unobstructed placement of a standard Tip into the uterus. Once in place, the Koh-Efficient is advanced along the shaft of the Handle and locks the KOH Cup into place around the cervix.

Advincula Arch Handle
(UMH750)

RUMI Uterine Manipulator Tip

Arch Koh-Efficient
(KC-ARCH or
KCS-ARCH)

Cervical Sizer
(KC-SIZER)

WARNINGS

- Contents supplied sterile. Do not use if sterile barrier is damaged.
- The integrated Colpo-Pneumo Occluder is designed to accommodate no more than 180 cc of inflation.
- The Koh-Efficient is single-use only, and should NEVER be reused
- Using an incorrect size KOH Cup could result in patient injury.
- The Ultem KOH Cup (KC-ARCH) is not intended to be used with Harmonic or other types of ultrasonic or laser energy sources.
- The stainless steel KOH Cup (KCS-ARCH) is not intended to be used with electrocautery. Use of electrocautery may result in inadvertent injury to adjacent structures.
- Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient. Dispose of in accordance with all applicable Federal, State, and local Medical/Hazardous waste practices.

CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician
- The Arch Koh-Efficient is to be used ONLY with the Advincula Arch™ Handle (UMH750).

INTENDED USE / INDICATIONS FOR USE

The Koh-Efficient is a combination KOH Cup™ and Colpo-Pneumo Occluder™ that is indicated for use in laparoscopic procedures where it is desirable to locate the position of the vaginal fornices, typically during a total laparoscopic hysterectomy or supra-cervical laparoscopic hysterectomy.

CONTRAINDICATIONS


The Koh-Efficient should not be used in patients who are pregnant or who are suspected of being pregnant, in patients who have an IUD in place, in patients with suspected pelvic infections or in cases where the surgeon deems it inadvisable and finds it difficult to insert the silicone tip into the cervix or uterus.

PRECAUTIONS

Always confirm that the Koh-Efficient is locked in place prior to starting the colpotomy.



INSTRUCTIONS FOR USE

Fully Assembled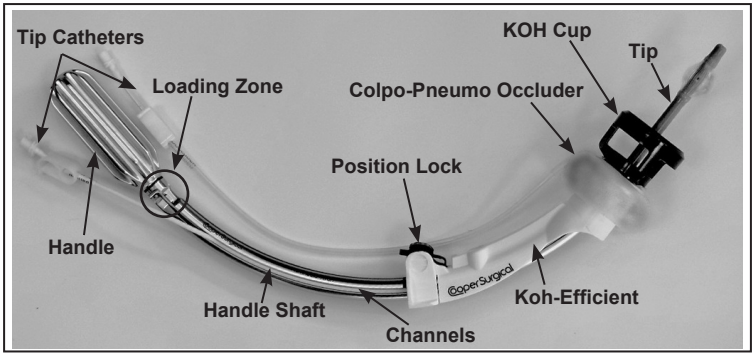


For complete information on how to prepare Arch Koh-Efficient System, please refer to the Instructions for Use for the Advincula Arch Handle (UMH750), the RUMI Uterine Manipulator Tips and the Cervical Sizer (KC-SIZER).

1. Prepare the Handle for use by following the Handle's Instructions for Use.
2. Attach the proper size Tip to the Handle. **Press the Tip Catheters completely into the channel positions along the Handle Shaft.** Confirm that the tubes protrude out of the channels at the Loading Zone.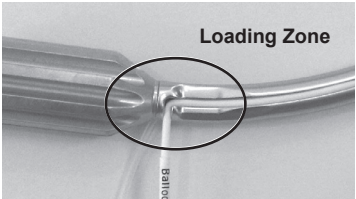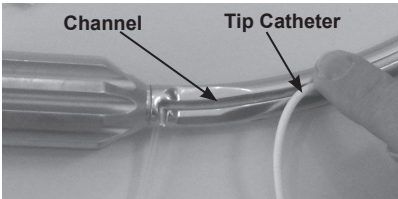


3. Using the Cervical Sizer, determine the proper size Koh-Efficient.


4. Slide the Koh-Efficient over the Tip and down the Handle Shaft.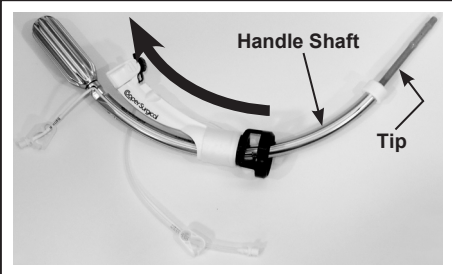

5. Position the Koh-Efficient's Squeeze Clamp over the Loading Zone and press firmly on the Thumb Pad, in the direction of the arrow, to "snap" into place.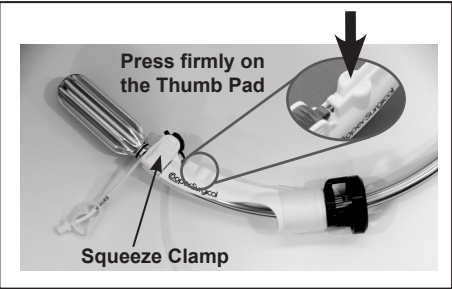

6. Test inflate the Colpo-Pneumo Occluder™ with 20 to 60 cc of sterile saline, then deflate prior to patient insertion.
7. Insert the Tip into the uterus and inflate per the Tip's Instructions for Use.
8. Press on the Push Pads to slide the Koh-Efficient up the Handle until it locks into place.
- Note: The last 1/4 of an inch of travel will have increased resistance.*

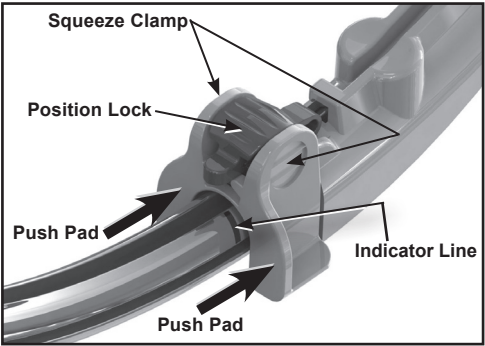9. Press the Position Lock down until you hear it lock. This confirms the Arch Koh-Efficient is in place.
10. Confirm the locked position by pulling back on the Push Pad and visually verifying the Indicator Line on the Shaft.
11. Confirm that the distal end of the KOH Cup™ resides in the fornices.
- Note:** If necessary to release and slide back the Koh-Efficient, unlock the position lock (by lifting the blue tab) and then squeeze the Squeeze Clamp to slide.

Arch Koh-Efficient™ • Instructions for Use • English (continued)

- 12. Just prior to making colpotomy incisions, inflate the Colpo-Pneumo Occluder balloon with 60 to 120 cc of sterile saline or water (do not use air).
- 13. After the procedure, dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

Once Colpotomy is complete, deflate the Colpo-Pneumo Occluder (Balloon) prior to removing the system.

EXPLANATION OF SYMBOLS

REF Reorder Number **LOT** Batch Code  Use-by date

STERILE EO Sterilized Using Ethylene Oxide

R_x Only U.S. Federal law restricts this device to sale by or on the order of a physician.

 Do not re-use

 Caution

 Consult instructions for use

 Do not use if package is damaged

 Do not re-sterilize

 Not made with natural rubber latex

EC REP Authorized Representative in the European Community.

 Manufacturer

The marks bearing the symbols “TM” and “®” are trademarks and registered trademarks of CooperSurgical, Inc.
Ultem® is a registered trademark of SABIC Innovative Plastics.
CooperSurgical is a registered trademark of CooperSurgical, Inc.

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As marcas que apresentam os símbolos "TM" e "®" são marcas comerciais e marcas registadas da CooperSurgical, Inc.
CooperSurgical é uma marca registada da Coopersurgical, Inc.