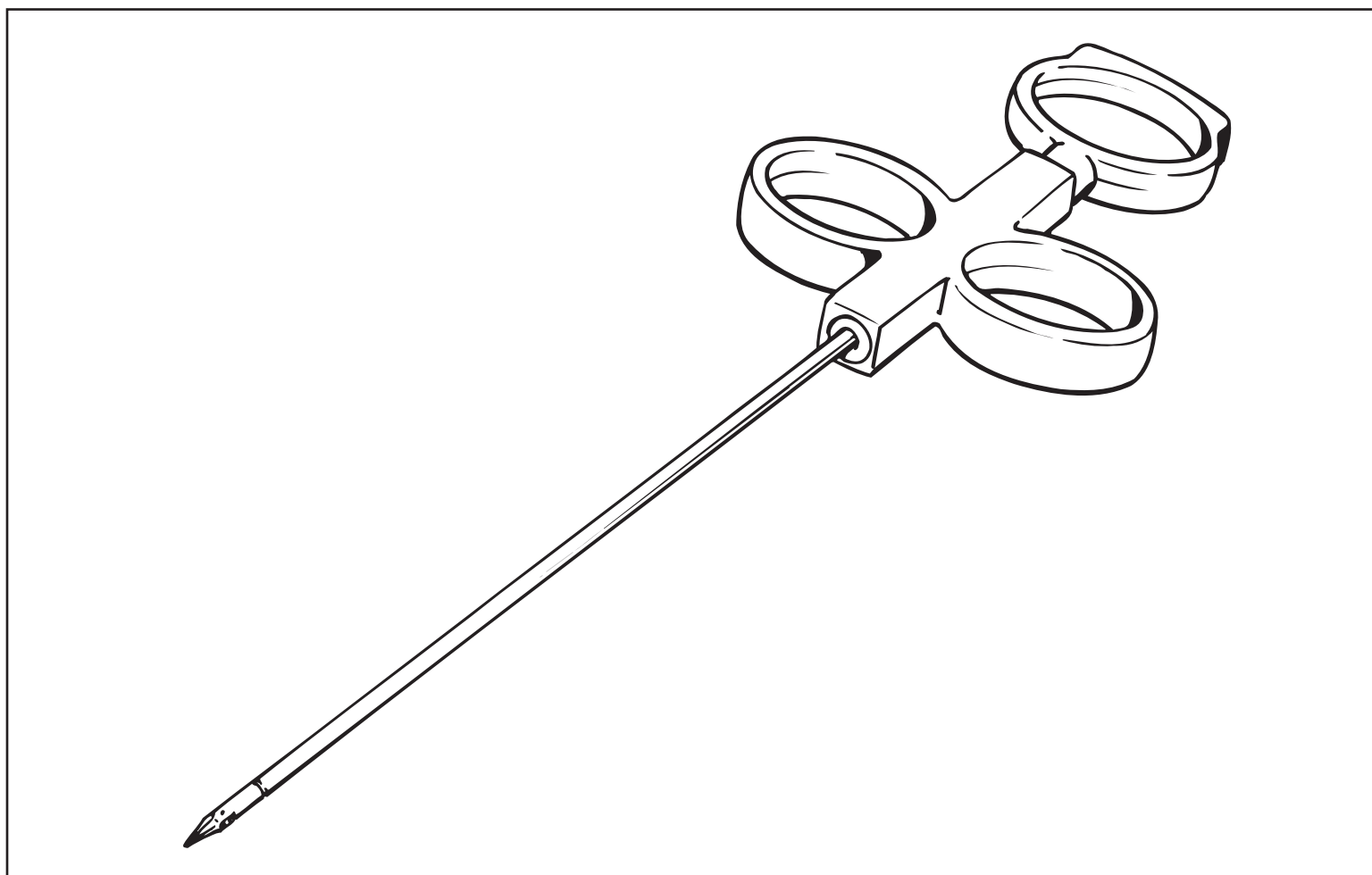
CTI-SP • Suture Passer
Instructions for Use (English)

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Instructions for Use (English)

DEVICE DESCRIPTION

The Carter-Thomason® Suture Passer is a sterile, single-case use, hand-held suture grasping device that is designed to pass suture through soft tissue.

CAUTION:

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- Dispose of in accordance with all applicable Federal, State, and local Medical / Hazardous waste practices.

INDICATION FOR USE

The device is intended to pass suture through soft tissues of the body during endoscopic/laparoscopic surgery. It is to be used only by surgeons trained in endoscopic/laparoscopic surgery.

PRECAUTIONS

- A surgeon should not begin clinical use of the device without a full understanding of the directions for use.
- The device is not recommended for use in surgical procedures where the position of the needle tip cannot be clearly determined.
- The jaws of the device must be closed completely in order to form a needle point to pass suture through tissue. Passage of the device through tissue without pressure on the thumb ring may result in losing grasp of the suture or the inadvertent capture of tissue.
- The suture passer jaws can injure internal tissues and/or any personnel in contact with the pointed end of the instrument. The suture passer jaw should be protected at all times with the protective cap when the instrument is not in use.

CARTER-THOMASON SUTURE PASSER INSTRUCTIONS FOR USE

The suture passer has two operating positions: Jaws Open and Jaws Closed (see illustrations).

1. To suture, open the needle point tip to grasp suture by pulling back on the thumb ring. We recommend the use of Vicryl Size 0 braided sutures (CSI approved suture material via validations).
2. Place suture in the jaw depression opening nearest the hinge point.
3. Close the jaws around the suture by releasing the thumb ring, thus forming a needle point.
4. Pass the needle point with the suture through tissue by pushing distally on the handle while maintaining a gentle pressure on the thumb ring to keep the needle tip closed.
5. Open the jaw to drop the suture.
6. Close the jaw and remove the suture passer.
7. With the jaws closed, reinsert the suture passer in proximity to the first entry point.
8. Open jaws to retrieve suture by pulling back on thumb ring.
9. Close jaws by releasing the thumb ring to form a needle tip.
10. While removing the suture passer (suture in place) maintain gentle pressure on the thumb ring.

Jaws Open

Jaws Closed

EXPLANATION OF SYMBOLS

REF Reorder Number

LOT Batch Code

Use-by date

STERILE EO Sterilized Using Ethylene Oxide.

Consult instructions for use.

Do not re-use

Do not use if package is damaged

Caution

Do not resterilize

Rx Only **Caution:** U.S. Federal law restricts this device to sale by or on the order of a physician.

CE 2797 **Product conforms to the Medical Device Directive 93/42/EEC**

EC REP Authorized Representative in the European Community.

Manufacturer

Not made with natural rubber latex.

This product is protected under one or more of the following U.S. Patents: 5,899,911 • 6,183,485 and Others Pending.

Carter-Thomason CloseSure System® is a registered trademark of CooperSurgical, Inc.

CooperSurgical® is registered trademark of CooperSurgical, Inc.

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© 2021 CooperSurgical, Inc.
Carter-Thomason CloseSure System® è un marchio registrato di CooperSurgical, Inc.
CooperSurgical® è un marchio depositato di CooperSurgical, Inc.