
Hoyte Sacro Tips

EN

LH-SCOLPO-SML, LH-SCOLPO-LRG, LH-SCERV-SML, LH-SCERV-LRG

Instructions for Use (English)

CE 2797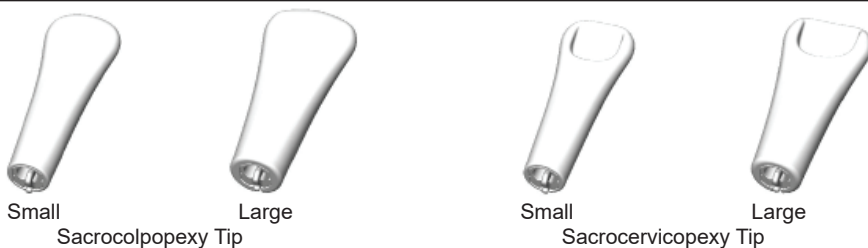

DEVICE DESCRIPTION

The Hoyte Sacrocolpopexy Tips (Hoyte Sacro Tips) are used for laparoscopic vaginal prolapse repair procedures where the cervix has been removed. The Hoyte Sacrocolpopexy Tip is made out of a hard polycarbonate plastic offered in a small and large tip size.

The Hoyte Sacrocervicopexy Tips (Hoyte Sacro Tips) are used for laparoscopic vaginal prolapse repair procedures where the cervix is intact. The Hoyte Sacrocervicopexy Tip is made out of a hard polycarbonate plastic offered in a small and large tip size.

WARNINGS

Contents supplied sterile. Do not use if sterile barrier is damaged.

For single use only. Do not reuse, reprocess or resterilize. Reuse, reprocessing or resterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or resterilization may also create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Dispose of in accordance with all applicable Federal, State, and local Medical/Hazardous waste practices.

CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

INDICATIONS FOR USE

The Hoyte Sacro Tips are intended to be connected to the Advincula Arch™ handle (UMH750) and utilized for any laparoscopic or open procedure where elevating the vaginal tissue into the operative field would be beneficial.

The single use Hoyte Sacro Tips are intended to provide a solid base, or “thimble-like” surface to suture against.

CONTRAINDICATIONS

The Hoyte Sacro Tips should not be used when a patient has a non-patent vagina or vaginal septum.

INSTRUCTIONS FOR USE

Connecting the Hoyte Sacro Tip to the Advincula Arch™ handle (UMH750).

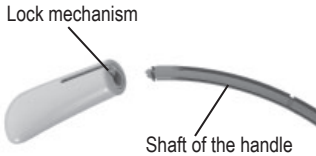
Fig. 1

Proper position of the lock mechanism once locked.

Fig. 2

1. Grasp the Hoyte Sacro Tip with the lock-mechanism facing toward the bottom of the shaft of the handle. (Fig.1)
2. Slide the Hoyte Sacro Tip over the shaft approximately 2 inches (5.08 cm) until it lock into place. (Fig. 2)
3. Attempt to pull back on the Hoyte Sacro Tip to confirm it is locked.

EXPLANATION OF SYMBOLS

Reorder number

Batch code

Use-by date

Consult instructions for use

Do not re-use

Do not re-sterilize

Do not use if package is damaged

Caution

Not made with natural rubber latex

Sterilized using ethylene oxide

Authorized Representative in the European Community

Caution: U.S. Federal Law restricts this device to sale by or on the order of a physician

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CooperSurgical, CooperSurgical, Inc. şirketinin tescilli ticari markasıdır.
Advincula Arch™, CooperSurgical, Inc. şirketinin tescilli ticari markasıdır.