
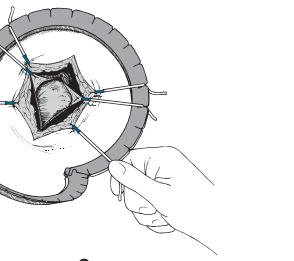
Lone Star® Retractor System

INSTRUCTIONS FOR USE

Retractors and Elastic Stays

CE 2797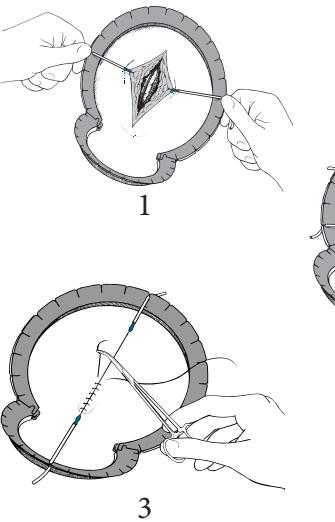

Rx Only

ENGLISH

Lone Star® Retractor System - Instructions for Use
Retractors and Elastic Stays**DEVICE DESCRIPTION/INTENDED USE**

The Lone Star Retractor System is comprised of a variety of adjustable, self-retaining Retractor Rings and Elastic Stay hooks that offer access and visualization across a wide range of surgical applications. The Retractor Rings are single-use, disposable devices that are available in a range of flexible, articulating shapes appropriate to the surgical site. The Elastic Stays are single-use, disposable devices that are used to adapt the Retractor Ring to the site's anatomy.

INDICATIONS FOR USE

The Lone Star Retractor System is indicated for use to provide incision retraction and to aid in visualization during surgical procedures.

CONTRAINDICATIONS

There are no known contraindications associated with the use of this device.

WARNINGS

- Do not use if sterile barrier is damaged.
- Do not use if the device has been opened or damaged.
- For single use only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or re-sterilization may create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.
- Dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

DIRECTIONS FOR USE

1. Place Retractor Ring around surgery site. Connect Elastic Stays in directly opposing quadrants with equal tension to hold Retractor Ring in place.
2. As dissection progresses, advance position of Elastic Stays into deeper tissue. Add or reposition Elastic Stays to desired angle for balanced retraction.
3. Maintain constant, opposing tension of Elastic Stays while suturing.

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ADVARSLER

- Må ikke bruges, hvis den sterile barriere er beskadiget.
- Må ikke bruges, hvis den sterile barriere er beskadiget.
- For enkelte brug. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or re-sterilization may create a risk of contamination of the device and/or cause patient infection or cross-infection, including, but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.
- Dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

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