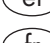
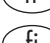
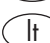
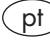
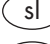
Cytobrush® Plus GT

R_x Only  STERILE|EO  2797

-  cs Návod na použití
-  da Brugsanvisning
-  de Gebrauchsanweisung
-  el Οδηγίες χρήσης
-  en Instructions for use
-  es Instrucciones de uso
-  et Kasutusjuhend
-  fr Notice d'utilisation
-  fi Käyttöohjeet
-  hu Használati utasítás
-  it Istruzioni per l'uso
-  lt Vartojimo instrukcija
-  lv Lietošanas pamācība
-  nl Gebruiksaanwijzing
-  pl Instrukcja użytkowania
-  pt Instruções de uso
-  sk Návod na použitie
-  sl Navodilo za uporabo
-  sv Bruksanvisning

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Cytobrush® Plus GT

Sterile

Cytobrush® Plus GT sterile is a single use device.
Discard after use.

CAUTION

For use by trained medical staff only.

INDICATIONS FOR USE

Cytobrush® Plus GT sterile is used to collect cell samples from the endocervix for

1. Pap smears (the ectocervical sample is usually collected with a spatula).
2. Diagnosis of infections, e.g. Chlamydia.

CONTRAINDICATIONS

Do not use Cytobrush® Plus GT after the first ten (10) weeks of gestation in pregnant patients.
Do not use Cytobrush® Plus GT for endometrial sampling.

WARNINGS

Never allow Cytobrush® Plus GT to enter too far into the endocervix. A few bristles of the brush should at all times be seen in the lower part of the cervix.

CAUTION: The product is for single use and not to be reused due to risk of contamination.

ADVERSE REACTIONS

Slight bleeding may occur during sampling and is acceptable. Check that bleeding has ceased before end of gynaecological examination.

STERILITY

Observe the expiry date!
Do not use if package is opened or damaged.
Do not resterilize.

SAMPLING INSTRUCTIONS

1. Gently insert Cytobrush® Plus GT into the endocervical canal. A few bristles of the brush should be seen outside the cervix.
2. Rotate the brush slowly one (1) turn.
3. Withdraw Cytobrush® Plus GT

PREPARATION OF SAMPLE FOR CYTOLOGY (Pap smear)

- Roll Cytobrush® Plus GT over designated slide area.
- Fix immediately with cytology spray fixative or 95% ethanol.

PREPARATION OF SAMPLE FOR MICROBIOLOGY (e.g. Chlamydia)

Consult your local microbiology laboratory for detailed instructions regarding specific tests.

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© 2021 CooperSurgical, Inc.
CooperSurgical, Inc.