# Cytobrush® Plus

R<sub>x</sub> Only (2) **STERILE** EO **CE**2797

- (cs) Návod na použití
- (da) Brugsanvisning
- (de) Gebrauchsanweisung
- (el) Οδηγίες χρήσης
- (en) Instructions for use
- (es) Instrucciones de uso
- (et) Kasutusjuhend
- (fr) Notice d'utilisation
- (fi) Käyttöohjeet
- (hu) Használati utasítás
- (it) Istruzioni per l'uso
- (lt) Vartojimo instrukcija
- (lv) Lietošanas pamācība
- (nl) Gebruiksaanwijzing
- (pl) Instrukcja użytkowania
- (pt) Instruções de uso
- (sk) Návod na použitie
- (sl) Navodilo za uporabo
- (sv) Bruksanvisning

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# Cytobrush® Plus

## Sterile

Cytobrush® Plus sterile is a single use device.  
Discard after use.

### CAUTION

For use by trained medical staff only.

### INDICATIONS FOR USE

Cytobrush® Plus sterile is used to collect cell samples from the endocervix for

1. Pap smears (the ectocervical sample is usually collected with a spatula).
2. Diagnosis of infections, e.g. Chlamydia.

### CONTRAINDICATIONS

Do not use Cytobrush® Plus after the first ten (10) weeks of gestation in pregnant patients.

Do not use Cytobrush® Plus for endometrial sampling.

### WARNINGS

Never allow Cytobrush® Plus to enter too far into the endocervix. A few bristles of the brush should at all times be seen in the lower part of the cervix.

**CAUTION:** The product is for single use and not to be reused due to risk of contamination.

### ADVERSE REACTIONS

Slight bleeding may occur during sampling and is acceptable. Check that bleeding has ceased before end of gynaecological examination.

### STERILITY

Observe the expiry date!

Do not use if package is opened or damaged.

Do not resterilize.

### SAMPLING INSTRUCTIONS

1. Gently insert Cytobrush® Plus into the endocervical canal. A few bristles of the brush should be seen outside the cervix.
2. Rotate the brush slowly one (1) turn.
3. Withdraw Cytobrush® Plus

### PREPARATION OF SAMPLE FOR CYTOLOGY

#### (Pap smear)

- Roll Cytobrush® Plus over designated slide area.
- Fix immediately with cytology spray fixative or 95% ethanol.

### PREPARATION OF SAMPLE FOR MICROBIOLOGY (e.g. Chlamydia)

Consult your local microbiology laboratory for detailed instructions regarding specific tests.

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CooperSurgical, Inc.