# 10022 • Mityvac® Reusable Obstetrical Vacuum Pump Instructions for Use (English)

## CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- Vacuum gauges can be rendered inaccurate during shipment despite care taken in packaging. Gauge calibration must be checked **before each use**.
- The Mityvac reusable obstetrical vacuum pump is manufactured and individually tested in strict adherence to specifications. Any repair or maintenance performed by anyone or entity other than the manufacturer could compromise the integrity of the pump and is done so at the risk and liability of the institution performing the work. All Mityvac reusable obstetrical vacuum delivery pumps should be thoroughly checked for proper performance and gauge calibration **prior to each use**.

## DEVICE DESCRIPTION

The Mityvac® reusable obstetrical vacuum extraction pump delivers the vacuum needed to aid in delivery of the fetus in both vaginal or Cesarean births. It is hand-held, manually operated and may be used in the proximity of oxygen. The Mityvac reusable obstetrical vacuum pump may be used either inside or outside of the sterile field.

### Tubing Remover


quickly releases tubing once delivery is complete.

### Pump Vacuum Nozzle

connects to tubing.

### Trigger vacuum release

allows the operator to reduce the vacuum gradually or release it rapidly.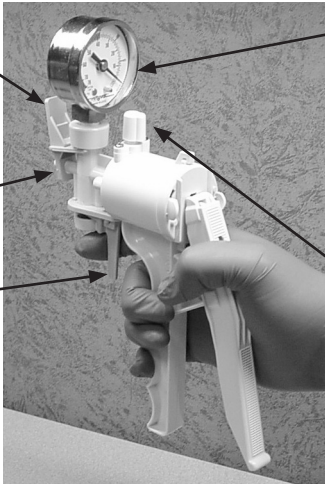

### Hand Pump Gauge

measures in centimeters and inches of mercury (Hg). It is color coded for ease of operation.

### Removable Exhaust Port Cover

allows easy cleaning.

## IMPORTANT INFORMATION ABOUT YOUR MITYVAC PUMP


We recommend that you check the equipment and the calibration of the gauge before each use.

Your pump's useful life may be extended if you follow these precautions:

1. Avoid drawing fluids into the pump during the procedure.
2. Remember that the Mityvac reusable obstetrical vacuum pump is a delicate instrument and should be handled accordingly.
3. The Mityvac reusable obstetrical vacuum pump should be sterilized using the Moist Heat Validation Sterilization Process, as indicated within the Sterilization Instructions.
4. To prevent breakage, do not force the pump handle. If operation becomes too difficult, lubricate piston shaft by adding a few drops of medical grade silicone oil (non-petroleum based) into the pump vacuum nozzle.
5. Do not twist tubing on the pump vacuum nozzle to remove it. Doing so may cause undue stress on the pump junction. Simply use the tubing remover supplied to easily remove tubing from pump.

## SETTING UP THE SYSTEM

1. Check to ensure the pump needle is in its resting position within the zero box on the gauge.
2. Connect the sterile tubing to the pump vacuum nozzle (with cup, or with cup and filter already assembled).
3. Now check for integrity of system by pumping with cup pressed to flattened palm of gloved hand. Rapidly pump until the gauge needle is in the green area between 38 and 58 cm Hg (centimeters of mercury) or between 15 and 23 in. Hg (inches of mercury). Gauge needle should move upon initiation of vacuum. Once the desired vacuum level is reached the gauge needle should remain steady.
4. If gauge needle does not remain steady during test pumping:
  - Check cup pressed to palm. The lines in your hand may be causing the vacuum leak.
  - Check the connection of tubing to pump vacuum nozzle. Be sure it is pushed on tight.
  - Check the connection of tubing to the cup handle. Be sure it is pushed on tight.
  - Check the connection of tubing to filter. Be sure it is pushed on tight.
5. To release or reduce vacuum at the pump, gently squeeze the pump trigger. Vacuum may be released slowly or rapidly. To discontinue vacuum, squeeze the vacuum release trigger until gauge needle drops into the zero box.
6. If, after following steps 1-4, the integrity of the pump is in question, **do not use it**. Contact your CooperSurgical representative.
7. A 4-foot (1.2 meter) section of tubing is provided so that the Mityvac pump may be operated by an assistant outside the sterile field.
8. When delivery has been completed, remove tubing at pump and dispose of extractor cup unit. The pump may be cleaned for reuse.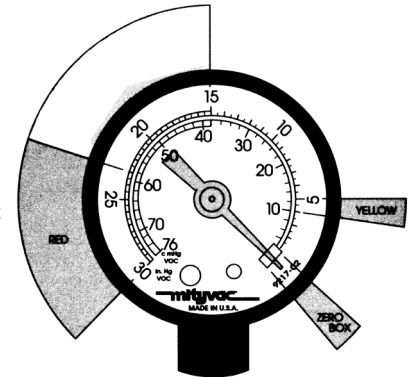

## Using the tubing remover

Snap tubing remover into place with pump vacuum nozzle in hole and tubing remover legs around junction (**Figure A**).

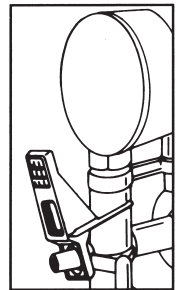
Figure A

## Removing the tubing

Hold pump in hand with fingers on top pad section of tubing remover. Press toward gauge and tubing remover will "rock" tubing off of pump vacuum nozzle (**Figure B**).

In an extremely tight tubing fit, the hand may be reversed and the thumb, or both thumbs, may be used to press the pad section.

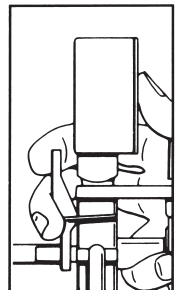
Figure B

**OPERATING INSTRUCTIONS**

The physician must assess the delivery situation to evaluate the appropriateness of using vacuum extraction to aid in delivery based on training and experience with vacuum extraction and the unique circumstances of each birth. Always use with caution and professional judgment. Willingness to abandon this method after three attempts must be practiced.

These Instructions for Use are specific and different for each configuration of Mityvac vacuum extraction cup. Follow the Instructions for Use included in each pre-packaged, sterile, disposable vacuum extraction delivery kit. Read these instructions thoroughly before operating the unit. We recommend that the pump and gauge calibration be checked prior to each use. To check calibration compare the pump gauge against a master gauge traceable to NIST Standards at three calibration points:

**Pump gauge tolerances:**

Calibration Points	0 in. – 7.5 in. Hg (0–19 cm Hg)	7.5 in. – 22.5 in. Hg (19–57 cm Hg)	22.5 in. – 30 in. Hg (57–76 cm Hg)
Tolerances	± 0.9 in. Hg (± 2.3 cm Hg)	± 0.6 in. Hg (± 1.5 cm Hg)	± 0.9 in. Hg (± 2.3 cm Hg)

VACUUM CONVERSION CHART							
Kg/cm <sup>2</sup>	kPa	mm Hg	cm Hg	In. Hg	cm H <sub>2</sub> O	lb/in. <sup>2</sup>	bar
0.13	13	100	10	3.9	134	1.9	0.13
0.27	27	200	20	7.9	268	3.9	0.26
0.41	40	300	30	11.8	402	5.8	0.39
0.47	47	356	36	14	478	6.9	0.46
0.51	51	382	38	15	512	7.4	0.50
0.54	53	400	40	15.7	538	7.7	0.53
0.61	61	459	46	18	615	8.9	0.60
0.68	67	500	50	19.7	670	9.7	0.66
0.82	80	600	60	23.6	804	11.6	0.79

**CLEANING**

1. The Reusable Silicone Vacuum Extraction Cup must be separated from the vacuum tubing and the MityVac pump. Both the cup and the pump must be cleaned separately.

2. Remove the pump gauge by turning the gauge counterclockwise while supporting the pump junction
3. Prepare a solution with protein solubilizing soap and lukewarm tap water following the detergent manufacturer’s recommendation. **Note:** Prolystica Enzymatic 2X detergent should be used at the concentration of 2 mL / 1L of warm utility (tap) water at 25-30 °C per manufacturer’s recommendation.
4. Submerge and soak the MityVac Pump.
5. Ensure the device remains completely submerged in the detergent solution. \*  
*\*Do not submerge the gauge*
6. Actuate pump by squeezing handle at least five (5) times to draw detergent solution through pump.
7. Soak for a minimum of 15 minutes.
8. Using a soft brush or sponge, help loosen and remove debris.
9. Use a channel brush to scrub the lumens of the MityVac pump (pump vacuum nozzle and exhaust port).
10. After soaking and cleaning, rinse the device with cool utility (tap) water at 18-22 °C for a minimum of 30 seconds.
11. Actuate pump by squeezing handle at least five (5) times to draw rinse water solution through the vacuum nozzle into the pump.

**DRYING**

12. Towel dry or blow dry with air jet.
13. If water remains on the device, wipe with 70% isopropyl alcohol.
14. Lubricate pump body with medical grade silicone oil.
15. Assemble the pump gauge by turning the gauge clockwise while supporting the pump junction.
16. Once the device is clean and dry it should be stored in a clean, dry and free from particulate environment.

**STERILIZATION INSTRUCTIONS**

**The Mityvac Reusable Obstetrical Vacuum Pump is supplied Non-Sterile and may be sterilized before use within the sterile field.**

1. Remove the pump handle by using a screw driver while supporting the pump junction

2. **MOIST HEAT STERILIZATION:**  
It can be sterilized using one of the two Moist Heat Sterilization cycles listed below.

STERILIZATION PROCESS	EXPOSURE TEMPERATURE	EXPOSURE TIME	DRY TIME
Pre-Vacuum	275 °F/135 °C	3 minutes	30 minutes
Gravity	250 °F/121 °C	30 minutes	30 minutes

**SUGGESTED PUMP TROUBLESHOOTING GUIDELINES**

The following troubleshooting guidelines include possible diagnosis. If after attempting to repair the pump it still is not fully functional, call your CooperSurgical representative.

**NOTE:** Refer to the exploded view of the pump as needed. Remove all accessories that may be attached to the Mityvac pump.

1. **Cannot pull enough vacuum.**

**Diagnosis:** Exhaust valve is not functioning properly.  
**Action:** Clean valve seat and replace valve.

**Diagnosis:** Contamination.  
**Action:** Inspect pump and components for particulates or foreign matter. Clean as directed in these instructions. Replace pump or parts as necessary.

**Diagnosis:** Defective gauge.  
**Action:** Remove and check the gauge; pump could be functioning, but the gauge is not registering properly. Replace if defective; see gauge replacement guidelines on following page.

**Diagnosis:** Not enough lubrication.  
**Action:** Lubricate by applying a thin coating of medical grade silicone inside of pump body.

**Diagnosis:** Bent/fatigued body spring.  
**Action:** Replace spring.

**SUGGESTED PUMP TROUBLESHOOTING GUIDELINES (continued)**

**Diagnosis:** Loose screws on exhaust fitting.  
**Action:** Carefully tighten screws. **DO NOT OVER TIGHTEN.** Over tightening may result in stripping of the plastic threads.  
**Diagnosis:** Distorted piston.  
**Action:** Replace piston rod assembly. We recommend that the pump and gauge calibration be checked before each use.

**2. Leaking after vacuum pulled.**

**Diagnosis:** Umbrella valve contaminated.  
**Action:** Clean pump and replace valve. Properly align and place umbrella valve stem into center hole of pump body. Pull stem through hole until knob of valve is completely locked in place.  
**Diagnosis:** Fatigued components in vacuum release assembly.  
**Action:** Replace parts as necessary.

**Diagnosis:** Contaminated or defective junction O-ring.  
**Action:** Clean pump; replace O-ring.

**Diagnosis:** Loose screws on junction and/or trigger assembly.  
**Action:** Carefully tighten screws. **DO NOT OVER TIGHTEN.** Over tightening may result in stripping of the plastic threads. We recommend that the pump and gauge calibration be checked before each use.

**3. Damaged/Defective.**

**Action:** Contact a CooperSurgical representative.

**GAUGE REPLACEMENT GUIDELINES**

**NOTE:** A 9/16 in. open-end wrench may be used (at the square base of the gauge) when replacing gauge. Refer to the exploded view of the pump.


**To remove gauge:**

1. Support the pump junction.
2. Turn gauge counterclockwise.
3. Clean junction as needed.

**To install new gauge:**

1. Support the pump junction.
2. Wrap Teflon tape tightly two times around the gauge threads or apply a thin coating of silicone oil onto gauge threads. **DO NOT APPLY CAULKING.**
3. Align and place gauge into pump junction. Turn gauge clockwise, being careful not to strip junction threads.
4. The gauge should be firmly in place and seated approximately 1/16 in. – 1/8 in. above the pump junction. **DO NOT OVER TIGHTEN.** Over tightening may result in stripping of the plastic threads.
5. Test pump performance as described in these directions. Test gauge for accuracy against an accurately calibrated gauge (see Vacuum Conversion Chart for equivalents) per ASME B40.100 standard.

Also see OPERATING INSTRUCTIONS section of this guideline.

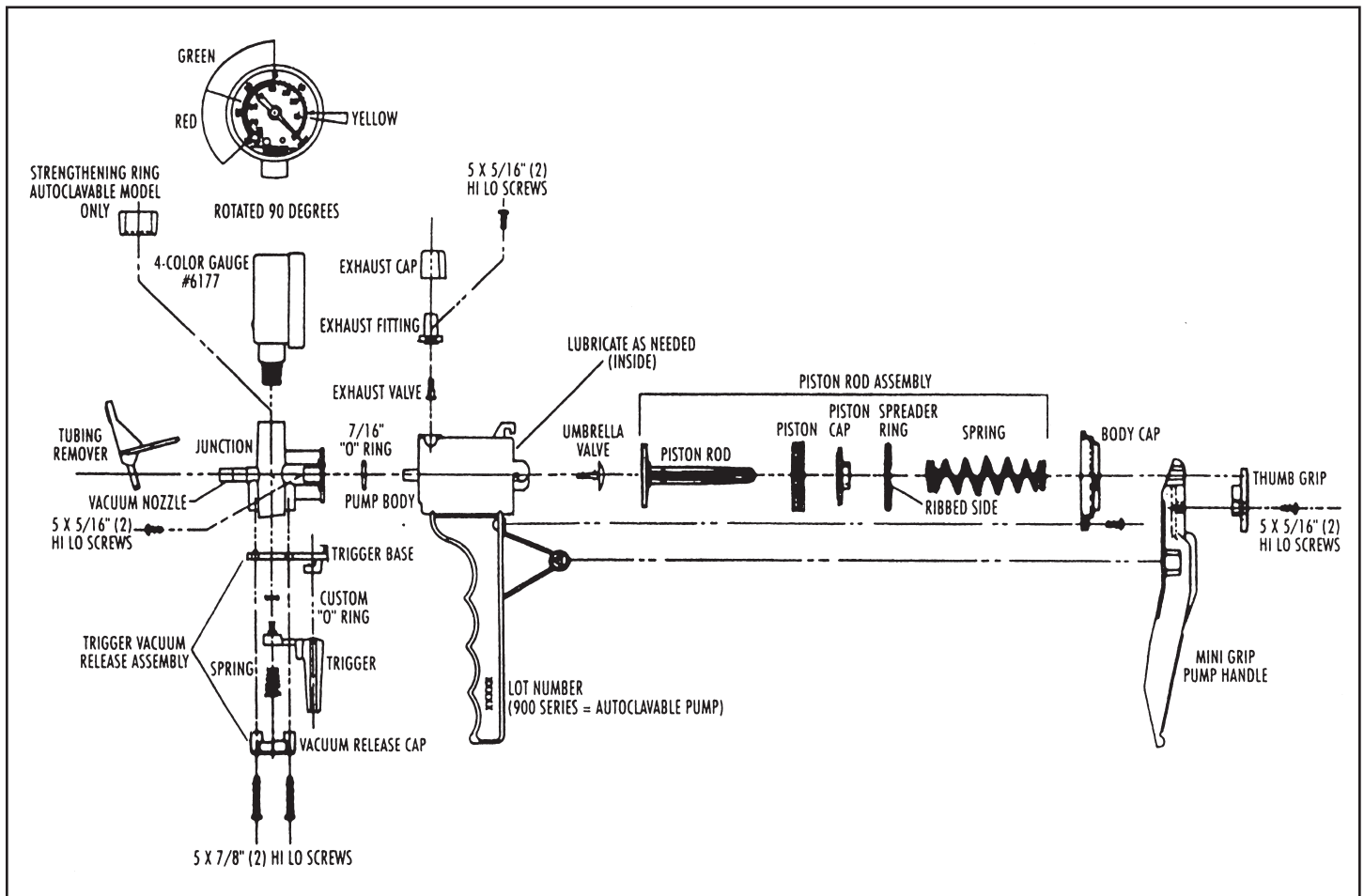**MAINTENANCE KITS AVAILABLE**

**P/N 16710** Maintenance kit for Mityvac Reusable Obstetrical Vacuum Pump.

Maintenance kit includes the internal valves, O-rings, screws, spring, and vacuum release replacement parts necessary to perform routine maintenance on your pump.

**P/N 16177** Replacement Gauge, color coded, calibrated in inches of Hg and centimeters of Hg.

**Questions:** Contact your local Mityvac obstetrical products distributor or call CooperSurgical at 800-243-2974.

**ASSOCIATED PRODUCTS**

Product Number	Cups (Description)	Diameter
10004	Pearl Edge Bell Cup with 4-foot (1.2 meter) tubing & filter	60 mm
10007LP	M-Style Mushroom Cup with 4-foot (1.2 meter) tubing & filter	50 mm
10008	Super M-Style Mushroom Cup with 4-foot (1.2 meter) tubing & filter	56 mm
10020	MitySoft Bell Cup with 4-foot (1.2 meter) tubing and filter	64 mm
10137	M-Select Mushroom Cup with 4-foot (1.2 meter) tubing and filter	50 mm
10500	Reusable Vacuum Delivery Cup	70 mm

**LIMITED WARRANTY**

This Mityvac Pump is warranted to be free of all defects in material and workmanship for one (1) year from the date of original purchase. This warranty does not apply to any defects caused by negligence of the user, nor to the use of the product in any manner other than in accordance with the instructions provided with the product. The warranty also does not apply to damage caused by accidents, nor does it cover any pump damaged by intake of liquids, foreign particles or other contamination.

Within the period of this warranty, we (CooperSurgical) will repair or replace any part proved to be defective in material or workmanship. No one other than CooperSurgical can change this warranty in any way. We reserve the right to make changes or improvements in our products without having to similarly alter products previously sold. Any warranty implied by law, including any warranty of merchantability or fitness, is expressly limited to the terms set above.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER REMEDIES. WE SHALL NOT BE LIABLE FOR ANY OTHER EXPENSE, LOSS OR DAMAGE, OF ANY KIND, ARISING IN CONNECTION WITH THE SALE OR USE OF, OR INABILITY TO USE, THE MITYVAC REUSABLE OBSTETRICAL VACUUM PUMP FOR ANY PURPOSE.**

Some states do not allow limitations on how long any implied warranty lasts or the exclusion or limitation of incidental or consequential damages so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights which may vary from state to state.

**EXPLANATION OF SYMBOLS**

Reorder Number

Non-sterile

Do not use if package is damaged

Batch Code

Caution

Keep Dry

Consult instructions for use

Authorized Representative in the European Community.

Fragile, handle with care

Not made with natural rubber latex

Manufacturer

U.S. Federal law restricts this device to sale by or on the order of a physician.

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© 2021 CooperSurgical, Inc.