
10500 • Mityvac® Reusable Silicone Vacuum Extraction Cup

Instructions for Use (English)

CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

DEVICE DESCRIPTION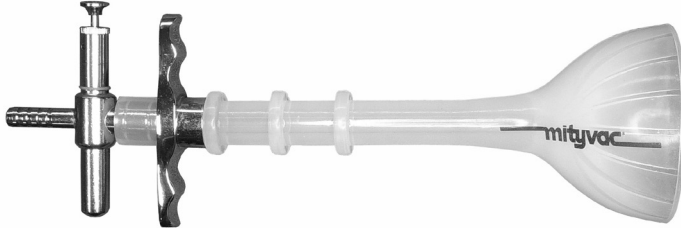

The Mityvac® obstetrical vacuum delivery system is utilized to assist a clinician in the delivery of an infant during childbirth. The Mityvac Reusable Vacuum Extraction Cup is constructed of silicone and features a Mityvac logo for positioning reference. The shaft has three molded ridges to help facilitate a firm grip. The handle and valve assemblies are manufactured of high-quality chrome-plated brass. The reusable cup will withstand repeated uses and sterilization. Product life can vary greatly depending on the care extended during reprocessing and use.

INDICATIONS FOR VACUUM-ASSISTED VAGINAL DELIVERY

No indication is absolute.

Fetal:

- Non-reassuring fetal status
- Failure to deliver spontaneously following an appropriately managed second stage of labor

Maternal:

- Need to avoid voluntary expulsive efforts
- Inadequate expulsive efforts

CONTRAINDICATIONS

Do not initiate vacuum if any of the following conditions exist:

1. Non-vertex positions (breech or transverse lie/position) or face or brow presentation
2. Suspected cephalopelvic disproportion
3. Previous scalp sampling
4. Suspected macrosomia, or risk of shoulder dystocia
5. Failed vacuum or forceps attempt
6. Less than 34 weeks gestation
7. Unengaged vertex
8. Incompletely dilated cervix
9. Need for active device rotation
10. Suspected fetal bleeding abnormalities

ADVERSE EVENTS

Fetal Injuries:

Head trauma, bruises, contusions, lacerations, scalp edema, skull fracture, cephalhematoma, subgaleal hematoma, subdural hemorrhage, parenchymal hemorrhage, intracranial hemorrhage and retinal hemorrhage.

Maternal Injuries:

Vaginal, cervical, uterine, bladder, rectal tissue tears

WARNINGS

1. DO NOT exceed recommended vacuum levels.

2. Never apply the vacuum cup to any portion of the infant's face.
3. Vacuum-assisted delivery should only be performed or supervised by a trained and experienced operator.
4. "There should be a willingness to abandon attempts at vacuum extraction if satisfactory progress is not made." (ACOG Technical Bulletin #196, August 1994)
5. Abandon vacuum-assisted delivery if:
 - a. Vacuum cup becomes disengaged (pops off) three times.
 - b. Vertex has not advanced substantially with each traction attempt.
 - c. There is evidence of fetal scalp trauma.
 - d. Cumulative traction time exceeds 10 minutes or total vacuum procedure time exceeds 15 to 30 minutes.
6. If the extractor cup becomes disengaged, check for trauma to the fetal scalp before reapplying.
7. This device is Non-Sterile and should be sterilized according to the CLEANING AND STERILIZATION INSTRUCTIONS in this document prior to each use. Any deviations from these directions should be validated by the user.

PREREQUISITES FOR VACUUM-ASSISTED VAGINAL DELIVERY

- Term infant
- Ruptured membranes
- Empty bladder and rectum
- Engaged vertex position
- Complete cervical dilation and effacement
- Willingness to abandon procedure and plan for alternate delivery route

VACUUM-ASSISTED VAGINAL DELIVERY GUIDELINES

These directions are intended as general guidelines. Practitioners should refer to current institutional and recognized guidelines that address vacuum-assisted delivery.


1. Sterilize cup per the cleaning and sterilization instructions.
2. Connect sterile cup to pump using sterile suction tubing.
3. **For Mityvac 10022 Hand-Held Vacuum Pump:** Check the integrity of the system by pressing the cup to the palm of your gloved hand and applying vacuum. The gauge needle should remain steady. (NOTE: the gauge needle should rest in the "ZERO" box when vacuum is not applied.)
4. **For Electric Vacuum Source:** Connect sterile cup, tubing and connector to the electric vacuum pump by placing the tapered end of the connector securely into the hole of the collection jar lid. (Tubing and connector may be purchased separately.) Preset the desired vacuum levels at the vacuum source. Refer to the Vacuum Conversion Chart as necessary.

CAUTION: DO NOT USE DEVICE IF GAUGE NEEDLE DOES NOT INDICATE ZERO IN THE ABSENCE OF VACUUM.
5. Carefully examine fetal presentation and position prior to positioning the cup over or near the flexion point.

CAUTION: DO NOT USE PRODUCT IF FLEXION POINT IS NOT ACCESSIBLE.
6. Wipe the scalp as clean as possible.
7. A lubricant may be applied to cup as desired.
8. To insert, grasp the cup portion of the extractor with fingers. The sides of the cup may be folded inward to facilitate insertion. Separate the labia with the other hand. Press the cup portion downward and inward over the fourchette to make contact with the fetal scalp.

(Continued)

Apply the center of the cup over the flexion point (see Figure). The flexion point can be located by identifying the posterior fontanelle and then moving the finger anteriorly along the sagittal suture approximately 1.2 in. (3 cm). The sagittal suture should pass under the middle of the cup.

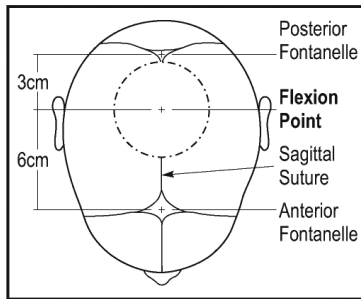CAUTION: NEVER APPLY THE CUP TO ANY PORTION OF THE INFANT'S FACE.

9. Gently move a finger around the rim of the cup to limit intrusion of extraneous tissues and to ensure proper cup placement.
10. With the cup placed over the flexion point, raise the vacuum level to approximately 3.9 in. Hg (10 cm Hg) to initiate vacuum. Recheck the rim of the cup for interposed extraneous tissue. **For an electric vacuum source refer to the Vacuum Conversion Chart.**

CAUTION: BEFORE RAISING VACUUM TO TRACTIVE LEVELS, MAKE CERTAIN THAT NO EXTRANEOUS TISSUE IS DRAWN INTO THE VACUUM CUP.

11. With the onset of contraction, rapidly raise the vacuum to 15-22.8 in. Hg (38-58 cm Hg) and begin applying traction. Always apply traction in harmony with contractions and along the pelvic axis. **For an electric vacuum source, refer to the Vacuum Conversion Chart.**

CAUTION: DO NOT EXCEED RECOMMENDED VACUUM LEVELS.

12. When the contraction is no longer effective, discontinue traction. You may reduce the vacuum to approximately 3.9 in. Hg (10 cm Hg) while awaiting the next contraction or maintain vacuum at current level.
13. Recheck for interposed extraneous tissue prior to each tractive effort.
14. If traction is misaligned or too forceful, the vacuum cup may disengage (pop off). In case of pop off, check fetal scalp for trauma before reapplying vacuum cup.

CAUTION: ABANDON VACUUM-ASSISTED DELIVERY IF THE VACUUM CUP DISENGAGES (POPS OFF) THREE TIMES.

15. With each successive contraction, draw the head gently over the perineum. Once the head is delivered, release vacuum and remove the cup. Continue delivery in the usual manner.

CAUTION: DO NOT OPERATE THE VACUUM PUMP AT TRACTION LEVELS FOR MORE THAN TEN CUMULATIVE MINUTES OF TRACTION OR TOTAL PROCEDURE TIME OF 15 TO 30 MINUTES.

16. Rinse vacuum cup immediately after each use, ensuring cup stem is clear of any debris. Do not use sharp instruments to clean channels. SEE CLEANING AND STERILIZATION INSTRUCTIONS.
17. Document the use of the vacuum extraction cup and notify nursery staff per hospital protocol.

VACUUM CONVERSION CHART

Kg/cm ²	kPa	mm Hg	cm Hg	In. Hg	cm H ₂ O	lb/in. ²	bar
0.13	13	100	10	3.9	134	1.9	0.13
0.27	27	200	20	7.9	268	3.9	0.26
0.41	40	300	30	11.8	402	5.8	0.39
0.47	47	356	36	14	478	6.9	0.46
0.51	51	382	38	15	512	7.4	0.50
0.54	53	400	40	15.7	538	7.7	0.53
0.61	61	459	46	18	615	8.9	0.60
0.68	67	500	50	19.7	670	9.7	0.66
0.82	80	600	60	23.6	804	11.6	0.79

CLEANING AND STERILIZATION INSTRUCTIONS

The Mityvac Reusable Silicone Vacuum Extraction Cup can be cleaned, sterilized and reused many times without product degradation, but the user should examine the cup prior to each use for the following defects:

- Cracks or holes in the stem of the cup
- Cracks or tears on the soft lip of the cup
- Any embrittlement of the soft silicone material

If any of the foregoing is detected, the cup should be immediately discarded.

1. After each use, clean the cup in the following manner:
 - a. The Reusable Silicone Vacuum Extraction Cup must be separated from the vacuum tubing and the MityVac pump. Both the cup and the pump must be cleaned separately.
 - b. Prepare a solution with protein solubilizing soap and lukewarm tap water following the detergent manufacturer's recommendation. **Note:** Polystica Enzymatic 2X detergent or equivalent should be used at the concentration of 2 mL / 1L of warm utility (tap) water at 25-30 °C.
 - c. Submerge and soak the delivery cup.
 - d. Ensure the device remains completely submerged in the detergent solution.
 - e. Soak for a minimum of 15 minutes.
 - f. Using a soft brush or sponge, help loosen and remove debris.
 - g. Use a channel brush to scrub the lumens of the delivery cup.
 - h. After soaking and cleaning, rinse the device with cool utility (tap) water at 18-22 °C for a minimum of 30 seconds.

DRYING

- i. Towel dry or blow dry with air jet.
 - j. If water remains on the device, wipe with 70% isopropyl alcohol.
 - k. Once the device is clean and dry it should be stored in a clean, dry and free from particulate environment.
2. Sterilize the cup using one of the two Moist Heat Sterilization validation processes with the cycle parameters listed below.

STERILIZATION PROCESS	EXPOSURE TEMPERATURE	EXPOSURE TIME	DRY TIME
Pre-Vacuum	275 °F/135 °C	3 minutes	30 minutes
Gravity	250 °F/121 °C	30 minutes	30 minutes

PRODUCT WARRANTY

The Mityvac Reusable Silicone Vacuum Extraction Cup is warranted to be free of all defects in material and workmanship for one year from date of original purchase. This warranty does not apply to any defects caused by negligence of the user, nor to the use of the product in any manner other than in accordance with the instructions provided with the product. The warranty also does not apply to damage caused by accidents.

Within the period of this warranty, CooperSurgical, Inc., will repair or replace any part proven to be defective in material or workmanship.

Some states do not allow limitations on how long any implied warranty lasts or the exclusion or limitations of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty provides you with specific legal rights, which may vary from state to state.

EXPLANATION OF SYMBOLS

Reorder Number

Manufacturer

Batch Code

Authorized Representative
in the European Community.

Consult instructions
for use

Do not use if package is damaged

Caution

Keep Dry

Not made with
natural rubber latex

R_x Only

U.S. Federal law restricts this device to sale
by or on the order of a physician.

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