# Os Finder™ Cervical Dilator

REF 1176

CE 2797

## Instructions for Use (English)

CooperSurgical

### DEVICE DESCRIPTION

The sterile, single use Os Finder™ naturally seeks and gently opens the cervical os. It is made of extremely smooth and malleable teflon-like material. The dilator is 21.5 cm long with a taper from 2 to 4 mm.

### CAUTION

U.S. Federal law restricts this device to sale by or on the order of a physician.

### CONTRAINDICATIONS

This instrument should not be used under any of the following conditions:

- Pregnancy or suspicion of pregnancy
- Acute or active inflammation of the cervix, endometrium, fallopian tube, ovary or peritoneum
- In the case of cervicitis, endometritis, tubo-ovarian inflammatory disease or pelvic inflammatory disease
- Invasive cancer that is visible on examination

### PRECAUTIONS

Do not use the Os Finder if the package or instrument are damaged.

### INDICATIONS FOR USE

This instrument is ideal for use in intracervical dilation, uterine sounding, or for intrauterine procedures.

### INSTRUCTIONS FOR USE

The Os Finder is used to find the external os by simply pushing the narrow end gently against the cervix until it finds its way. Further insertion of the flexible tip will follow the natural path of the canal until it reaches and goes through the internal os.

Dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

### EXPLANATION OF SYMBOLS

 Reorder number

 Batch code

 Use-by

 Consult instructions for use

 Do not re-use

 Caution

 Not made with natural rubber latex

Do not use if package is damaged

**STERILE EO**

Sterilized using Ethylene Oxide

**R<sub>x</sub> Only**

U.S. Federal law restricts this device to sale by or on the order of a physician

**EC REP**

Authorized Representative in the European Community

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© 2021 CooperSurgical, Inc. Все права защищены.