INTENDED USE: Provides warmth during transport of infant within the hospital or between hospitals.

Rx Only CAUTION: U.S. Federal law restricts this device to sale by or on the order of a physician.

WARNINGS: Failure to follow these warnings can lead to severe burns

- Routinely and regularly check the infant's skin for redness and the infant's core temperature
- Do not leave on skin longer than 10 minutes
- Do not apply an additional heat source to the infant when using this product
- Consult a physician when using this product on premature newborns or infants
- Consult a physician when using this product in a non-transport environment
- Single use only. Do not reuse or reheat. Dispose of after use.
- Store at room temperature

INSTRUCTIONS FOR USE

Do Not Reuse • Disposable • Store at room temperature

Temperature when activated: 40 °C (from 24 °C start), 104 °F (from 75 °F start)

To Activate:

1. Locate metal disc and grasp with thumb and forefingers of both hands.
2. Flex (bend) disc rapidly until crystals begin to form.
3. Massage pack to soften and to increase the activation rate.
4. After pack is fully activated, indent center with palm to form a small nest.
5. Lay infant in nest on the non-woven fabric surface.
6. Do not leave on skin longer than 10 minutes.
7. Check infant skin regularly for redness and monitor infant core temperature.

CONTENTS

- Sodium acetate (food grade), water and thickeners.
- Do not swallow.
- All contents are non-toxic; however, if contents come in contact with skin or eyes, flush with water.

20421 • TransWarmer® Infant Transport Mattress with WarmGel® Instructions for Use • English (Continued)

EXPLANATION OF SYMBOLS

REF	Reorder number		Do not re-use
LOT	Batch code		Caution
	Not made with natural rubber latex	EC REP	Authorized Representative in the European Community.
	Use-by date	Rx Only	Caution: U.S. Federal law restricts this device to sale by or on the order of a physician
	Consult instructions for use		Manufacturer

MR Conditional

The **TransWarmer® Infant Transport Mattress** was determined to be MR-conditional.

Non-clinical testing demonstrated that the TransWarmer Infant Transport Mattress is MR Conditional. A patient with this device can be scanned safely immediately after placement under the following conditions:

Static Magnetic Field

- Static magnetic field of 3-Tesla or less
- Maximum spatial gradient magnetic field of 720-Gauss/cm or less

MRI-Related Heating

In non-clinical testing, the TransWarmer Infant Transport Mattress produced the following temperature rise during MRI performed for 15-min of scanning (i.e., per pulse sequence) in the 3-Tesla (3-Tesla/128-MHz, Excite, HDx, Software 14X.M5, General Electric Healthcare, Milwaukee, WI) MR system:

Highest temperature change +1.6 °C (2.88 °F)

Therefore, the MRI-related heating experiments for the TransWarmer Infant Transport Mattress at 3-Tesla using a transmit/receive RF body coil at an MR system reported whole body averaged SAR of 2.9 -W/kg (i.e., associated with a calorimetry measured whole body averaged value of 2.7-W/kg) indicated that the greatest amount of heating that occurred in association with these specific conditions was equal to or less than +1.6 °C (2.88 °F).

Artifact Information

MR image quality may be compromised if the area of interest is in the exact same area or relatively close to the position of the TransWarmer Infant Transport Mattress. Therefore, optimization of MR imaging parameters to compensate for the presence of this device may be necessary. The maximum artifact size (i.e., as seen on the gradient echo pulse sequence) extends approximately 35-mm relative to the size and shape of the TransWarmer Infant Transport Mattress.

Pulse Sequence	T1-SE	T1-SE	GRE	GRE
Signal Void Size	4,049-mm ²	3,600-mm ²	6,031-mm ²	4,616-mm ²
Plane Orientation	Parallel	Perpendicular	Parallel	Perpendicular

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