REUSABLE STERILIZABLE CRYOTIPS AND DISPOSABLE PLASTIC SHIELDS

Instructions for Use (English)

Product Number (REF)	Description	Product Number (REF)	Description
900200AA	Microderm Tip (T0219)	900212AA	Long Exo-Endocervical Tip (T-1920)
900201AA	Cone Tip (T-0219)	900213AA	Flat Tip (T-2500)
900202AA	Bevel Tip (T-0524)	900214AA	Exocervical Tip (T-2507)
900203AA	Condyloma Tip (T-0519)	900215AA	Exo-Endocervical Tip (T-2514)
900204AA	Procto Tip (T-0832)	900217AA	Procto Curved Tip (T-0832)
900205AA	General Purpose Tip (T-0819)	900300AA	2.5mm HPV Tip (T-0200)
900206AA	Cone Tip (T-0826)	900301AA	5.0mm HPV Tip (T-0500)
900207AA	Bevel Tip (T-0823)	900302AA	7.0mm HPV Tip (T-0700)
900208AA	Flat Tip (T-1900)	900303AA	9.0mm HPV Tip (T-0900)
900209AA	Exocervical Tip (T-1905)	900305AA	13.5mm HPV Tip (T-1300)
900210AA	Exo-Endocervical Tip (T-1920)	900306AA	15.0mm HPV Tip (T-1500)
900211AA	Exo-Endocervical Tip (T-1910)	900221AA	Disposable Plastic Shields (Bag of 100)

READ ALL SAFETY INFORMATION AND INSTRUCTIONS BEFORE USING THIS PRODUCT

WARNING: DO NOT USE ETO, E-BEAM OR GAMMA STERILIZATION.

CAUTION: U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN.
Not made with natural latex rubber.

WALLACH REUSABLE, STERILIZABLE CRYOTIPS SHOULD NOT BE USED WITHOUT DISPOSABLE CRYOSURGICAL TIP SHIELDS DUE TO THE FOLLOWING SAFETY HAZARDS!

- The shield acts as an insulator to prevent the tip from adhering to tissue in unwanted areas.
- The tip could leak or disengage if not attached properly to the insulator tube; the shield will assure proper attachment.
- The tip will not freeze properly if threaded on too deeply. The shield works as a positive stop to prevent this from happening, provided that the tabs on the shield are correctly fitted into the grooves on the metal tip.

INSTRUCTIONS FOR CLEANING STERILIZABLE CRYOTIPS

- Remove disposable shield from cryotip and discard per local Medical Hazardous Waste Regulations.
- Plug cryotip before cleaning and sterilizing. (Plug Orifice does not require sterilization).
- Scrub tip to remove blood and particles. Do not use an abrasive material to clean tips.
- Rinse with water.
- Cryotips can be steam sterilized by Gravity or Pre-vacuum using the following procedures:

	Stage / Process	Parameter Set-Points	
		Gravity	Pre-vacuum
Set-up	Cycle Profile		
Pre-Exposure (Conditioning)	Chamber Purge	1 minute	1 minute
Exposure (Steam Injection)	Sterilization Temperature	132 °C (270 °F)	132 °C (270 °F)
	Sterilization Time	15 minutes	4 minutes
Post-Exposure (Exhaust / Dry)	Dry Time	15 minutes	20 minutes

- Attach new disposable plastic shield to the cryotip, ensuring that the tabs on the shields are correctly fitted into the grooves on the metal tip.
- Cryotip is now ready to use.

EXPLANATION OF SYMBOLS

REF Reorder Number

SN Serial Number

LOT Batch Code
(900221AA only)

NON-STERILE
Sterilize before Use

Do Not Re-use
(900221AA only)

Not made with natural
latex rubber.

R_x Only

U.S. Federal law restricts this device
to sale by or on the order of a physician.

Authorized Representative in the
European Community.

Caution

Consult instructions for use

Manufacturer

© 2021 CooperSurgical, Inc.

Wallach® is a registered trademark of CooperSurgical, Inc.

Phone: (800) 243-2463 • (203) 799-2000
Fax: (203) 799-2002

IMCRY005-IFU-B • Rev. A • 01/21

CooperSurgical, Inc.
95 Corporate Drive
Trumbull, CT 06611 USA

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Wallach® Es una marca registrada de CooperSurgical, Inc.