CooperSurgical

ALLY UPS®

Cart/Kar/Chariot/Wagen/  
Carrello/카트/Carro/Araba

Instructions for Use/Gebruiksaanwijzingen  
Notice d'utilisation/Gebrauchsanweisung  
Istruzioni per l'uso/사용법 설명서  
Instrucciones de uso/Kullanım Talimatları

REF AU-CART

## Description

The ALLY UPS® Cart (AU-CART) is intended to provide a storage solution for the ALLY Uterine Positioning System® when it is not in use.

Photo 1 Front of the Cart with the ALLY UPS®

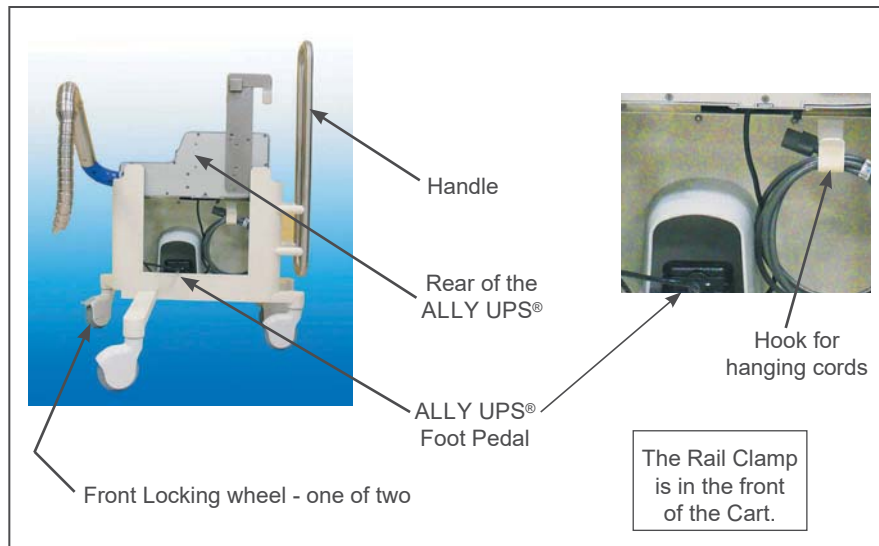
Photo 2 Rear of the Cart with the ALLY UPS®

## Intended Use

The ALLY UPS® Cart's primary function is to provide storage for the ALLY UPS®. This Cart has a compact design for easy maneuverability of the ALLY UPS®. It also allows the user to deliver the ALLY UPS® right to the edge of the Operating Room Table before lifting it onto the OR Table rail.

The ALLY UPS® Cart has a rear storage compartment that allows for storage of the Foot Pedal/ Cable/Power Cord, with enough storage space for an extension cord if it is used with ALLY UPS®. There is also a pocket to store the ALLY UPS® Rail Clamp.

## Caution

- **NEVER** use the ALLY UPS® while it resides in the Cart.
- **DO NOT** leave any cords hanging outside the storage compartment while transporting the ALLY UPS® since they may catch on other devices that pass by.

## Instructions for Use

When it is time to attach the ALLY UPS® to the Operating Room table, go through the following steps to ensure the safe movement of the ALLY UPS®. These instructions assume that the ALLY UPS® and Rail Clamp have been removed from the shipping box and placed in the ALLY UPS® Cart.

1. Wheel the Cart (and ALLY UPS®) up close (next) to the Operating Room Table (this allows the ALLY UPS® to be lifted the shortest distance possible to get it into position). Lock the two Front Wheels to prevent the Cart from rolling. See Photo 3.

**Photo 3** Moving the Cart in Position

2. Remove the Rail Clamp from the Cart and place on the Rail, sliding into position. Notice how close (the top of) the ALLY UPS® is to the OR Table. See Photo 4.

**Photo 4** Attaching the Rail Clamp to the Rail

---

## ALLY UPS® Cart (ENGLISH)

---

- Once the ALLY UPS® (and Cart) are in place at the side of the OR Table, lift the ALLY UPS® (**by the blue areas only**) out of the Cart and into position in the Rail Clamp. **NOTE:** The ALLY UPS® weighs over 20 pounds. Make sure the ALLY UPS® is hanging vertically and secured in the Rail Clamp. See Photos 5a and 5b. Refer to the ALLY UPS® Instructions for Use for positioning and mounting information.

Photo 5a Correct Hand Positions


Photo 5b Positioning the ALLY UPS®

- Unlock the Front Wheels. Move to the rear of the Cart, remove the Foot Pedal and cords and put into position at the foot of the OR Table. Now move the Cart away from the OR Table.
- When the OR procedure is completed, go through Steps 1-4 in reverse order to place the ALLY UPS®, Foot Pedal, Cords, and Rail Clamp back on the Cart.

## Specifications

**Package Weight:** 78 pounds (35 Kg)

**Dimensions:** 25.75 "L x 23.50 "D x 36.60 "H  
(654.0 mm L x 596.9 mm D x 929.6 mm H)

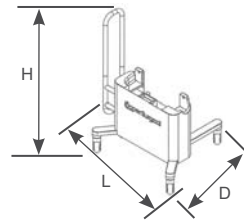## Explanation of Symbols

Reorder number

Batch code

Caution

Consult instructions for use

Not made with natural rubber latex

Authorized Representative in the European Community.

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© 2018 CooperSurgical, Inc.