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About this manual

Using this manual

This manual is not an instruction manual, but instead provides general information regarding disassembly, cleaning, sterilization and assembly of urologic instrument sets from COOPERSURGICAL, INC.. This manual presumes knowledge of the "Cleaning, sterilization and care of COOPERSURGICAL, INC. instruments". This manual is where the various cleaning and sterilization methods are described in detail.

Particularly sensitive product groups, such as telescopes or fiberscopes, have their own manuals. Separate instruction manuals and service manuals are available for units. Other general regulations (Normal use, User qualification, Safety precautions when using instruments, etc.) must be heeded.

Note: For the most part, all of the methods described in the section 'General notes on cleaning and sterilization' can be used for the instrument set described in these instructions. Where this is true, the suitable methods are no longer specified separately in the descriptions of the individual instruments.

When certain methods (e.g., ultrasound cleaning, steam sterilization) are not suitable for particular instruments, the necessary restrictions are specified in the descriptions of the individual instruments.

In addition, a table can be found at the end of each section giving an overview of the individual instruments and the preparation methods suitable for each.

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Symbols and hazard information

Please read this manual and follow its instructions carefully. The words **Warning**, **Caution** and **Note** convey special meanings. Wherever they are used in this manual, they should be carefully reviewed to ensure the safe and effective operation of this product. To make the words **Warning** and **Caution** stand out more clearly, they are accompanied by a pictogram.

Warning: A Warning indicates that the personal safety of the patient or physician may be involved. Disregarding a Warning could result an injury to the patient or physician.

Caution: A Caution indicates that particular service procedures or precautions must be followed to avoid possible damage to the instrument.

Note: A Note indicates special information about operating the instrument or clarifies important information.

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Safety information

Warning: Make certain before every application that the instruments are in perfect condition. If there is any sign of damage, the instrument must not be reused.

Warning: These instruments described in this manual are delivered **unsterilized** and must therefore be cleaned, disinfected and/or sterilized before the initial application as well as before any further use.

Warning: During the application, the tip of the instrument must remain in the viewing field of the user.

Warning: The listed cleaning, disinfection and sterilization methods alone do not guarantee successful sterilization. This can only be achieved if the staff follows recognized and validated preparation methods.

Warning: For all cleaning of contaminated endoscopes and instruments, liquid-proof protective gloves must be worn. When using a compressed air gun, safety goggles are necessary. Aerosols should be removed using a suction device.

Warning: Remember that sterilization is only then successful if

- the instruments have been freed of all organic materials and chemical residues. Only clean, disinfected surfaces can be reached by the sterilization.
- work proceeds according to recognized and validated methods.
- the recommended sterilization parameters are maintained. These are only valid with sterilization equipment that is properly maintained and calibrated.

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Caution: Use of preparation methods other than those listed here may have a negative effect on the function and service life. All damages resulting from this are not covered by the guarantee.

Caution: The following information on cleaning and sterilization only applies to the thermostable instrument set described in these instructions. For HOPKINS telescopes and fibroscopes, special rules apply (see instructions for “telescopes” and “fibroscopes”).

Caution: For the production and application of solutions, precisely observe the information from the manufacturer regarding mixing ratios and immersion times. A maximum immersion time of 60 minutes, including sterile water, must not be exceeded. Prolonged immersion can lead to damages.

Caution: Constant change between various preparation methods is especially hard on the materials and must therefore be avoided. A one-time change of method, due to procurement of new cleaning and sterilization equipment for example, is however possible.

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Observe the guidelines of the Employer's Liability Insurance Association and equal ranking organizations.

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General instructions on cleaning and sterilization

The medical device cycle shown below illustrates the sequence of steps for optimal instrument preparation:

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According to the instrument cycle, preparation of the instruments described here generally comprises the following 6 steps:

 **Caution:** The instructions for cleaning and sterilization in the manual 'Cleaning, sterilization and care of COOPERSURGICAL, INC. instruments' must be observed. The methods for cleaning, disinfection and sterilization are explained there in detail.

Note: Recommended cleaning accessories and sterilization containers can be found in the catalog chapter 'Cleaning, sterilization and care'.

1. Manual preliminary cleaning

The instruments may require manual preliminary cleaning on account of their design (e.g. slits and long, narrow lumens) and depending on the type and degree of soiling.

In addition, a prolonged waiting time before preparation (which allows the contamination to dry on) may make preliminary cleaning necessary while still in the operating room. Organic residues and pharmaceutical products, for example, must be removed mechanically (by brushing, rinsing through with a water gun, etc.), and the instruments must subsequently be rinsed off with completely demineralized or microbiologically pure/sterile water.

Used and contaminated instruments should always be prepared with the minimum possible delay.

Separable instruments must be disassembled following the instructions. Articulating instruments and scissors must be treated whilst open. Tissue remnants sticking or baked to the jaws must be removed with a cleaning brush, and the instrument must be pre-cleaned with a 3% H₂O₂ solution as soon as possible after use. Special care must be taken when cleaning the jaws, as they are precision-engineered parts.

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2. Ultrasound treatment

All non-separable scissors, coagulation instruments and microinstruments may be pre-cleaned with ultrasound (3 - 5 minutes exposure time). Instruments can be cleaned either ultrasonically or manually. The cleaning effect of the ultrasound bath reaches even inaccessible hollow spaces. For this reason, all non-separable scissors and forceps, with or without irrigation channel, rail-sheath instruments, aspirators, circular punches, coagulation instruments and microinstruments must be preliminarily cleaned with ultrasound. Ultrasound has no cleaning effect on soft plastic parts because the oscillations are attenuated. Hollow instruments (aspirators etc.) should be held at an angle in the ultrasound bath so that the hollow spaces can fill. Position large instruments in the bath so that no acoustic shadows are formed. The following items, amongst others, are not suitable for cleaning in an ultrasound bath: all rigid and flexible telescopes, electrical devices, cables, prisms, rubber articles, magnetic discs and fiberoptic light cables. After ultrasound treatment, rinse thoroughly with completely demineralized water. Place the instruments in a suitable instrument carrier (e.g. a tray holder).

Note: If manual preliminary cleaning is mandatory, the necessary steps are explained in this manual for each instrument.

3. Manual/machine cleaning and disinfection

For preparation, multi-part instruments must be dismantled, and seals, cock plugs and protective caps removed.

Manual cleaning/disinfecting cleaning

For manual cleaning or disinfecting cleaning, use active-cleaning, non-protein-fixing chemicals, where necessary with a verified range of action (DGHM (German Association of Hygiene and Microbiology) listing or comparable).

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Caution: When preparing and using the solutions, the manufacturer's instructions concerning dilution and immersion time must be strictly adhered to. Prolonged immersion can lead to damage.

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When immersing instruments in cleaning and disinfectant solutions, ensure that air can escape from hollow spaces so that all the internal surfaces are completely wetted. It is best to immerse the instruments at an angle until no more air bubbles can be seen. During cleaning, always ensure that all hollow spaces (sheath lumens, stopcocks, valves etc.) are also cleaned and that there are no obstructions (flow test, for example with a syringe).

Thoroughly clean surfaces, interiors and jaws with a suitable soft brush, sponge or cleaning gun, and rinse. Take care when cleaning small parts.

After cleaning, rinse the instruments copiously with microbiologically pure/sterile water and dry with sterile compressed air.

Thermal cleaning and disinfection in a machine

As far as possible, thermal cleaning/disinfection should be carried out in a machine, followed by steam sterilization at 134°C for 4 minutes, with a minimum dry time of 15 minutes. The advantage of preparation by machine is that it reduces the risks for personnel and standardizes the cleaning procedure (validatable or validated processes are demanded).

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Miniature sheaths

Clean the outer surfaces of the sheaths using a sponge.

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Clean the sheath lumen with a flexible brush (see accessories).

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Zu- und Abflußstutzen durchbürsten.
Brosser les raccords d'entrée et de sortie.

Clean the inlet and outlet nozzles with a brush.

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Disassemble the stopcocks for further cleaning.
Loosen the thumbscrew on the stopcock, remove
cock plug.

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Clean the valves thoroughly with a brush.

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Flush through with water and dry with sterile
compressed air.

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Pediatric Urology Cystoscopes, Resectoscopes and Urethrotomes

Assembly

Lightly grease the contact surfaces on the cock plug (1) (special grease for stopcocks).

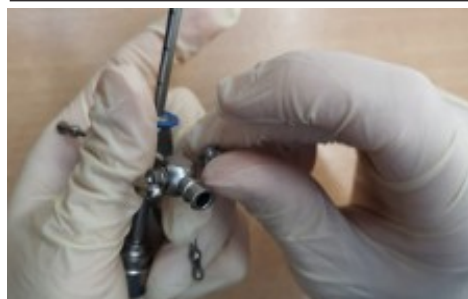
Note: This thin layer of grease not only serves to lubricate the metal-to-metal surfaces in contact with one another, but also prevents the stopcocks from leaking. Oil is not suitable as a sealant.

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Insert the cock plug and fix in place with the thumbscrew. Tighten slightly.

Test the cock plug for proper operation.

Take apart the cock plug again for sterilization.

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Miniature working elements

To change the electrode, press down the side locking button (locking mechanism) and pull out the electrode towards the front.

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Clean the working element with a brush.

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Clean the lumen with a flexible brush.

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Flush through with water and dry with sterile compressed air.

To clean the loop holder the instrument is rinsed out through HF cable port and then dried with compressed air.

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Oil the moving parts.

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Defekte Dichtung
Defective seal
Junta defectuosa
Joint défectueux

Original
Original
Original
Original

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Seals

Although high quality plastics are used almost exclusively in endoscopic instrument sets, these are also subject to a natural aging process which is accelerated by temperatures above 80 °C, expansion during storage as well as by light (UV rays). Usually, discoloration, brittleness or hardening occurs. Penetration of liquids or gases may sometimes lead to irreversible damage due to the occurrence of swelling.

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Undo the knurled screw and remove the rear handle part from the groove of the forceps insert.

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Pull off the front handle part towards the rear.
Then clean in an ultrasonic bath.
Flush through with water and dry with sterile compressed air.

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Then oil the joint of the forceps jaws, holding the instrument so that the oil can flow into the jacket and thus also wet the wire pull.

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Cooper Surgical delivers only tested products to your customers. All our products are designed and manufactured to meet the highest quality standards. Cooper Surgical as the distributor of the products excludes any warranty claims and assumes no liability for direct or consequential damages resulting from:

- misused use
- improper use, application or handling
- improper preparation and sterilization
- improper maintenance and repair
- disregard of the instructions for use

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