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Symbol Descriptions: English

-  In this document, indicates conditions or practices that could lead to illness, injury, or death (“WARNING”).
-  In this document, indicates conditions or practices that could damage the equipment or other property (“CAUTION”).
-  Consult Instructions for Use
-  Do not re-use, Single use device
-  Do not re-sterilize
-  Catalogue Number
-  Lot Number
-  Use By Date
-  Manufacture Date
-  Quantity
-  Manufacturer's contact information
-  Not made with natural rubber latex
-  Fragile
-  ^{35°C} Temperature limit ^{5°C}
-  ^{85%} Humidity limitation ^{45%}
-  ^R_{Only} For Professional Use
-  Do not use if package is damaged

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Indications for Use

The single-use lighted anal speculum is intended for physician use to examine the anal sphincter, anus, rectum, and, using additional devices, to perform various diagnostic and therapeutic procedures.

-  **Warnings and Cautions**
WARNING: Inspect each device for damage prior to use
WARNING: If a device is damaged, do not use any device from the same box.
WARNING: When in transit or storage, device may be subject to damage beyond the control of the manufacturer or supplier.
WARNING: Never use device with laser equipment.
WARNING: Treat used devices as bio hazardous infectious material. Dispose of used device in suitable disposal unit or in accordance with local regulations.
WARNING: Devices are not compatible with gamma radiation or autoclave sterilization and should never be re-sterilized by any method.
WARNING: No modification of this equipment is allowed
WARNING - Risk of reuse: The strength of the device may be significantly compromised and the resulting risk to the patient and/or user is unknown.
The light may not function properly.
CAUTION: Use by trained personnel only.
CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician.
CAUTION: Light source is single-use and of limited duration. Do not activate light source until you are ready to use the device.

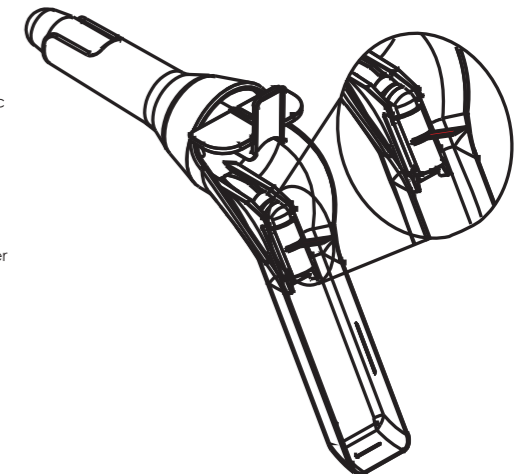
Warranty
 Please visit www.obpmedical.com “Terms and Conditions” page for warranty information. OBP Medical should be notified immediately via phone or email of any defective units.

obp™
 simple, bright ideas.

ANOSPEC® Single-Use Anoscope
 with Integrated LED Light Source

REF C060001 thru C069999

 **OBP Medical**
 360 Merrimack Street
 Building 9
 Lawrence, MA 01843 USA
 + 1-978-291-6853
www.obpmedical.com

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands

Australian Sponsor:
 Emergo Australia
 201 Sussex Street
 Darling Park, Tower II,
 Level 20
 Sydney, NSW 2000
 Australia

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