




FLUID MANAGEMENT PACK

Remove Fluid Management Pack contents from tyvek bag and place on sterile surface. Unfold under buttocks drape and remove covering from sticky tape on the top underside of the drape. Stick under buttocks drape firmly to patient table allowing drape to hang down with collection bag facing outward (see example "A" below).

Example A

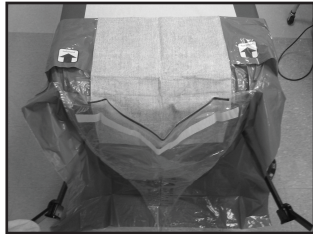

Attach outflow bag tubing to the outflow port of hysteroscope. Place collection bag on floor or in position that allows fluid to flow downward (see example "B" below).

Example B



Attach cysto tubing to inflow port of hysteroscope using luer-lock adaptor. Spike fluid bag with opposite end of tubing to allow fluid to flow.

PL-05-0102 REV C

尺寸: 148.5*105mm

Symbol Descriptions



In this document, indicates conditions or practices that could lead to illness, injury, or death ("WARNING").



In this document, indicates conditions or practices that could damage the equipment or other property ("CAUTION").



Consult Instructions for Use



Do not re-use, Single use device



Catalogue Number (Reorder #)



Lot Number



Use By Date



Manufacture Date



For Professional Use



Do not use if package is damaged



Sterilized using ethylene oxide (only applicable to versions marked as sterile on unit packaging)



Temperature limit



Humidity limitation



Quantity



Manufacturer's contact information



Not made with natural rubber latex



Fragile



Warnings and Cautions

WARNING: Inspect each product for damage prior to use
WARNING: When in transit or storage, product may be subject to damage beyond the control of the manufacturer or supplier.

WARNING: Treat used product as bio hazardous infectious material. Dispose of used product in suitable disposal unit or in accordance with local regulations.
WARNING: Product is not compatible with gamma radiation or autoclave sterilization.

WARNING: No modification of this equipment is allowed



CAUTION: Use by trained personnel only.
CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician.

Warranty

Please visit www.obpmedical.com "Terms and Conditions" page for warranty information.

OBP Medical should be notified via phone or email of any defective product and the item should be securely packaged and immediately returned with prepaid postage from OBP Medical

Manufacture & Sterilization

Product is manufactured in accordance with FDA GMP Quality System Regulations, 21 CFR, Part 820.

Product is sterilized using ETO

PL-05-0102 REV C

对应客户图纸编号: PL-05-0102-C

▲			数量		名称		冲洗引流包说明书		
▲			比例		1:1		图纸编号		
▲			日期		变更理由		担当		
设计			校对		确认		材料		
马李							70g白纸		
2016.3.2							颜色		
						第1张		共1张	
杭州富善医疗器械有限公司									

Rev	Release Date	CR #	Originator	Summary of Change
A	09/08/10	n/a	M Traub	Initial Release
B	09/14/16	16-0030	JJ Swift	Update format & add symbol descriptions
C	06/11/18	18-0001	M Traub	Updated logo and font on page 1