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Symbol Descriptions: English

	In this document, indicates conditions or practices that could lead to illness, injury, or death („WARNING“).
	In this document, indicates conditions or practices that could damage the equipment or other property („CAUTION“).
	Consult Instructions for Use
	Do not re-use, Single use device
	Do not resterilize
	Catalogue Number
	Lot Number
	Use By Date
	Manufacture Date
	Quantity
	Manufacturer's contact information
	Not made with natural rubber latex
	Fragile
	Temperature limit
	Humidity limitation
	For Professional Use
	Do not use if package is damaged

Indications for Use

The single-use lighted anal speculum is intended for physician use to examine the anal sphincter, anus, rectum, and, using additional devices, to perform various diagnostic and therapeutic procedures.

Warnings and Cautions

- WARNING:** Inspect each device for damage prior to use
- WARNING:** If a device is damaged, do not use any device from the same box.
- WARNING:** When in transit or storage, device may be subject to damage beyond the control of the manufacturer or supplier.
- WARNING:** Never use device with laser equipment.
- WARNING:** Treat used devices as bio hazardous infectious material. Dispose of used device in suitable disposal unit or in accordance with local regulations.
- WARNING:** Devices are not compatible with gamma radiation or autoclave sterilization and should never be resterilized by any method.
- WARNING:** No modification of this equipment is allowed
- WARNING – Risk of reuse:**
- The strength of the device may be significantly compromised and the resulting risk to the patient and/or user is unknown.
- The light may not function properly.
- CAUTION:** Use by trained personnel only.
- CAUTION:** U.S. federal law restricts this device to sale by or on the order of a physician.
- CAUTION:** Light source is single-use and of limited duration. Do not activate light source until you are ready to use the device.

Warranty

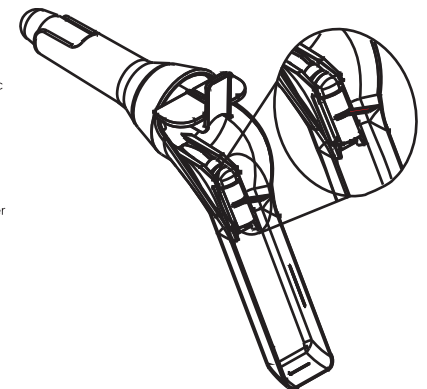
Please visit www.obpmedical.com “Terms and Conditions” page for warranty information. OBP Medical should be notified immediately via phone or email of any defective units.

obpTM
simple, bright ideas.

ANOSPEC® Single-Use Anoscope
with Integrated LED Light Source

REF C060001 thru C069999

 **OBP Medical**
360 Merrimack Street
Building 9
Lawrence, MA 01843 USA
+ 1-978-291-6853
www.obpmedical.com

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