
**CTI-1012P, CTI-1015P, CTI-15P**  
Carter-Thomason® II Port Closure System

**CTXL-P**  
Carter-Thomason® II Port Closure System XL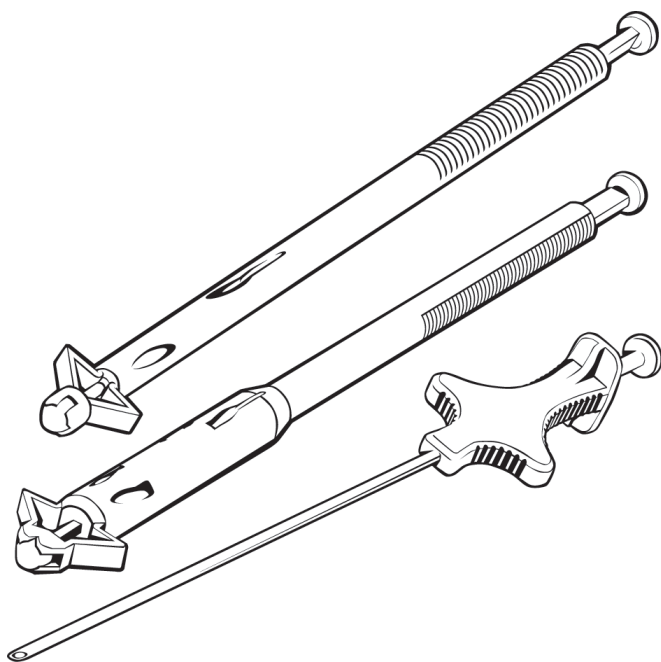

Instructions for Use (English)  
Gebrauchsanweisung (Deutsch/German)  
Instrucciones de uso (Español/Spanish)  
Notice d'utilisation (Français/French)  
Istruzioni per l'uso (Italiano/Italian)

English..... 3

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Español/Spanish..... 9

Français/French..... 12

Italiano/Italian ..... 15

# CTI-1012P, CTI-1015P, CTI-15P Carter-Thomason® II Port Closure System CTXL-P • Carter-Thomason® II Port Closure System XL

## Instructions for Use (English)

### DEVICE DESCRIPTION

The Carter-Thomason II Port Closure System is available in four kit configurations consisting of the following components:

#### CTI-1012P:

- One (1) Suture Passer
- One (1) 10/12 mm Suture Guide

#### CTI-1015P:

- One (1) Suture Passer
- One (1) 10/12 mm Suture Guide
- One (1) 15 mm Suture Guide

#### CTI-15P

- One (1) Suture Passer
- One (1) 15 mm Suture Guide

#### CTXL-P:

- One (1) XL Suture Passer
- One (1) 10/12 mm XL Suture Guide
- One (1) 15 mm XL Suture Guide

The Suture Passer is a sterile, single-case use, hand-held suture grasping device that is designed to pass suture through soft tissue.

The Suture Guide is a single use device that is designed to work with the Suture Passer for placement of the suture through soft tissue.

### CAUTION

- U.S. Federal law restricts this device to sale by or on the order of a physician.
- Dispose of in accordance with all applicable Federal, State, and local Medical / Hazardous waste practices.

### INDICATION FOR USE

The Suture Passer is intended to pass suture through soft tissues of the body during endoscopic/laparoscopic surgery.

It is to be used only by surgeons trained in endoscopic/laparoscopic surgery.

### PRECAUTIONS

- A surgeon should not begin clinical use of the Suture Passer without a full understanding of the directions for use.
- The Suture Passer is not recommended for use in surgical procedures where the position of the needle tip cannot be clearly determined.
- The grasping fingers of the instrument must be closed completely in order to form a needle point to pass suture through tissue. Release button on the Suture Passer to draw suture completely into mechanism.
- The Suture Passer needle point can injure internal tissues and/or any personnel in contact with the pointed end of the instrument. The Suture Passer needle point should be protected at all times with the protective cap when the instrument is not in use.

### CARTER-THOMASON II SUTURE PASSER INSTRUCTIONS FOR USE

The Suture Passer has two operating positions: Fingers Open and Fingers Closed (see illustrations).

1. To suture, open the finger grasping mechanism by pressing on the plunger on the handle of the guide. We recommend the use of Vicryl Size 0 braided sutures (CSI approved suture material via validations).
2. Place suture in the finger mechanism.
3. Close the grasping fingers by releasing the plunger.
4. Pass the needle point with the suture through tissue by pushing distally on the handle.
5. Open the grasping fingers to drop the suture.
6. Close the grasping fingers and remove the Suture Passer.
7. With the grasping fingers in the closed position, reinsert the Suture Passer in proximity to the first entry point.
8. Open the grasping fingers to retrieve suture by pressing on the plunger on the passer handle.
9. Close the grasping fingers by releasing the plunger.
10. Remove the Suture Passer with the suture in place.


**Grasping Fingers Open**


**Grasping Fingers Closed**


# TROCAR WOUND CLOSURE SURGICAL TECHNIQUE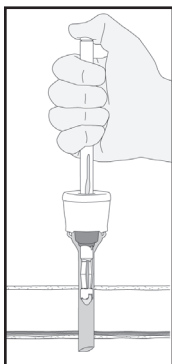


**Step 1**  
Press the plunger on the Suture Guide and place through the trocar.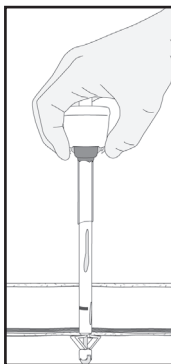


**Step 2**  
Once the Suture Guide anchoring wings are visible below the peritoneum and extending beyond the trocar, release the plunger. Remove the trocar.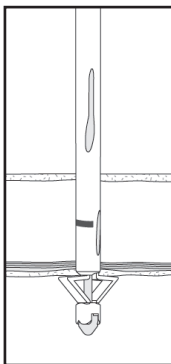

**Step 3**  
Ensure that the suture entry holes are aligned perpendicular to the trocar wound with anchoring wings resting against the peritoneum.\*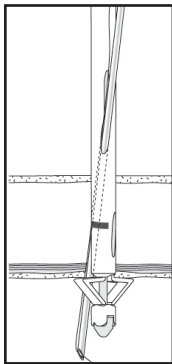


**Step 4**  
With the Suture Passer loaded and the grasping fingers in the closed position, push suture through Suture Guide, fascia, muscle, peritoneum, and into the abdomen. Drop suture and remove Suture Passer.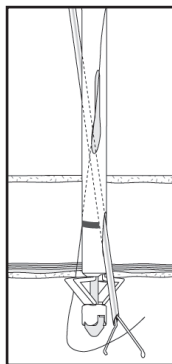


**Step 5**  
With the grasping fingers in the closed position, push Suture Passer through opposite side of the Suture Guide and pick up suture. Drop suture and remove Suture Passer.

\*Insure that the blue depth marking ring is below or at the skin level before proceeding with Step 4. If the marking above the skin level, move the suture guide down into the defect aligning the ring at the skin level or slightly below and then proceed with Step # 4.


## 15 mm Suture Guide

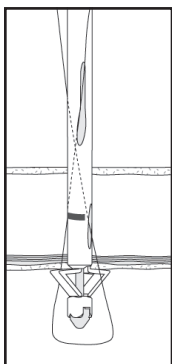
**Step 6**  
Pull the suture up through the peritoneum, muscle, fascia, & Suture Guide.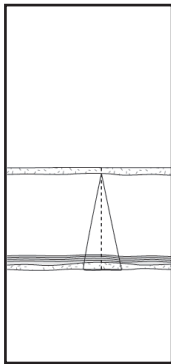


**Step 7**  
Remove Guide by depressing the plunger and withdrawing. Tie suture to complete the closure.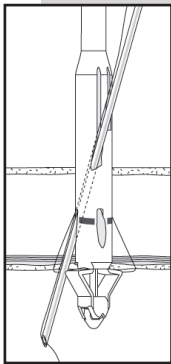

**If using the 15 mm Suture Guide:**  
Perform Steps 4-5 using the blue entry holes.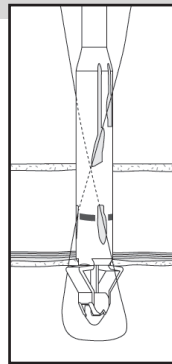

With another piece of suture, repeat Steps 4-5 using holes colored orange.

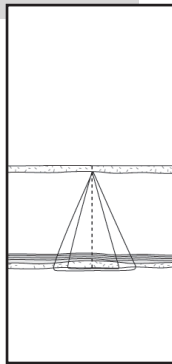Remove Suture Guide. Tie each suture to complete the closure.

## EXPLANATION OF SYMBOLS

Reorder Number

Batch Code

Use-by date

Sterilized Using Ethylene Oxide. Contents sterile unless package has been opened or damaged. Do not use if sterile barrier has been breached.

Consult instructions for use

Do not re-use due to risk of septicemia

Caution

Do not use if package is damaged

**RxOnly**

U.S. Federal law restricts this device to sale by or on the order of a physician.

Authorized Representative in the European Community.

Manufacturer

Not made with natural rubber latex.

Do not resterilize

Patents: Refer to [www.csipatents.com](http://www.csipatents.com)

Carter-Thomason® is a registered trademark of CooperSurgical, Inc.

CooperSurgical® is a registered trademark of CooperSurgical, Inc.

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© 2021 CooperSurgical, Inc.
Carter-Thomason® è un marchio registrato di CooperSurgical, Inc.
CooperSurgical® è un marchio depositato di CooperSurgical, Inc.