How to use and take care of your

English

Rx Only

CE 2797

CooperSurgical

MXWS+Size Arcing Diaphragm
MXWF+Size Omniflex Diaphragm

DEVICE DESCRIPTION
The Wide-Seal Silicone Vaginal Diaphragm is a shallow cup with a flexing rim. When the diaphragm is coated with spermicidal gel and placed over the cervix (the opening of the uterus), it prevents sperm from entering the uterus and provides a reservoir for the spermicidal gel or cream. The diaphragms are available in Arcing or Omniflex styles.

AVAILABLE SIZES
60 mm, 65 mm, 70 mm, 75 mm
80 mm, 85 mm, 90 mm and 95 mm.
Read the entire sheet carefully, even if you have used a diaphragm in the past. If you have additional questions, ask your doctor or healthcare specialist.

DIAPHRAGM PLACEMENT
The diagram shows the vaginal walls, the cervix (protruding slightly into the vaginal vault) and, above it, the pubic notch. When the diaphragm is properly fitted and inserted, it fits over the cervix and is tucked into the pubic notch and against the rear vaginal wall.

WARNINGS

- Keep your diaphragm and spermicide out of the REACH of CHILDREN.
- If you are using any vaginal medication check the labeling and/or check with your doctor or healthcare specialist to assure there is no interaction with the diaphragm.
- All methods of contraception must be used according to directions; however, no method can guarantee 100% results against pregnancy.
- Daily use of spermicide can increase the risk of contracting STDs.

CONTRAINDICATIONS

- An association has been reported between diaphragm use and Toxic Shock Syndrome (TSS), a serious condition which can be fatal.
- Known sensitivities to silicone and/or prior history of Toxic Shock Syndrome.
- High-risk HIV women or those living with HIV and/or receiving antiretrovirals are contraindicated for this form of contraceptive.

Wash diaphragm with mild soap and thoroughly rinse prior to initial use.

CONTRAINDICATIONS (continued)

- Women with pelvic organ prolapse are contraindicated for this form of contraceptive.

IMPORTANT—Continuous prolonged wearing of the diaphragm may encourage the growth of certain bacteria in the vagina. These bacteria can lead to symptoms of Toxic Shock Syndrome; get sore throat, red eyes, and general feeling of weakness.

SYMPTOMS OF TOXIC SHOCK SYNDROME (TSS)

- Sudden high fever (102 °F / 39 °C, or over)
- Nausea, vomiting and/or diarrhea
- Fainting, feeling faint or dizziness when standing up
- Rash over much of body (may resemble sunburn)
- Weakness, tenderness and/or aching or pain of muscles and/or joints
- Sore throat, red eyes, and general feeling of weakness

If you have a sudden high fever and one or more of the above symptoms remove the diaphragm and call your doctor or other healthcare professional immediately.

INSTRUCTIONS FOR USE

PREPARING THE DIAPHRAGM FOR INSERTION

- With clean hands, place the previously washed diaphragm on a flat surface with the dome of the cup down. Add spermicidal gel or cream into the cup of the diaphragm.
- With your forefinger, spread the gel around the inside of the cup, under and around the surface of the silicone lip. Applying a thin film of gel or cream on the first entering lip of the diaphragm makes it easier to insert and helps to hold it securely in place. Remember, in order to be effective, your diaphragm must always be used with spermicidal gel or cream.

INSERTING THE DIAPHRAGM

- The diaphragm can be inserted while standing with one leg on a low chair or stool, lying down, squatting, or sitting forward on the edge of a chair. Remember, however, that the position of the cervix and vaginal vault changes with each position that you take, so always be certain that the cervix is covered.

- Hold the diaphragm with the dome down. Arcing Diaphragms are folded for insertion by pressing the two notches in the diaphragm lip together. Omniflex Diaphragms can be folded for insertion at any point. Now spread the lips of the vagina with one hand and gently insert the diaphragm (dome down) into the vagina along the rear wall as far as it will comfortably go. Your index finger can guide the diaphragm as you gently push it into place. Do not pinch the diaphragm.


POSITIONING THE DIAPHRAGM

- Tuck the front rim of the diaphragm up into the pubic notch so it's pressed against the front wall of the vagina.

- As an added precaution, feel the cervix through the silicone shield (it is shaped like the tip of a dented nose) to make certain that the diaphragm is securely in place and that the entire cervix is covered.

- Do not douche or remove the diaphragm for at least six hours after intercourse. The diaphragm should be removed as soon after this as possible. Removal of the diaphragm before six hours may increase the risk of pregnancy. Additional gel or cream should be applied if you desire to have intercourse again.

The diaphragm should never be left in place continuously, over 24 hours.

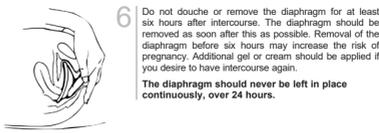- To reapply spermicidal gel or cream squeeze contents onto one or two fingers. Reapply as shown with each additional intercourse episode. At this time, make sure the diaphragm covers the cervix. This added precaution should be followed with all applications of the spermicide.

REMOVING THE DIAPHRAGM

To remove the diaphragm, insert your index finger, behind the front rim to break the suction. With your finger inside the rim pull the diaphragm down and withdraw it, but be careful not to puncture the silicone dome with your fingernail. Do not pinch the diaphragm—simply pull it gently with your finger.

CLEANING INSTRUCTIONS

After each use, follow the cleaning instructions provided below for the Arcing and Omniflex Diaphragms.

1. CLEANING

Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.

2. SOAK AND SCRUB

- Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
- Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent contamination of contaminants.
- Following scrub, inspect device for visible soil residue.

3. RINSE

- Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
- Allow the device to dry.

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RECOMMENDATION
When you have a routine physical examination, your doctor may find it necessary to reft you. Physiological changes may require a different size Arcing or Omniflex diaphragm than you initially needed.

CONTACT YOUR DOCTOR OR HEALTHCARE SPECIALIST
If you have questions or uncertain about the insertion, placement and use of your diaphragm.

If the diaphragm no longer fits snugly and slips out of place when you cough or walk.

If you have gained or lost more than 6.61 pounds (3 kg).

If you have had a baby, an abortion or vaginal surgery.

If you notice any signs of deterioration in the diaphragm.

If there is blood on the diaphragm when it is removed (other than during menstruation).

If the diaphragm is felt by you or your partner during intercourse.

If you are unable to remove the diaphragm.

NOTE: CooperSurgical has followed a policy of supplying its diaphragms only to Licensed Practitioners.

Pregnancy Rates for Birth Control Methods (For One Year of Use)
The following table provides estimates of the percent of women likely to become pregnant while using a particular contraceptive method for one year. These estimates are based on a variety of studies.

Contraception Method	6-month Pregnancy Probability	12-month Pregnancy Probability
Surgical Sterilization Injectable Hormones IUDs	Less than 1%	Less than 1%
Hormone Pills Vaginal Ring	1-2%	1-2%
Male Condom	7%	11%
Contraceptive Diaphragm	8%	17%
Cervical Cap	11%	17%
Female Condom	13%	21%

* These 1-year probabilities are projected since most barrier studies today are conducted as 6-month studies.

Record the date the diaphragm is first used on the space provided on the compact label supplied with the diaphragm. It is advisable to replace your diaphragm every 2 years or if indications of wear are observed.

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Dawn® est une marque déposée de Procter & Gamble Company. CooperSurgical est une marque déposée de CooperSurgical, Inc.