
CooperSurgical[®] Lone Star[®] Retractor System

INSTRUCTIONS FOR USE Retractors and Elastic Stays

C E 2797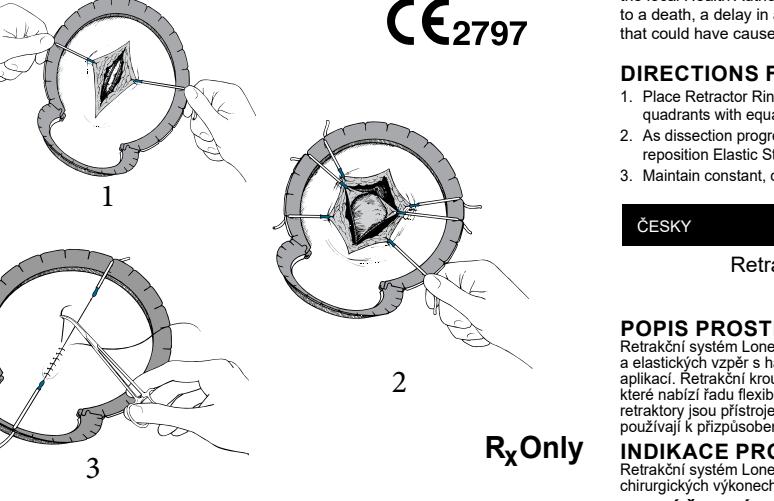

ENGLISH

Lone Star[®] Retractor System - Instructions for Use
Retractors and Elastic Stays

DEVICE DESCRIPTION / INTENDED USE
The Lone Star Retractor System is comprised of a variety of adjustable, self-retaining Retractor Rings and Elastic Stay hooks that offer access and visualization across a wide range of surgical applications. The Retractor Rings are available in both single-use and reusable configurations that offer a range of flexible, articulating shapes appropriate to the surgical site. The single-use retractors are disposable devices. The Elastic Stays are single-use, disposable devices that are

INDICATIONS FOR USE
The Lone Star Retractor System is indicated for use to provide incision retraction and to aid in visualization during surgical procedures.

INTENDED USERS
The Retractor system is used by healthcare professionals, nurses, or technicians in a professional environment such as a hospital, clinic, or doctor's office. Patients are selected based on their surgical needs and the need of the medical healthcare professionals for incision retraction and to aid in visualization during surgical procedures.

PATIENT POPULATION
The Retractor system is intended to be used by healthcare professionals on human patients. The patient population includes patients undergoing surgical procedures in a professional environment such as a hospital, clinic, or doctor's office. The system is used during general operations to retract tissue in the surgical wound site with no restriction on age or gender.

WARNINGS
• For single use products only. Do not reuse, reprocess, or re-sterilize. Reuse, reprocessing or re-sterilization may compromise the structural integrity of the device and/or lead to device failure which, in turn, may result in patient injury, illness or death. Reuse, reprocessing or re-sterilization may create a risk of contamination of the device and/or cause patient infection or cross-infection. Contamination of the device may lead to injury, illness or death of the patient.
• Do not use if sterile barrier is damaged.
• Do not use if the device has been opened or damaged.

- For reusable retractors, both aluminium and stainless steel, they should be thoroughly cleaned and sterilized to prepare them for use. Reusable instruments must be inspected each time they are processed to determine when the end of serviceable life has been reached and can no longer be safely used. **Refer to Reprocessing Instructions.**
- Dispose of in accordance with all applicable Federal, State and local Medical/Hazardous waste practices.

PLEASE NOTE: If a serious incident is suspected from using the Lone Star Retractor System, report the details of the incident to CooperSurgical via phone number +1 203 601-5200 Ext. 3100 or by email to ProductSurveillance@coopersurgical.com and to the local Health Authority in your country. A serious incident may have caused or contributed to a death, a delay in a procedure which resulted in death or serious injury, or a malfunction that could have caused an adverse event.

DIRECTIONS FOR USE

- Place Retractor Ring around surgery site. Connect Elastic Stays in directly opposing quadrants with equal tension to hold Retractor Ring in place.
- As dissection progresses, advance position of Elastic Stays into deeper tissue. Add or reposition Elastic Stays to desired angle for balanced retraction.
- Maintain constant, opposing tension of Elastic Stays while suturing.

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