LEVER Folding Pessaries


a CooperSurgical Brand

CE2797

For stress uterine incontinence

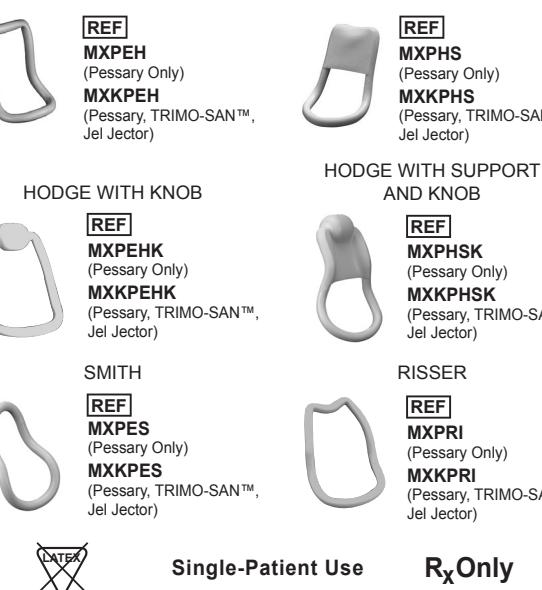AVAILABLE MODELS

Wash pessary with mild soap and thoroughly rinse prior to initial use.

DESCRIPTION

The Milex® LEVER Pessaries are a medical device made from silicone that is inserted into the vagina to function as a supportive structure of the uterus, bladder and/or rectum.

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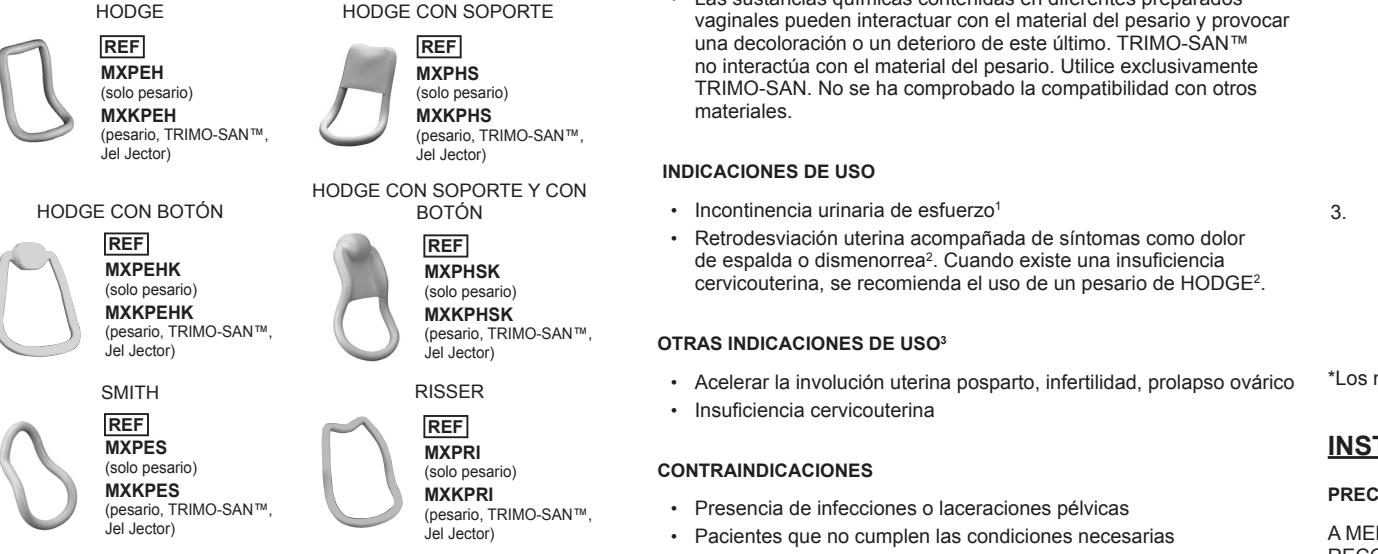
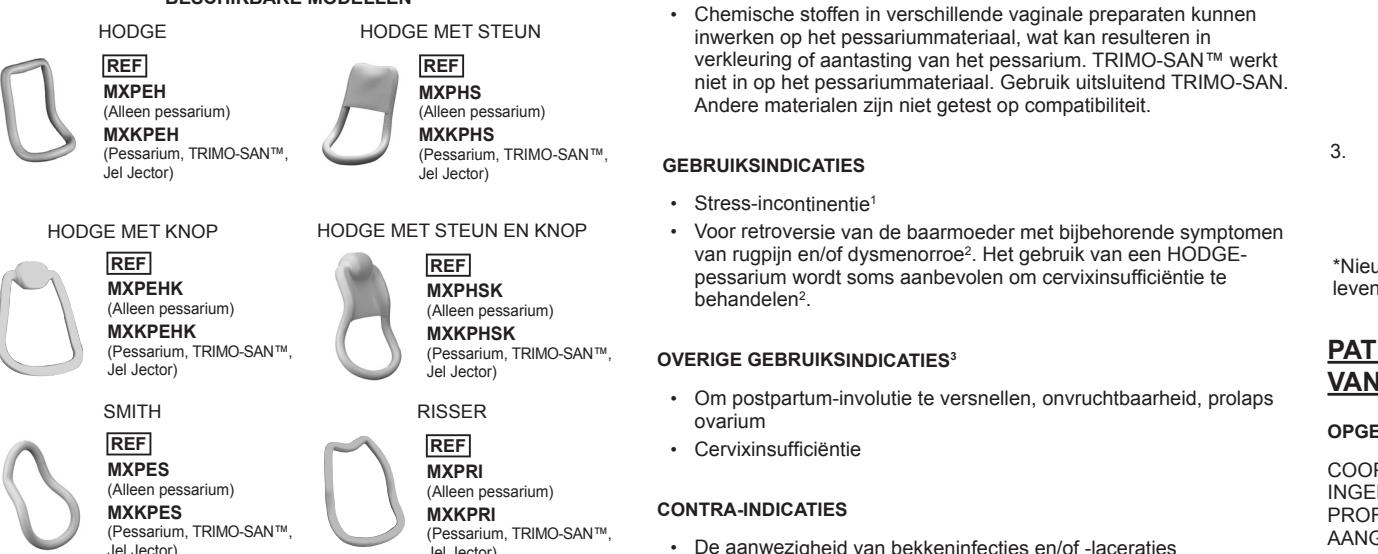
CooperSurgical

WARNINGS

- These pessaries contain a wire coil which must be removed before X-ray or MRI.
- Do not leave pessaries in place for long periods of time, as serious complications can occur which may require surgical intervention.
- Do not use these pessaries on a patient with a known silicone allergy.
- Chemicals in various vaginal preparations can interact with the pessary material, resulting in discoloration or deterioration of the pessary. TRIMO-SAN™ does not interact with the pessary material. Other materials have not been tested for compatibility.

INDICATIONS FOR USE

- Stress urinary incontinence.¹
- For uterine retrodisplacement with accompanying symptoms of backache and/or dysmenorrhea². The use of a HODGE pessary is sometimes recommended to manage an incompetent cervix³.

OTHER INDICATIONS FOR USE³

- To hasten postpartum involution, infertility, prolapse ovary
- Incompetent cervix

CONTRAINDICATIONS

- The presence of pelvic infections and/or lacerations
- A noncompliant patient
- Endometriosis
- Pregnant patient

¹ Miller D.S.: Contemporary use of the Pessary. Gynecology & Obstetrics, J. Sciarra & Wm Droege, JB Lippincott Co, Vol 1, 1992 pp 8-9

² Reich, W.J., Nechtow, M.J. Practical Gynecology 2nd Edition, pp 345-355

³ Oster, S., Javert, K.T., Treatment of incompetent cervix with the SMITH Pessary. Obstet and Gynec. 28: 206-208.

RECOMMENDED CLEANING INSTRUCTIONS*

- Prepare a cleaning solution by mixing a mild soap (such as Dawn® or equivalent) with tap water using the soap manufacturer's recommended concentration. Prepare this solution in a container large enough to fully submerge the device.
- Soak and Scrub
 - Soak the device in the container of prepared soap solution for a minimum of 5 minutes.
 - Following the 5-minute soak period, scrub the device for a minimum of 15 seconds with a soft-bristled brush, such as a tooth brush and/or pipe brush. Scrub device below water line to prevent aerosolization of contaminants.
 - Following scrub, inspect device for visible soil residue.
 - Rinse
 - Remove the device from the soap solution and thoroughly rinse under flowing tap water for a minimum of 30 seconds.
 - Allow the device to dry.

*New pessaries are powdered with food-grade powder.

PATIENT INSTRUCTIONS FOR PESSARY USE

CAUTION

COOPERSURGICAL RECOMMENDS PESSARIES BE INSERTED AND REMOVED BY THE PHYSICIAN OR OTHER HEALTHCARE PROFESSIONAL UNLESS OTHERWISE DIRECTED.

- Ensure the desired correction of your condition, your healthcare professional needs your full cooperation.
- It is essential that your healthcare professional inspect your vagina at frequent intervals for evidence of pressure and/or allergic reaction.
- A gradual increase in the interval of inspection may be considered at the discretion of the treating practitioner. It is important to evaluate the effects of the vaginal gel TRIMO-SAN.

TRIMO-SAN is available at the prescription counter of your pharmacy. A prescription is required.

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© 2020 CooperSurgical, Inc.